


QUICK GUIDE

Unfolding the eFOLDi Explorer Scooter

Before starting stand the scooter upright in an open space away from pets and small children.

Unfolding the scooter is straightforward, with a little practice you will be able to set it up in seconds.

We advise you to practise with the scooter before venturing out for the first time.

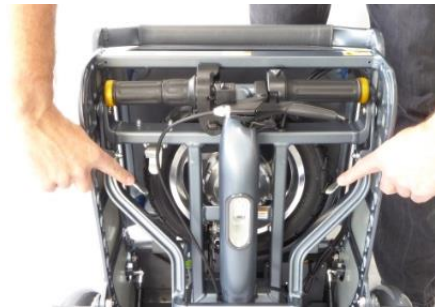

1 Locate 'B' and 'C' latches.



2 Press and hold 'B' and 'C' latches and...



3 ...pull the drag/carry handle upright.



4 Release the handlebar by pulling towards you.



5 While pushing the handlebar down and inwards release the 'E' latch.



6 Pull the steering column up and over the carry handle until it **locks** in place.



7 Release the front wheel by a sharp pull or twisting.



8 Lower the scooter to the ground holding carefully.



9 Pull the steering column upright...



10 ...and make sure the column key **locks** into place under latch 'A'.



11 Pull the seat back upright.


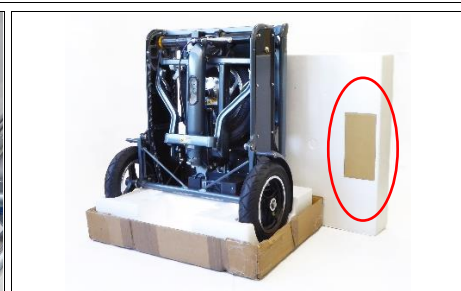



12 Push the footpegs down. Your scooter is now ready.

Folding the eFOLDi Explorer Scooter

		
<p>1 Fold the foot pegs to an upright position.</p>	<p>2 Fold the seat back down.</p>	<p>3 Press and release latch 'A'.</p>
		
<p>4 Fold the steering column flat against the seat.</p>	<p>5 Lift the scooter to an upright position.</p>	<p>6 Make sure the front wheel is flat and locked to the frame by its magnet.</p>
		
<p>7 Locate latch 'D'.</p>	<p>8 While supporting the steering column press latch 'D'.</p>	<p>9 ...and bring the steering column over the carry handle.</p>
		
<p>10 Press the steering column in to the frame until latch 'E' locks.</p>	<p>11 Press handlebars against the frame until the magnet locks them in place.</p>	<p>12 Press and hold latches 'B' and 'C' and fold the carry handle down.</p>

The eFOLDi Explorer Battery/Charger

		
<p>Remove the battery by unlocking using the key and applying gentle pressure. The battery has points for charging and for powering a mobile device.</p>	<p>Your charger is located in the base of one of the polystyrene liners.</p>	<p>Insert key and turn to power on go. Recharge battery every 3 months or when power gauge shows 24 or less.</p>