

Troubleshooting:

Problem #1: The toy/device is not activating.

Action #1: Replace the 12-V battery in the transmitter by following Operation instruction #3.

Action #2: Replace the 2 AA batteries in the receiver by following Operation instruction #2.

Action #3: Make sure that the connection to the toy/device is very tight. There should be NO gaps. Make sure that you are using a MONO adapter, if applicable.

Action #4: Try using your toy/device with a different switch in order to rule out the toy/device as the source of the problem. You may need to change the batteries in the toy/device.

Care of Unit:

The Wireless Ultimate Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Wireless Ultimate Switch - #2700

USER'S GUIDE

Our exclusive Ultimate Switch is THE most universal and versatile switch on the market. We've now made it wireless! It's designed to be used by even the most profoundly challenged or neurologically impaired individuals. It requires only a slight touch to activate (10 grams of force), yet can withstand the abuse of gross body movements. This cordless switch transmits 20 feet through the air by radio waves. Comes complete with 19" gooseneck, universal clamp, transmitter and receiver. Requires 1 12-V and 2 AA Batteries.

Operation:

1. The Wireless Ultimate Switch requires 2 AA batteries and one 12-V (not included) (**Model 23A** battery) RadioShack part No. 23-144 Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Lift and then slide the battery compartment cover off of the receiver and install 2 AA batteries according to proper polarity. Slide the cover back on. Please Note: The On/Off/Chime switch has been locked in position for this device. Do not move or force the switch.
3. To install the battery in the transmitter, which is located on the gooseneck's mounting clamp slide the battery cover off, the use of a coin or flathead screwdriver may be needed. Install the 12-V battery (- spring side first) according to proper polarity and replace the cover.

4. Plug the receiver unit into your toy/device. Make sure that the connection is tight. There should be NO gaps. If you use an adapter for devices with ¼ inch jacks, make sure that it is a MONO adapter, not stereo.
5. Mount the Ultimate Switch in the desired location/position using the universal clamp and rubber-reinforced gooseneck. **Please Note:** The effective range of this device is 15 ft.; under ideal conditions you may get up to 20 ft.
6. Actuate the Ultimate switch in order to turn on your toy/device. You can also bypass the ultimate switch and directly use the small white button right on the transmitter box to turn on your toy/device.
7. Your toy/device will only remain activated while you hold the Ultimate switch in the actuated position. Once you release pressure on the Ultimate switch (or on the white button), the toy/device will turn off.

NOTE: Use caution when not using the Wireless Ultimate Switch or when storing it. Make sure that the toggle element (the actual switch mechanism) is not actuated, therefore leaving the switch on... If it does remain actuated, the 12-V battery will die.