



Wrist Talker

**Catalog # 3037 (Blue)
3037-B (Black)**

User's Guide

Enabling Devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Wrist Talker

The slightest push on the face of this wrist communicator will provide 10 seconds of any message, which you have recorded. Great sound quality will enable the user to communicate any greeting. Easy to record and re-record. Available in Blue and Black
Size 2 ¼ " x 1 ½ " Requires 4 LR44 Button Cell Batteries (Included).

Operation:

1. Wrist Talker requires 4 LR44 Button cell batteries to operate (Included).
Please Note: Before use remove and discard the round foam disk with hole in the center. This was used to prevent battery draining during shipping.

To Record:

2. To make a recording simply twist the cover until the opening lines up with the gray rubber record button (see illustrations). Next depress the record button and speak your message from six to eight inches away from it. When finished recording release the record button. You have a maximum record time of ten seconds.

To Replace Batteries:

3. Turn the Wrist Talker over, and then carefully remove the black Velcro wrist strap (see illustrations). Next with one hand turn the unit back over, catch the white disk and voice module in your other hand as it drops out. There are three tabs on the voice module that need to be depressed in order to replace the batteries. Hold the voice module with the speaker side facing down. Next with your thumb nail press in and lift gently on the three tabs on the sides of the cove one at a time.
Please Note: On the body of the voice module you will see three grooves, just above them are the tabs you need to depress (see illustrations). Once this is done the cover should lift off. Observing proper (+) and (-) battery polarity, install four LR44 button cell batteries into the holders (+ side facing up). Replace the speaker and circuit board, making note to line up the groove in the speaker and circuit board with the groove in the white casing (see illustrations). If this is not done the unit will not go back together properly.

Message Playback:

4. To play back your message slid the cover plate back so that the microphone is showing through the opening in the cover. Next depress the center white button. If you are satisfied with your message replace the disk back into the Wrist Talker's body, with the speaker side viewable through the top of the Wrist Talker's body.
5. Replace the back plate with black rubber disk side facing into the casing, and then replace the Velcro wrist strap.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure that you have assembled all the parts correctly according to the instructions and illustration sheet.

Action #2: Check your batteries – There is a 99% chance that either one is installed incorrectly and/or one is weak/dead.

Action #3: Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries. If the unit does not work correctly, replace all the batteries paying careful attention to proper polarity.

Care of the Unit:

The Wrist Talker can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.

For Technical Support:

Call our Technical Service Department,
Monday through Friday, 9am to 5pm (EST).
1-800-TEC-TOYS or 1-800-832-8697
Or email at customer_support@enablingdevices.com