

## **More Communication Products:**

Rocking Recorder #1333

Cheap Talk 4 Direct & Jacks #1336

Cheap Talk 4 Direct #1392

Rocking Plate #1398

Rocking Say It Play It #1399

Twin Talk #1401

Twin Talk and Play #1402

One Message Recorder #1444

Vocal Traffic Light #1820

Talking Bubble #2220

Talking Bubble Busy Box #2230

Finger Isolation Talking Bubble #2260

Cheap Talk 4 Direct Inline #3054

Cheap Talk 4 Direct Inline & Jacks #3055

Cheap Talk 4 Inline Say It Play It #3056

Cheap Talk 4 Direct with Overlay Option #6054

---

---

# **Mini-Com**

---

---

## **Catalog # 9151**

### **User's Guide**

## Mini-Com

Place these communicators far apart or join them for building communication skills. Easy to use picture clip allows for you to change icons easily. Twenty seconds hi-fidelity recording time allows you to record and re-record at the touch of a button!

### **Operation:**

1. The Mini-Com requires 4 AA batteries to operate (Not included).
2. Gently turn the unit over to reveal the black battery compartment. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and slide the battery compartment cover off. Observing polarity, install four AA size batteries into the holder. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well. Replace the battery compartment cover and replace the screw.
3. Turn the unit on by turning the ON/OFF/VOLUME knob (black knob) located on the back panel of the unit clockwise. You will hear an audible click when the unit turns on.
4. To record, simply press and hold the red "RECORD" button on the back panel of the unit, while simultaneously holding down the plate switch. While continuing to hold the record button and the plate switch, speak your message. Your mouth should be about 6 inches from the microphone when recording (the Mic is the "fuzzy" black circle on the back of the unit). Release both the record button and the plate switch when you finish recording.
5. Slide a picture card or photograph in the opening of the plastic strip.
6. To play back your message, simply activate the plate switch. It is not necessary to continue holding the plate switch when listening to the message. The unit will continue to playback the entire message, even if pressure on the plate is released.
7. The unit will retain recorded information indefinitely, even if the batteries are disconnected. Previously stored information will be erased only when a new recording is performed.

8. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob located on the back panel of the unit.

### **Important Note:**

The memory of this unit is capable of holding up to a total of 20 seconds of speech information. If you go over this time, the message will record over itself.

### **Troubleshooting:**

**Problem:** The unit fails to operate, or operates incorrectly.

**Action #1:** Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries. If the unit does not work correctly, replace all the batteries paying careful attention to proper polarity.

**Action #2:** Make sure that you have the unit turned on and the volume turned up to an audible volume.

**Action #3:** Make sure that you are holding both the record button AND the plate switch SIMULTANEOUSLY while you are recording.

**Action #4:** Make sure that there is nothing covering the Mic

**Problem:** The unit is playing old messages fine, but will not record new messages.

**Action #1:** Check your batteries – There is a 99% chance that either one is installed incorrectly and/or one is weak/dead.

### **Care of the Unit:**

The Mini-Com can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

**Do not use** abrasive cleaners, as they will scratch the surface of the unit.

**Do not submerge** the unit, as it will damage the electrical components.