

**molift**<sup>®</sup>  
a part of Etac



# Molift EvoSling

Comfort, functionality  
and safety

**etac**<sup>®</sup>

# Molift EvoSling



Molift has been manufacturing high quality hoists and slings for generations. Quality, comfort and safety are key considerations during product development. We offer a wide range of sling models and sizes ensuring we have a solution, whatever the users requirements.

## What are the user's requirements?

For people who need help with mobility, it's important that the sling is both comfortable and safe. It must facilitate transfers and meet the user's requirements.

## Comfort, function and safety

Our slings have been developed in conjunction with carers and users, in order to meet their requirements in terms of functionality, comfort and safety. The slings must be comfortable and secure, while at the same time being easy to work with for both the user and carer.

Our standard models are designed for use in the usual types of transfers and are suitable for most users. They are available in three different back heights: HighBack, MediumBack and LowBack. We know that users may have a wide variety of requirements, so we've designed additional models with specific features to meet their needs, including the EvoSling Hygiene and EvoSling Ampu.

The design of our slings ensures that they distribute weight and pressure evenly for maximum safety and comfort.

## Individual testing

We recommend individual testing, and offer a wide range of different models and sizes. To assist therapists with this we have a Sling Assessment Guide.



### Colour coding

Colour coding of hoisting loops helps you find the right fit.

### Padding

Padding in back and leg support distributes the weight over a larger area, making the sling more comfortable for the user.

### Belt

EvoSling Hygiene has a belt for extra safety as well as anti-slip protection on the inside.

### Guide straps

Guide straps assist with positioning during the lifting process.

EvoSling MediumBack Padded.

### Flexibility

Multiple hoisting loops on both head and leg supports allow the distance to the suspension and the user's sitting angle to be adjusted.

### Three different back heights

HighBack, MediumBack and LowBack to suit different needs.

### Labelling

Clear, practical information on a highly visible label. Washing instructions and technical information are found in a protective pocket at the back.

### Application pocket

Pockets in the back and on the leg supports make application easier.

## Informative, clear labelling

The large label contains all the information you will need for day-to-day use. Washing instructions, serial number and other technical information can be found behind the label in a pocket which protects against wear and tear during washing.


## Periodic inspection

The sling has a label on which you can record when a periodic inspection has taken place. It can be found in the label pocket. By recording the date on which the sling is first used on the label, you can plan when the first inspection should take place.


## Materials

The polyester slings are hardwearing and durable. The material is flexible and easy to maintain and has a dirt-repellent surface.

The polyester net slings are equally hardwearing, durable and flexible. The material also offers good ventilation.

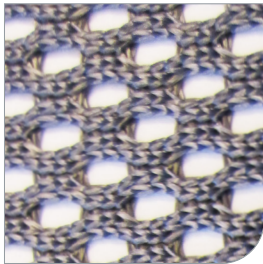

Polyester net

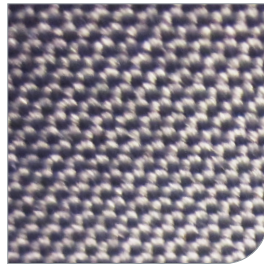

Polyester

Periodic inspection performed

MONTH	YEAR							
	2014	2015	2016	2017	2018	2019	2020	2021
1								
2								
3								
4								
5								
6								
7								
8								
9								
10								
11								
12								

User name / Name

.....

.....

.....

## Detailed information about EvoSling

### Recommended user weight

XS	17–25 kg
S	25–50 kg
M	45–95 kg
L	90–160 kg
XL	160–240 kg
XXL	230–300 kg

### Materials

Polyester  
PE-cellfoam (padded models)

### Washing instructions



Washing at higher temperatures wears out the material faster.

### Colour coding



# CONTENTS

**molift**<sup>®</sup>  
a part of Etac

Molift EvoSling	2
Molift EvoSling HighBack	7
Molift EvoSling MediumBack	8
Molift EvoSling LowBack	9
Molift EvoSling Hygiene	10
Molift EvoSling Ampu MediumBack	11
Molift EvoSling Comfort MediumBack	13
Accessories for the Molift EvoSling	14

---

[www.etac.com](http://www.etac.com)



# Molift EvoSling HighBack

FOR USERS WHO NEED A LOT OF SUPPORT

## Comfortable support

EvoSling HighBack supports the entire back and has a comfortable head support. The sling is suitable for most users, including those with poor core and head stability.

EvoSling HighBack is suitable for lifting to and from a lying position, as well as from the floor. The sling provides a comfortable reclined sitting position.

The pocket in the lower back and on the leg supports facilitates application of the sling.

## Two options

Molift EvoSling HighBack comes with two options: polyester with padding in the back and leg supports and a soft polyester net with padded head and leg supports.




### Easier positioning

The polyester net sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.




### Molift EvoSling HighBack

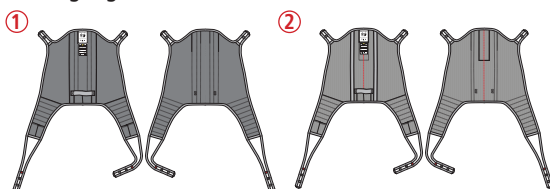

Size	① Padded Item no.:	② Net Padded Item no.:
XS	1722810	1722710
S	1722820	1722720
M	1722830	1722730
L	1722840	1722740
XL	1722850	1722750
XXL	1722860	1722760

Size	Ⓐ Back height	Ⓑ Width
XS	70 cm	30 cm
S	80 cm	35 cm
M	93 cm	40 cm
L	100 cm	47 cm
XL	105 cm	54 cm
XXL	110 cm	60 cm

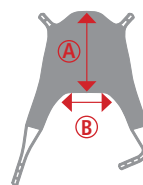

# Molift EvoSling MediumBack

FOR USERS WHO HAVE HEAD CONTROL BUT NEED FULL BACK SUPPORT

## Full back support

EvoSling MediumBack is a versatile model which is suitable for the most common lifting situations. The sling provides a comfortable upright sitting position with full back support.


Users place their arms inside the sling. A pocket in the lower back and on the leg supports facilitates application of the sling. The polyester net sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.

## Two options

Molift EvoSling MediumBack comes with two options: polyester with padding in the back and leg supports and a soft polyester net with padded leg supports.




### Molift EvoSling MediumBack

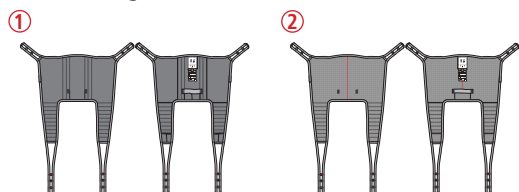

Size	① Padded Item no.:	② Net Padded Item no.:
XS	1723010	1723110
S	1723020	1723120
M	1723030	1723130
L	1723040	1723140
XL	1723050	1723150
XXL	1723060	1723160

Size	Ⓐ Back height	Ⓑ Width
XS	54 cm	30 cm
S	56 cm	37 cm
M	60 cm	49 cm
L	67 cm	57 cm
XL	70 cm	72 cm
XXL	73 cm	81 cm

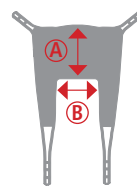



# Molift EvoSling LowBack

FOR ACTIVE USERS WITH GOOD CORE STABILITY



## Shoulder-height support

EvoSling LowBack is a versatile model which provides shoulder-height support. The sling provides a comfortable sitting position and users can place their arms inside or outside the sling.


A pocket in the lower back and pockets on the leg supports facilitate application of the sling. The polyester net sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.

## Three options

Molift EvoSling LowBack comes with three options: polyester with padding in the back and leg supports and soft polyester net with or without padded leg supports.

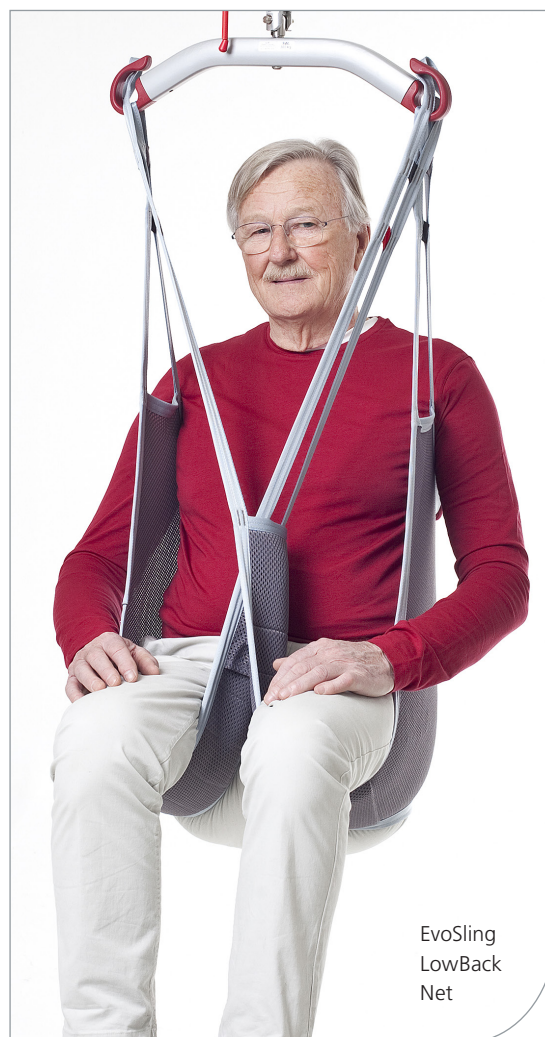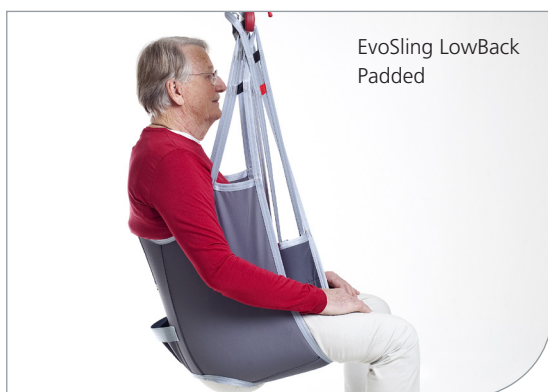


### Molift EvoSling LowBack

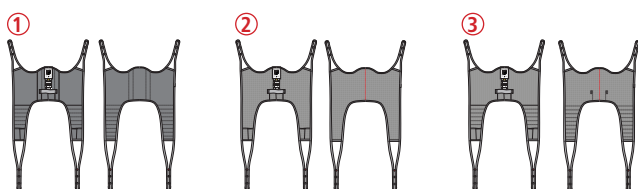

Size	① Padded Item no.:	② Net Item no.:	③ Net Padded Item no.:
XS	1722310	1722410	1722510
S	1722320	1722420	1722520
M	1722330	1722430	1722530
L	1722340	1722440	1722540
XL	1722350	1722450	1722550
XXL	1722360	1722460	1722560

Size	Ⓐ Back height	Ⓑ Width
XS	41 cm	40 cm
S	43 cm	43 cm
M	44 cm	48 cm
L	47 cm	55 cm
XL	51 cm	62 cm
XXL	55 cm	70 cm

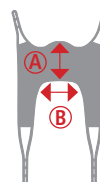

# Molift EvoSling Hygiene

FOR USERS WITH GOOD CORE STABILITY AND MUSCLE TONE

## Easy to position

EvoSling Hygiene has a low back and a large leg support opening to facilitate dressing and undressing when using the toilet.

## Application pockets



Molift EvoSling Hygiene is made of polyester and has a padded back and padded leg supports. The application pockets on the leg supports makes positioning them easier.

## Extra security

EvoSling Hygiene has a belt for added security during lifting.



### Molift EvoSling Hygiene

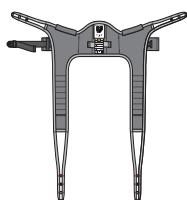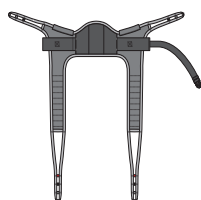

#### Size

XS  
S  
M  
L  
XL  
XXL

#### Padded Item no.:

1722210  
1722220  
1722230  
1722240  
1722250  
1722260

#### Size


XS  
S  
M  
L  
XL  
XXL

#### (A) Back height

29 cm  
33 cm  
33 cm  
33 cm  
34 cm  
35 cm

#### (B) Width of belt

35 cm  
46 cm  
52 cm  
70 cm  
73 cm  
76 cm

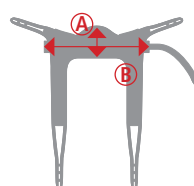

# Molift EvoSling Ampu MediumBack

FOR USERS WITH AN AMPUTATED LEG OR WITH POOR MUSCLE TONE


## Secure and comfortable

EvoSling Ampu MediumBack has a narrow leg support opening and a shoulder-height back. The sling's leg supports are wrapped under the legs. The leg support straps are passed through the strap guide loop to keep the leg supports together. A small leg support opening minimises the risk of the user slipping through the sling.



A pocket in the lower back and on the leg supports facilitates application of the sling. The sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.

## Anti-slip for added security

EvoSling Ampu is made of polyester net and has anti-slip material on the inside under the buttocks to provide added security during lifting. The user has a comfortable reclined sitting position. The leg supports are padded for optimum comfort.

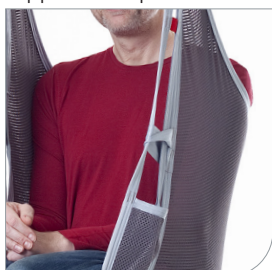

### Molift EvoSling Ampu MediumBack

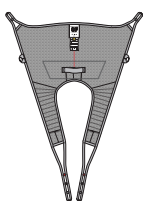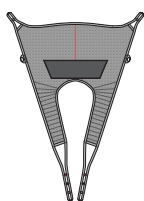

#### Size

XS  
S  
M  
L  
XL  
XXL

#### Net Padded Item no.:


1723210  
1723220  
1723230  
1723240  
1723250  
1723260

#### Size

XS  
S  
M  
L  
XL  
XXL

#### Ⓐ Back height Ⓑ Width

Size	Ⓐ Back height	Ⓑ Width
XS	47 cm	21 cm
S	52 cm	26 cm
M	63 cm	32 cm
L	77 cm	42 cm
XL	90 cm	48 cm
XXL	100 cm	54 cm

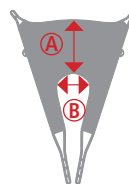



# Molift EvoSling Comfort MediumBack

A FULL BODY SUPPORT SLING FOR USERS WHO NEED EXTRA SUPPORT

## Support under the entire sitting area

EvoSling Comfort MediumBack Net is a full-body support sling without split leg support and with a shoulder-height back. The sling gives the user support under the entire sitting area and is ideal for users with joint problems or high sensitivity to pain.

The sling is positioned under the user in the lying position. It is made of polyester net, which provides good ventilation and can be left in the wheelchair underneath the user.

## Adjustable sitting angle

Multiple hoisting loops on both the head and leg supports allow the sitting angle to be adjusted.


The sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.



### Guide straps


The guide straps are built into the side straps to ensure that the sling is comfortable.

### Molift EvoSling Comfort MediumBack

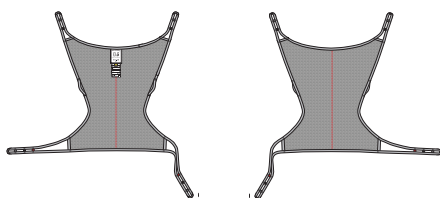

Size	Net Item no.:
XS	1722610
S	1722620
M	1722630
L	1722640
XL	1722650
XXL	1722660


Size	Ⓐ Back height	Ⓑ Width
XS	77 cm	85 cm
S	90 cm	86 cm
M	102 cm	94 cm
L	115 cm	113 cm
XL	118 cm	120 cm
XXL	120 cm	126 cm

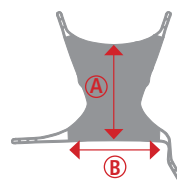

# Molift EvoSling accessories

## Molift EvoSling Support

### Accessories for slings without head support

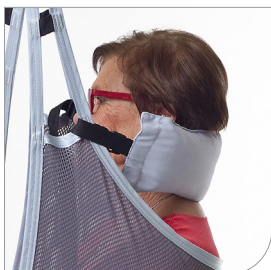

The head support is attached to the sling and its length can be adjusted as required using the plastic buckle. It gives the user comfortable support for the head and is a useful addition when the user requires more support.




**Product**  
Molift EvoSling Support

**Item number**  
1723400



## Molift EvoSling Soft Padding

### For pressure-sensitive users

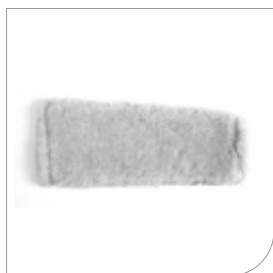

EvoSling Soft Padding consists of pressure-relieving padded sleeves which are slipped over the leg parts of slings with split-leg support.

**Product (sold in pairs)**  
Molift EvoSling Soft Padding  
XS-M  
Molift EvoSling Soft Padding  
L-XXL  
Molift EvoSling Hygiene Soft Padding  
XS-XXL


**Item number**  
1723710  
1723810  
1723740

## Molift EvoSling Extension Loop

### Straps for easier adjustment




Used to extend the hoisting straps or to keep the leg supports together on slings with split-leg support.

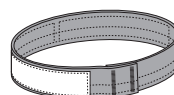

**Product**  
Molift EvoSling Extension loop  
16 (+9) cm  
Molift EvoSling Extension loop  
25 (+18) cm

**Item number**  
1723510  
1723520

## Molift Strap Marker

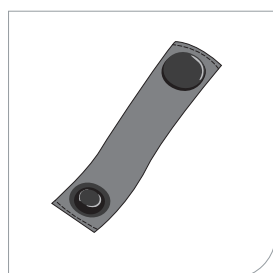

For marking a selected hoisting loop for head or leg support hoisting straps. Mark the hoisting loops selected during individual testing of the sling.

**Product (sold in pairs)**  
Etac Molift Strap Marker

**Item number**  
1723600

## Molift EvoSling FlexiStrap

### Avoid heavy lifting



The Molift EvoSling FlexiStrap is used to easily and comfortably lift a leg or arm during care or handling. The hoisting strap is made of padded polyester and comes in two widths. There is a pocket on one side of the strap to facilitate positioning.

**Product**  
Molift EvoSling FlexiStrap, standard  
15 x 95 cm  
Molift EvoSling FlexiStrap, wide  
25 x 95 cm

**Item number**  
1723310  
1723330



## Immedia SlingOn

### When the user can't be turned onto their side



If you combine an "Immedia SlingOn" with a sling, you can position the sling directly under the user, i.e. without first having to turn the user onto their side.

**Product**  
Immedia SlingOn, small  
80 x 100 cm  
Immedia SlingOn, medium  
80 x 170 cm  
Immedia SlingOn, large  
120 x 150 cm

**Item number**  
IM437/S  
IM438  
IM437



Immedia SlingOn



“Everybody is entitled to live a free and independent life and to fulfil their dreams whatever their physical circumstances.”