

# FOLDING SHOWER CHAIR

LVA3036GRY

vive




OWNER'S MANUAL

# FOLDING SHOWER CHAIR

## WHAT'S INCLUDED


- Main Base with Seat (A)
- Backrest (B)
- Legs x 4 (C)
- Foam Cushions (Seat and Backrest) (D)

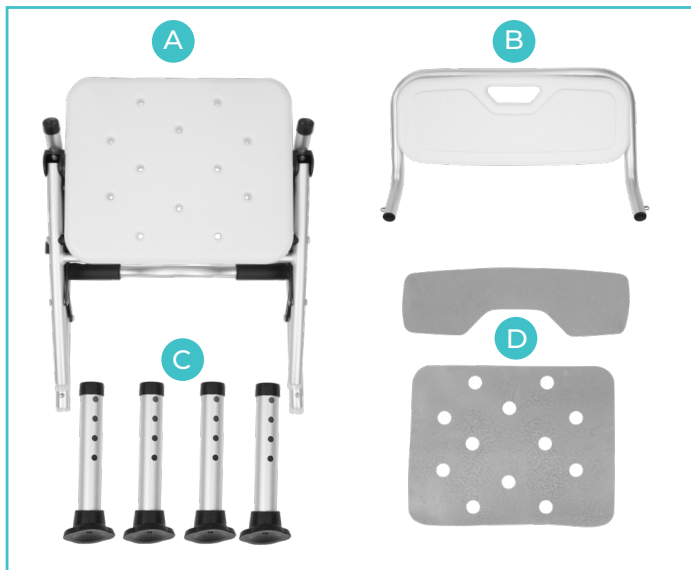

## INSTRUCTIONS

### STEP 1

Install the 4 Legs (C) on the Base (A). Turn the Chair over and adjust the height of each leg accordingly. Make sure all 4 legs are balanced.

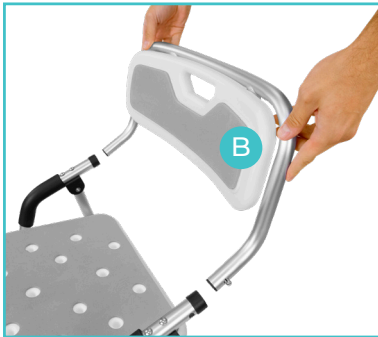

### STEP 2

Install the Backrest (B) onto the Base.

**NOTE:**

For Storage easily lift up the front of the seat to fold the chair.



## WARNING

- Make sure Shower Chair feet are wiped clean and dry from shower scum before each use.
- Maximum User Weight: 300lbs

## FAQ

### **Q: Is it mold and mildew resistant?**

A: Yes, the chair is resistant to mold and mildew, but should still be cleaned to maximize longevity and hygiene.

### **Q: How do I clean it?**

A: The chair can be wiped down with mild soap, an ordinary bathroom surface cleaner, or disinfectant. Also check to make sure water is draining properly through the holes in the rubber feet of the Leg Extensions, and isn't accumulating inside.