


Freedom Bath




Bathe and shower patient

1

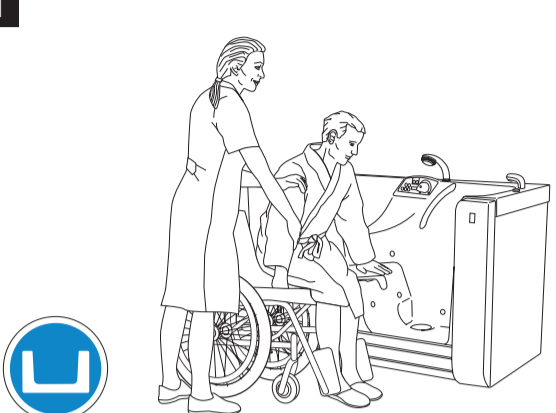

Press the door down button to lower the door to its lowest position. Help the patient enter the bathtub.

2

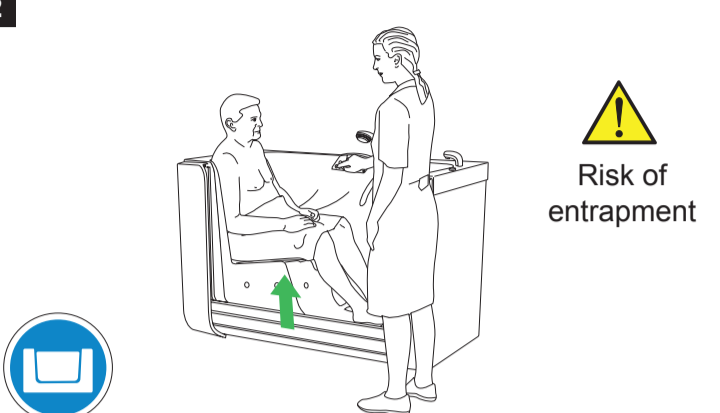

Risk of entrapment


For bathing: raise the door to the top position using the door up button.
For showering: leave the door down at least 25 mm (1 in) from the top position to keep drain open.

3




Press the door seal/drain button, the LED will flash a green light.

4



At proper inflation the LED has a steady green light.

5

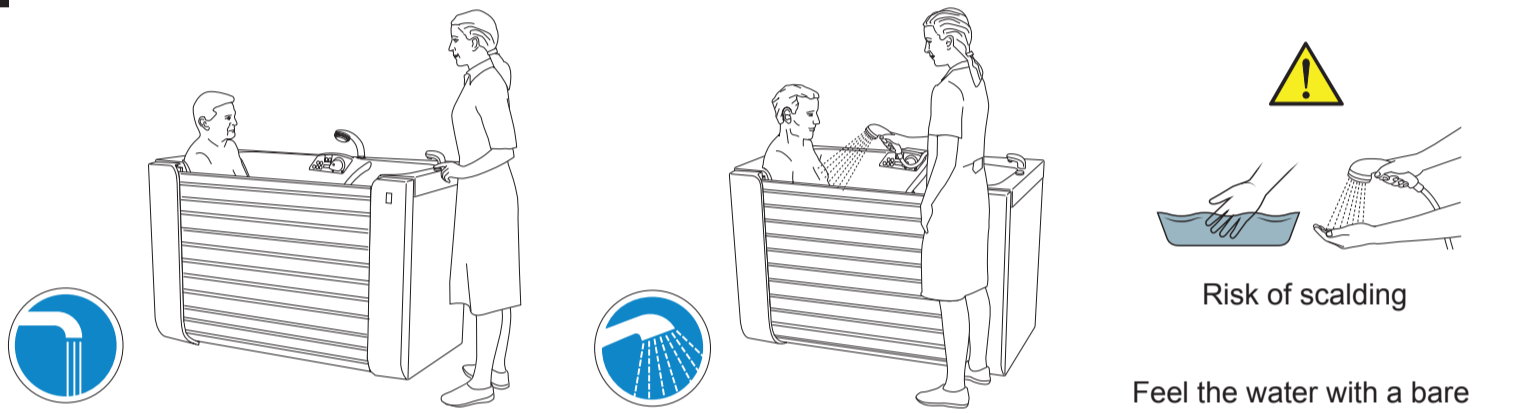


Risk of scalding

For bathing: fill bathtub using the bathtub filling button.
For showering: use the shower button.

Feel the water with a bare hand. Adjust temperature if needed.

End bath

6



Press the door seal/drain button once to open drain.

7




Press again to deflate the seal. During the deflating the LED will flash a green light.

8



Optional. Shower the patient. Lower door for more convenient access to the patient.

9

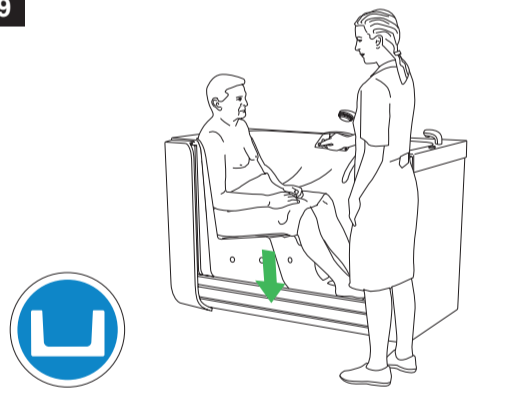

Press the door down button to lower the door to the lowest position. Help the patient leave the bathtub.