

FREEDOMTM

Concepts Inc.



CREATING A **CYCLE** OF *mobility*TM

OWNER'S MANUAL



Congratulations on the purchase of your new Freedom Concepts bike! This Owner's Manual refers to our complete bike line and has been color-coded to easily reference the options and components that apply to your bike needs.

If the manual and online video links do not answer your questions, please feel free to contact our Customer Service department at 1-800-661-9915.

We hope you enjoy your bike for many years to come!

Remember to always "Cycle Hard" and "Chill-Out" often!



Model # _____
 Serial # _____
 Date of Purchase: _____

www.freedomconcepts.com
**Register your bike online
 for a chance to win cool prizes!**

Maintenance



1. Keep tire pressure inflated to manufacturer's recommended pressure shown on the side of tire.
2. Keep bearings properly lubricated at all times to ensure smooth rotation and lasting operation.
3. Make sure all bolts and screws are tight.
4. Always keep chain lubricated.
5. Always keep bike clean and dust free.
6. Keep bike out of severe weather conditions.
7. Ensure that at least 4" of seat post is in the seat tube of frame. DO NOT raise seat post past maximum height indicator.
8. Adjust brakes regularly to ensure proper stopping.
9. Adjustments can be made using the *Freedom Concepts Multi-tool*, which is shipped with all orders.

Safety Considerations

- ✓ Rider must be assisted at all times.
- ✓ Position heels at back of foot plate when securing straps.
- ✓ Make sure belts and straps are securely fastened.
- ✓ Make sure toes do not hit the floor.
- ✓ Always wear appropriate head protection. **Additional safety gear recommended.*



© 2013 FREEDOM CONCEPTS INC.



**Did you know?
We are so much more
than just bicycles!**

Visit us online to
see all of our new
product offerings.

Visit us at
www.FreedomConcepts.com

 Find us on Facebook.

 Tweet us on Twitter.


 Visit us on YouTube.

Table of Contents



Looking for something in particular? Use our handy table of contents to source out the exact page for what you need or glance through the color-coded sections on the right-hand side for a particular category. Please note: not all instructions are relevant to each bike. Use the quick color-code reference from page 3 to ensure the instructions you are looking at are applicable to your bike's needs. Also, follow our QR Codes for online video tutorials for repairs.

Welcome	1	Steering	8-11	Braking	15-17
• Bike Registration		• Handlebar/Grip		• Hand Brake Tension	
• Maintenance Guidelines		• Alignment		• Dual Cable Junction	
• Safety Considerations		• Self-Centering Spring		• Parking Back Brake	
• Overview		• "U" Bar		• Caliper Tension	
Models	3	• Quick Release Lever		• V-Brake Tension	
• Discovery • Adventurer		• Handle Cable Tension		• Disc Brake Tension	
• Excursion • Odyssey		• Tandem Cable Tension		Wheels and Tires	18
• Expedition		• Rear Steering Handlebar Column.		• Tire & Wheel Options	
• Color-code Reference		Gear Shifting And Crank/Sprocket Adjustments	12-15	• Removable Rear Wheel	
Straps, Harnesses and Foot Plates	4-5	• Hand Grip Shifting		Customized Options & Accessories	19-20
• Seat Belts and Ankle Straps		• Gear Box Hub		• Neoprene Chest Harness	
• Chest Harness		• Thumb Pedal Control Shifter		• Handcycles	
• Velcro Straps		• Rear Control Gear Transfer		• Abductor/Adduction Leg Support	
• Toe-up Pulley Adjustments		• Thumb/Finger Shifter		• Stationary Trainer	
• Foot Plate Adjustments		• Multi-Speed Shifter		• Communication Tray	
Seats	6-7	• Rear Sprocket		• Assorted Accessories	
• Adjusting Height		• Electronic Motor		Warranty	21
• Adjusting Incline		• Multi-Length Crank		Reward Program	22
• T-Bar Seat Installation		• Bottom Bracket Crank		Mission	22
• Head Support		• Pedal Spacers			
• Swing Away Lateral Trunk Support					
• Head Rest					



Color-Code Quick Reference

To help you better, we have color-coded all instructions pertaining to each bike. When searching the manual, look for the quick color reference to find the information relevant to you and your bike.



All products and services are subject to change at anytime without notice, at the sole discretion of us. We reserve the right to discontinue any product at any time.

Series Models

Freedom Concepts prides itself in enabling children of all ages, and even those who are just young at heart, to have fun. Cycling provides so many physical and psychological benefits. It offers the advancement of motor skills, strengthens one's core muscles, develops confidence, and offers independence. Stay connected with Freedom Concepts as we are constantly researching and developing new and better ways for all to *create a cycle of mobility*.

DISCOVERY SERIES

(DCP MINI, DCP 12, DCP 12WG, DCP 12WGX, DCP16, DCP 16WG, DCP 16WGX)

Rider Age 18 mon.- 14 years
Rider Inseam.... 6"- 26"
Max Load 60 lbs - 150 lbs
Bike Weight..... 40 lbs - 45 lbs
Wheel Size 12"- 16"
Length 42"- 48"



ADVENTURER SERIES

(AS 2000, AS 2000JR, AS 2000K, AS 2000N, AS 2600)

Rider Age 14+ years
Rider Inseam.... 23"- 32"
Max Load 225 lbs - 350 lbs
Bike Weight..... 65 lbs - 68 lbs
Wheel Size 20"- 26"
Length* 62"- 72"
(*with rear steering add 12")
Width 31"- 35"



ODYSSEY SERIES

(ASR 2011)

Rider Age 12+ years
Rider Inseam.... 19"-34"
Max Load 225 lbs
Bike Weight..... 53 lbs
Wheel Size 20"
Length* 77"
(*with rear steering add 12")
Width 34"



EXPEDITION SERIES

(EHD U16N, EHD U16WG)


Rider Age 7-14 years
Rider Inseam.... 13"- 26"
Max Load 150 lbs
Bike Weight..... 50 lbs
Wheel Size 16"
Length 50"
(with Rear Steer™ add 12")
Width 29"-34"





SEAT BELT & NEOPRENE ANKLE STRAP ADJUSTMENT

1. Loop strap as shown and pull (A) to tighten to desired length.
2. To close, push buckle (B) into lock clasp (C).
3. To release, squeeze buckle teeth (B) from side of lock clasp (C) and pull out buckle strap.

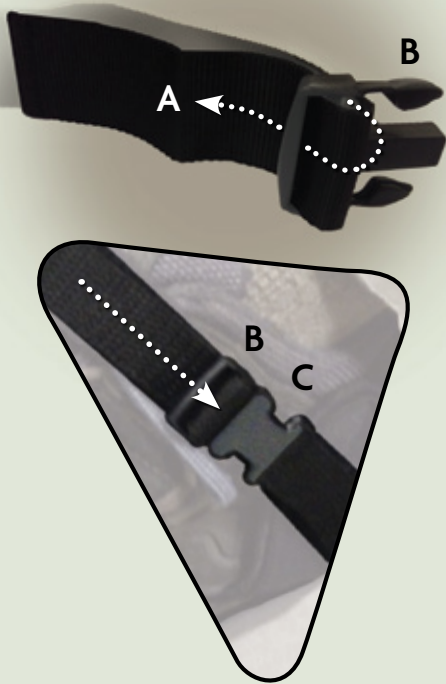

Straps, Harnesses and Foot Plates (Operations)


To ensure the bike rider is safely secured, please adjust the seat, straps, and harnesses to the proper settings for the individual.



RIGID AND NEOPRENE VELCRO STRAPS


Straps help keep the rider's feet firmly in place.

1. Velcro foot straps loop through the ring (D).
2. Pull to tighten and then press strap together so Velcro can lock in place.
3. To remove foot, simply peel apart Velcro straps.

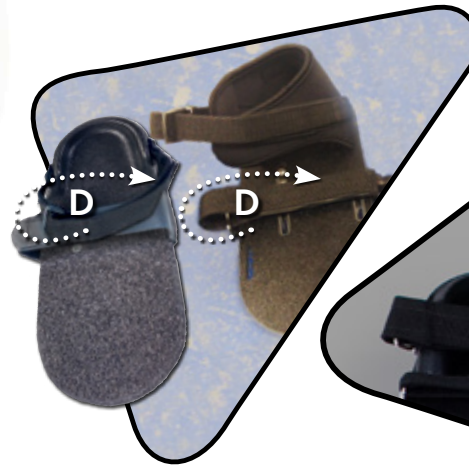


TOE-UP PULLEY TENSION ADJUSTMENT

The position on the toe-up pulley rope can be easily adjusted by simply loosening the tension on the rope lock (F) and shifting its position up or down the rope to the desired location. Once determined, tighten the tension back on the rope lock. Then double-check the position of feet and make any necessary minor adjustments by repeating the process.

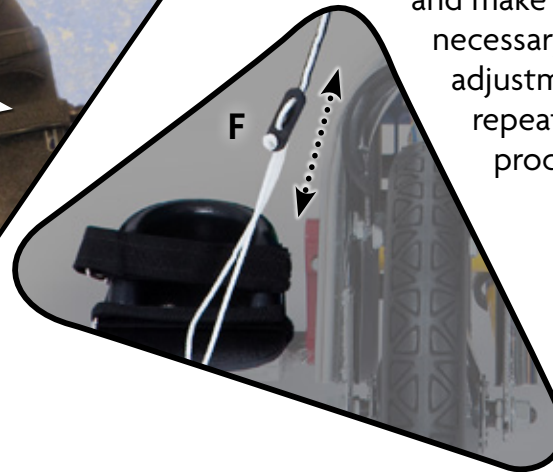

FOOT PLATE ADJUSTMENTS

Our Neoprene foot plates allow for quick and easy adjustments to properly match the individual's hip to foot ratio. Loosen the four foot plate screws (J) and shift the foot plate (K) forward or backwards. Once the desired position is determined, re-tighten the screws in place to the pedal base.

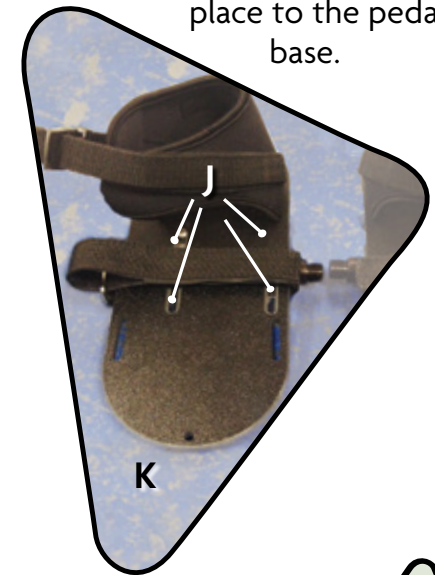

Always wear shoes when riding to protect the feet.

Secure the foot first by positioning the heel at the back of the foot plate (G) and then attaching the straps (H) around the foot until snug.





Learn More With Our How-To Videos.

Scan the QR Code with your phone and you will be linked to our Ready to Ride Video Series.



CHEST HARNESS STRAPS

Four #2 Robertson wood screws (F) are used to secure the chest harness to the seat back.

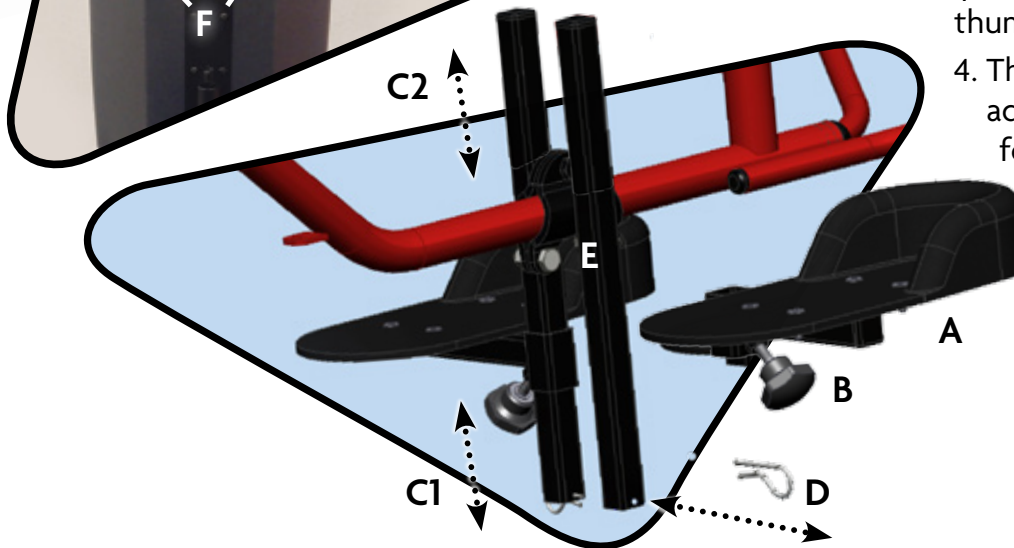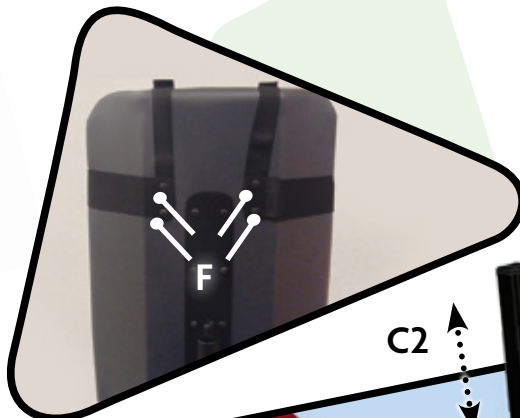

HANDCYCLE FOOTPLATE HEIGHT ADJUSTMENT

Once the handpedals and seat positioning have been determined, the footplates can be adjusted. The footplate can be positioned vertically at any height along the top or bottom of the vertical posts for optimal leg support and comfort.



1. To move the footplates (A), turn the thumbscrew (B) under each footplate counter clockwise until loose.
2. Then slide the footplate (A) up or down the post (C1 or C2) to the desired height. Re-tighten the thumbscrew (A).
3. To move the footplates (A) from the bottom post (C1) to the top post (C2) you need to remove the safety hitch pin (D). Simply pull the pin (D) straight out in a horizontal motion away from the post. Your safety hitch pin is provided to ensure you do not lose the footplates should you not tighten the thumbscrews (B) properly.
4. The frame clamp can also be adjusted to move the footplates forward or backwards. Two 1/2" wrenches are required to loosen the bottom clamp bolts (E) for repositioning to the desired location on the bike.

Please Note:
Seat lap belt must always be worn with the chest harness.

Proper use of chest, lap, and hip belts ensures trunk stability and promotes safe positioning.

ADJUSTABLE HEAD REST

1. Loosen thumb screws (I) on both sides of seat.
2. Raise or lower head rest (J) to desired setting for proper trunk and head heights.
3. Tighten thumb screws.

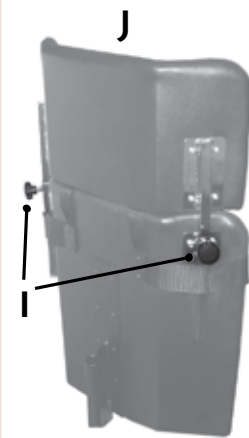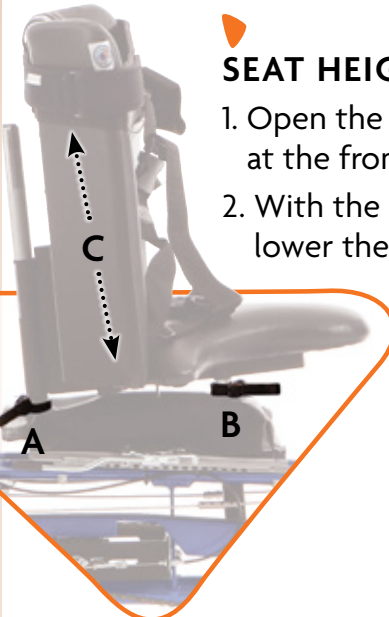


Seats (Adjustments and Operations)

Seats come with different optional adjustments. Please refer to the following pictures for an understanding of the different adjustments available.

**See page 9 for quick release lever tension adjustments.*

SEAT HEIGHT ADJUSTMENT

1. Open the quick release latches (A & B) at the front and back of the seat base.
2. With the latches open you can raise or lower the seat base (C) to your desired position.
3. Close the quick release latches (A & B) to clamp the seat base in the new position.

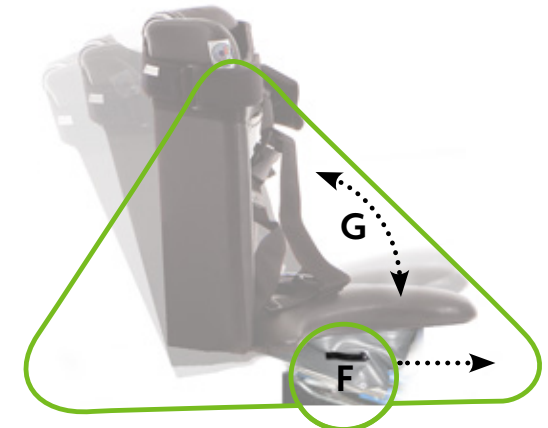

SEAT INCLINE ADJUSTMENT

1. Open quick release latches (F) found on both sides at the front of the seat base.
2. Tilt seat to desired position (G).
3. Close quick release latches (F) on both sides of seat to clamp seat incline at a new position.




SEAT BASE LOCATION ADJUSTMENT

1. Pull out seat base locking handle to release seat position (D).
2. Slide seat forward or backward to desired new location (E).
3. Close locking handle (D) and push seat back to ensure locked position.



Opened lever

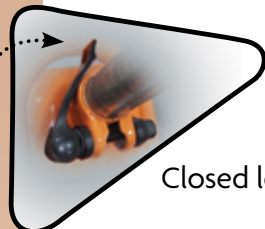


Closed lever

T-BAR SEAT INSTALLATION


If packaging requires, your T-Bar seat may need to be re-installed as shown.

1. T-Bar handle (D), behind seat, slides over the post all the way down to the bottom. Tighten with wedge bolt (E) beneath using 6mm key.
2. At the same time, slide the front seat post (F) into the frame as shown. Tighten quick release clamps to secure chair at desired height.
3. Close both levers to clamp seat in position. **See page 9 for quick release lever tension adjustments.*

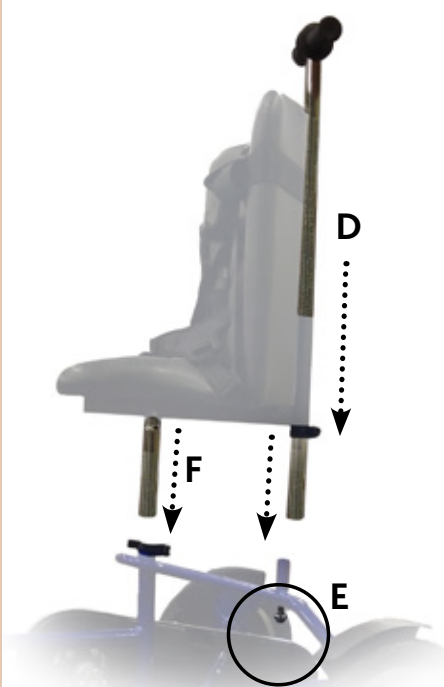

SEAT HEIGHT ADJUSTMENT

1. Open both quick release levers (A & B).
2. Adjust seat position to desired height (C).
3. Close both levers to clamp seat in position. **See page 9 for quick release lever tension adjustments.*

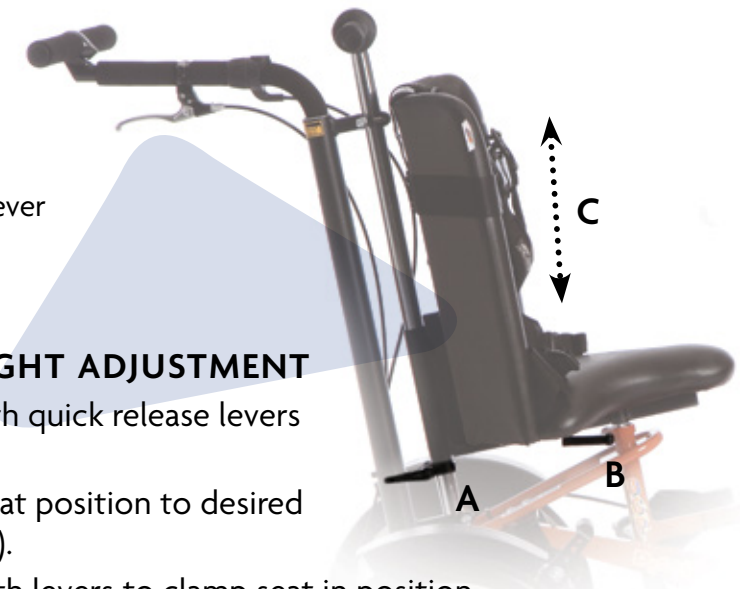

SWING AWAY LATERAL TRUNK SUPPORT

The red push button (G) allows the Lateral Pad to flip out of the way when getting on or off of the bike. The pad is reversible on the bracket for soft or firm support. Simply remove pins (H) on both sides of the pad by pulling. Switch pad as desired and then push in pins to lock back in place once again.



HEAD SUPPORT HARDWARE


Head support must be personally set exclusively to each individual's needs. See detailed product manual for proper hardware installation and adjustment settings.





SELF-CENTERING SPRING

Periodically check the self-centering spring to ensure proper tension. It is located on the bottom of the bike frame and connects to the forks on the front wheel. If the bike is not moving in a straight direction then the spring should be replaced.

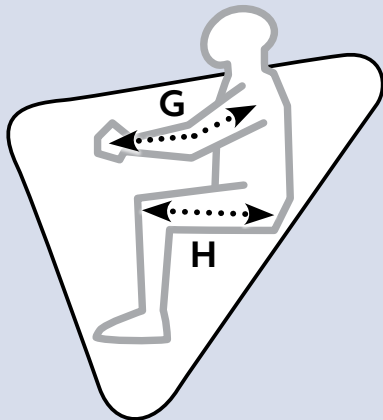

Steering (Adjustments and Operations)

All steering should be adjusted to individual needs. On the following pages we have provided some quick reference guides to help you with your steering requirements.

HAND DRIVE STEERING COLUMN POSITIONING

Easy adjustments (F) can be made on the Upright Handcycle to determine the ideal hand pedal positioning and to allow for ease with transferring on and off the bike.


RIDING

The ideal position for the hand pedals is more of a personal choice. Our recommendation though, is to have the hand pedals moved in towards the rider until there is a slight bend at the elbow(G) when they are grasping the hand pedal at its furthest position, away from the chest.

This is based on using a starting point where the knees and hips are in a horizontal plane(H) with the seat of the bike in the lowest possible position.

TRANSFERRING

For easier transfers on and off the bike, the crank bracket should be positioned as far away from the seat as possible (I).

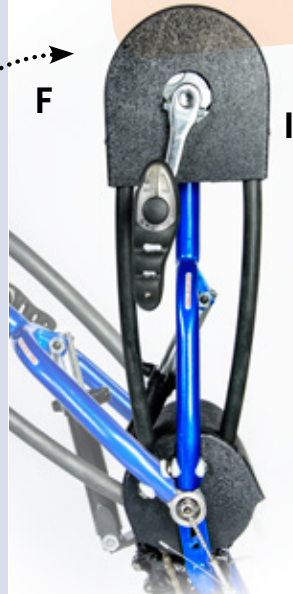


HAND DRIVE STEERING ADJUSTMENT RELEASE

To swing hand pedals towards or away from the seat you need to do the following:

1. Hold the top steering column (J)
2. Open quick release lever (K)
3. Position top steering column to desired position and close quick release lever.


**For a better understanding on how to work the quick release lever see page 9.*

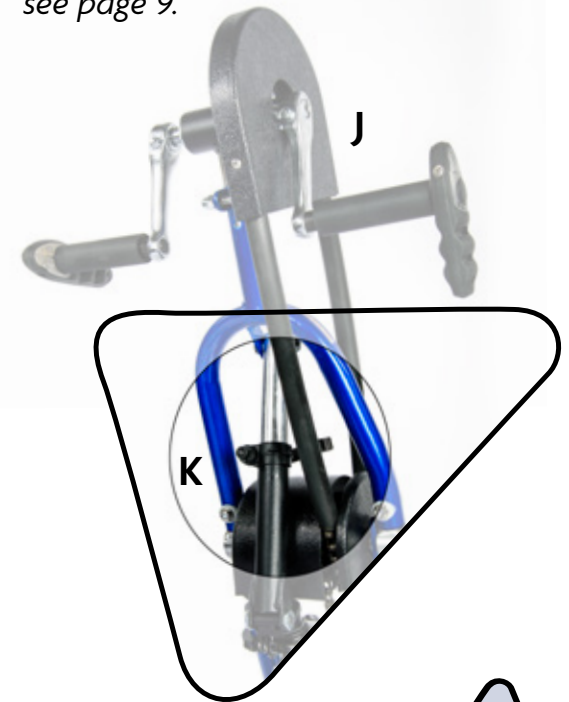

Quick Release Lever Adjustments


From time-to-time you may need to adjust the tension on your quick release levers.

1. Open lever (G), and rotate lever handle (H) clockwise to tighten, or in the opposite direction to loosen tension, while holding nut with other hand.
2. Once set, close the lever to lock in place. If lever handle cannot close, loosen tension by opening the handle and turning counter-clockwise to desired setting.

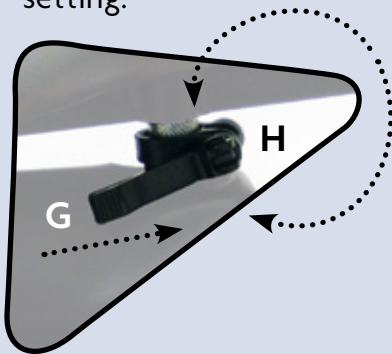

HANDLEBAR/GRIP ADJUSTMENTS


1. To adjust handlebars in or out, loosen 6mm socket head wedge bolt (A), approximately 4-5 full turns
2. Tap bolt (A) lightly with a hammer to unlock wedge nut (A1) from within handlebar shaft.
3. Turn handlebar shaft (B) to desired position and then tighten wedge bolt (A) until snug.
4. To adjust the grip, loosen 6mm socket head clamp bolt (C) and raise or lower the handlebar grip (D) to desired height.
5. Retighten clamp bolt (C).

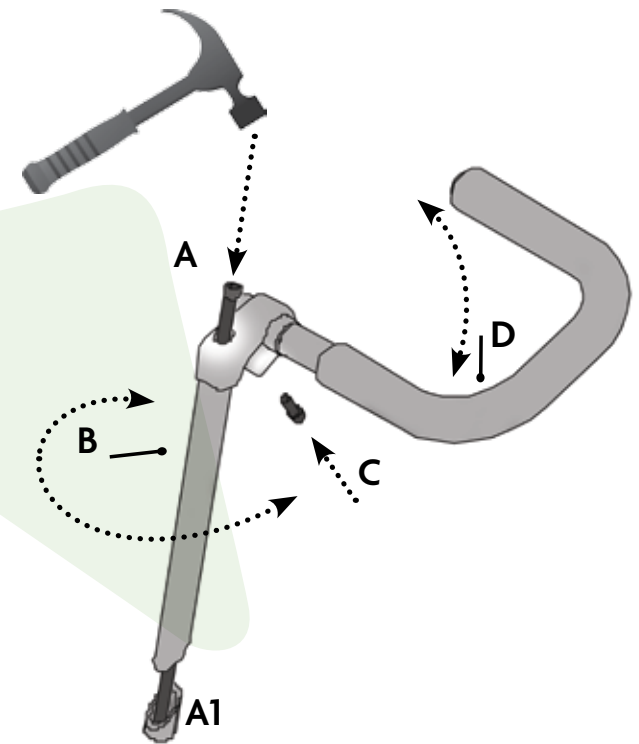

“U” BAR FRAME ADJUSTMENT*

Loosen quick release lever (F) to the unlock position for adjustment of the “U” bar frame. Move forward or backward to desired position. This is an excellent way to move handlebar assembly out of the way when transferring riders.

**Please note: In cases of extreme high tone the quick release lever may need to be replaced with the standard bolt and nut. Part # bolt: 4110440 Part # nut: 4130254*

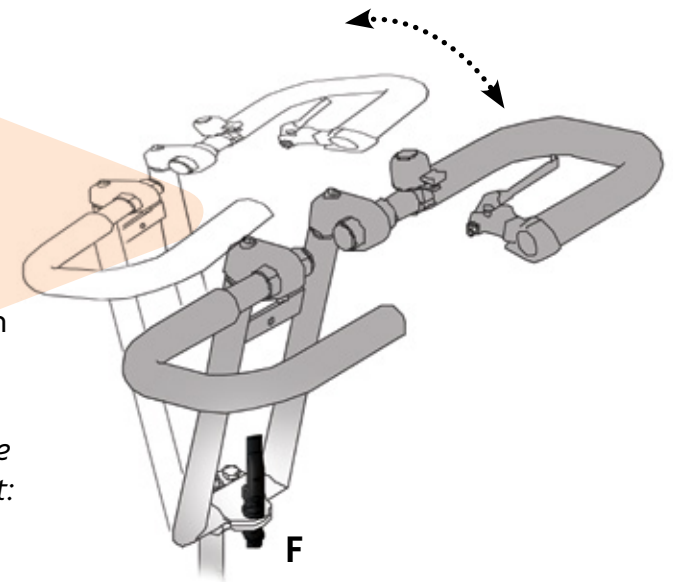





REAR STEER HANDLE TENSION ADJUSTMENT

To adjust the tension or re-install after shipping:

1. Loosen locking nut (A) with 7/16 wrench.
2. Slide outer barrel (B) of tie-rod towards locking nut to release from the ball joint, lifting up to disconnect.
3. Turn base of tie-rod (C) clockwise 1/2 to 1 full turn at a time to increase tension on steering.
4. Slide outer barrel of tie-rod towards locking nut and reconnect to ball joint.
5. Tighten locking nut (A).


**Disconnecting the rear steer handle during transportation of your bike will release the tension on the steering mechanism and allow the rear handle to turn out of the way—requiring less space for transportation.*

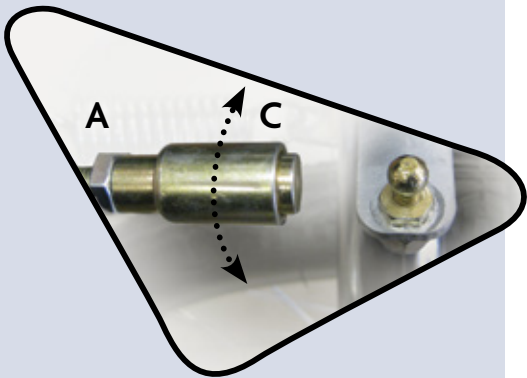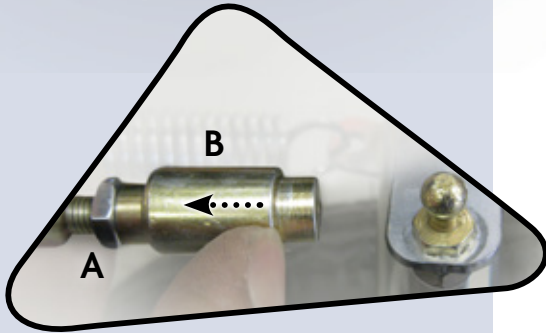

ADJUSTABLE REAR STEERING HANDLEBAR COLUMN

Rear steering column can be bent to allow for ease of transportation and storage.


1. Pop open quick release lever handle and pull outward (F) to release spring lock.
2. Tilt steering column on hinge bracket.
3. To re-attach column, simply tip upward.

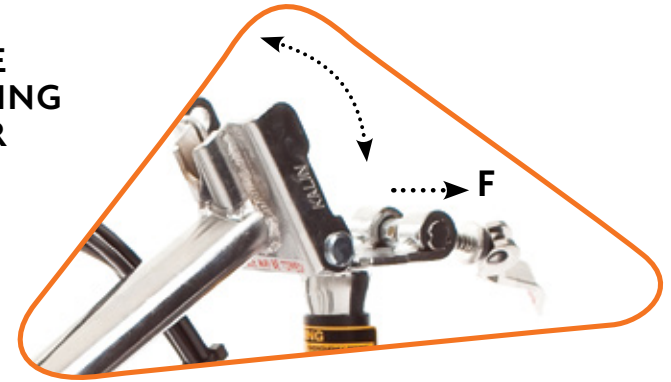

Need A Little Extra Help Or Don't Understand Something?


Our service department is glad to help!
Give us a call at:

1.800.661.9915

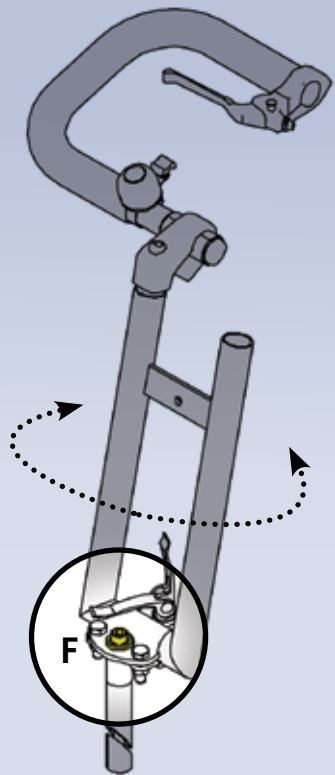

STEERING ALIGNMENT


Loosen 1/2" hex head stem bolt (F) approximately 4-5 turns, tap lightly with hammer, and rotate steering column left or right to align steering with front wheel.

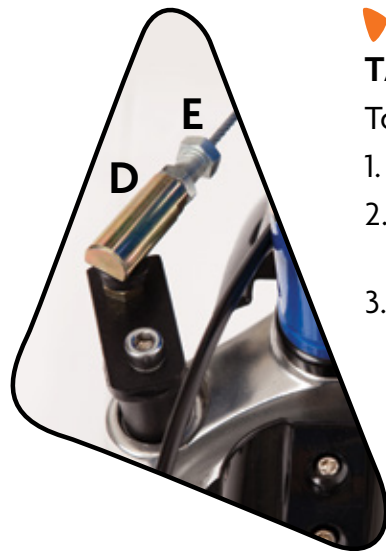

TANDEM CONNECTED CABLE TENSION ADJUSTMENT

To adjust cable tension:

1. Loosen locking nut (D) with 14mm wrench.
2. Adjust the tension nut (E) with the same 14mm wrench. Turn the nut clock wise to tighten and counter-clockwise to loosen cable.
3. Once desired tension is determined, tighten locking nut (D).

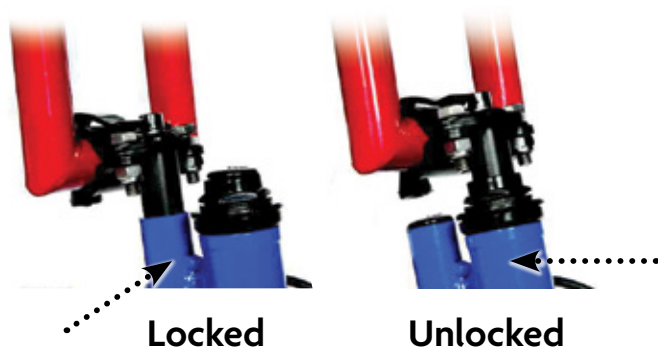

LOCKED/UNLOCKED STEERING ADJUSTMENT


1. When locked, the steering mechanism is fixed in position. Rear Steer™ required to steer bike when steering is locked
2. When unlocked, it is fully functional.

Handlebars with Toe-up Pulley

See page 4 for loosening toe-up pulley when moving handlebars out of the way.

Learn More With Our How-To Videos.

Scan the QR Code with your phone and you will be linked to our Ready to Ride Video Series.

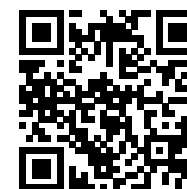

Gear Shifting (Adjustments and Operations)


All Freedom Concepts' adaptive bicycles are adjustable to individual needs. Provided are some quick reference graphics to help you with your shifting requirements.



AA: HAND GRIP SHIFTER OPERATION

Rotating the Revo-shifter (A) up towards the “-” sign will lower the gear for easier peddling. Rotating down towards the “+” sign will increase tension when peddling.

Please note: When changing gears with the hand grip shifter one must pause from pedaling. Once desired gear is selected pedaling can recommence.

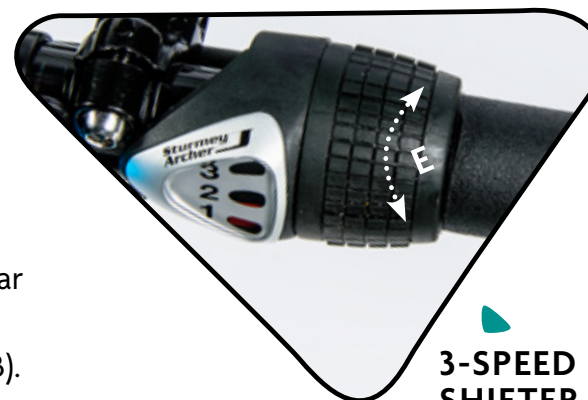

3-SPEED HAND GRIP SHIFTER OPERATION

Rotate shifter (E) up towards the “3” to increase tension when peddling. Rotating down towards the “1” sign will decrease tension when peddling.

Please note: When changing gears with the hand grip shifter one must pause from pedaling. Once desired gear is selected, pedaling can recommence.




BB: HAND GRIP / INTERNAL GEAR BOX HUB ALIGNMENT

To align hand grip shifter with the gear box hub adjust the following:


1. Shift hand grip shifter to gear “4” (B).
2. Check Nexus hub for alignment of yellow markers (C).
3. If yellow markers are not in alignment, turn the barrel adjuster (D) until both lines are equal.

Please note: Any major repairs should be handled by a qualified bike repair shop.

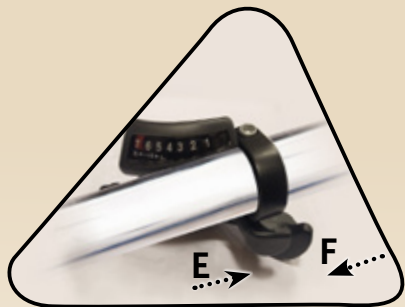


THUMB/FINGER GEAR SHIFTER OPERATION

Change gears up or down with an easy click. Using your thumb, push shifter lever (E) forward. With each click you raise a gear. Using your finger, pull shifter lever (F) and with each click, you lower a gear.

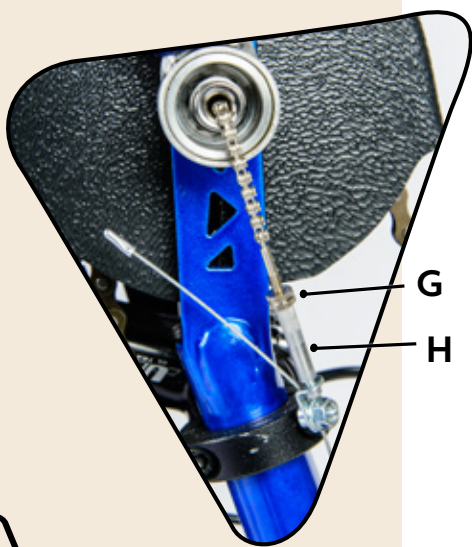

THUMB PEDAL CONTROL SHIFTER OPERATION

Tandem bikes allow the option for both riders to pedal the bike at the same time. When the shifter (A) on the handlebar is closest to the rider the (B1) setting has the front sprocket in a neutral position and the front rider doesn't contribute to the propulsion of the bike. Pushing the shifter (A) forward once to the (B2) position engages the front sprocket at a low setting so the front rider can contribute. Moving the shifter (A) forward once again to the (B3) position changes the sprocket to the high setting for more contribution from the front rider. Moving the shifter in reverse adjusts the setting back to the beginning.

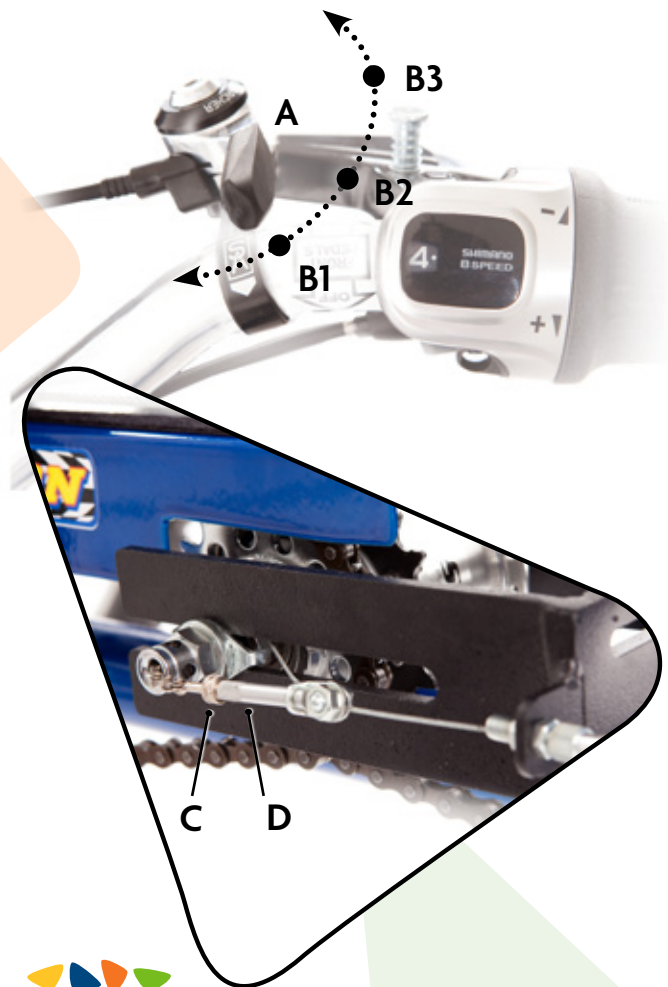

3-SPEED CABLE ADJUSTMENT

If the cable starts to stretch through normal use, quick adjustments can retighten the slack.

1. Shift to gear three.
2. Loosen lock nut (G) and turn barrel adjuster (H) until loose slacking cable is tight.
3. Tighten lock nut (G).




REAR CONTROL GEAR TRANSFER

If the cable starts to stretch through normal use, quick adjustments can retighten the slack.

1. Shift to neutral position (B1).
2. Loosen lock nut (C) and turn barrel adjuster (D) until loose slacking cable is tight.
3. Tighten lock nut (C).


ELECTRONIC ASSIST MOTOR

Did you know we offer the option of assist motors? They are ideal for those who take extended bike rides or for those who need assistance on inclines or when tandem riding. Ask your rep today to learn more.

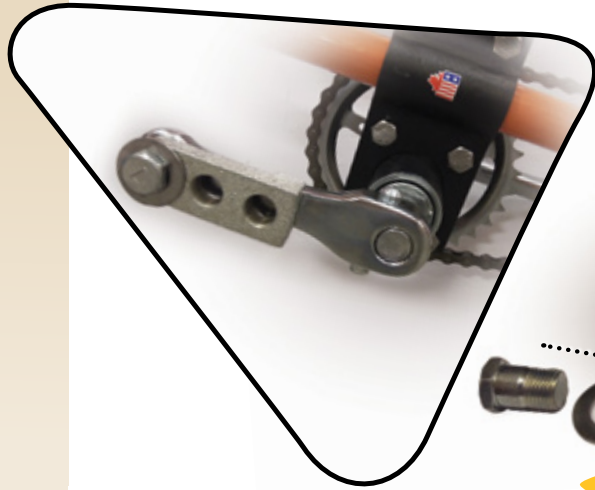

REAR SPROCKET ADJUSTMENT


Check set screw (A) periodically for tightness. Use 1/8" Hex Key wrench. This ensures good propulsion of the bike at all times.

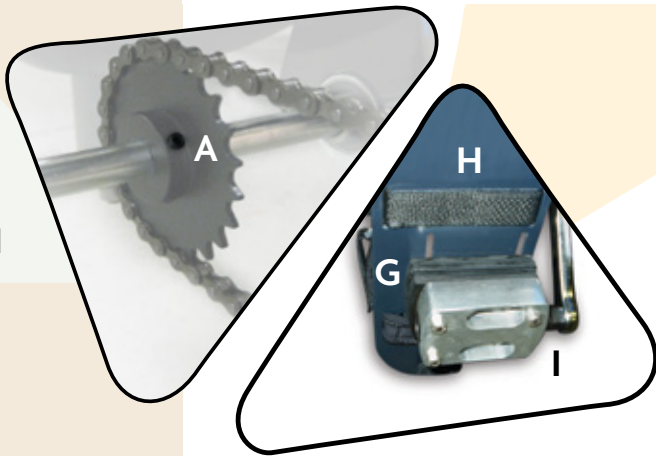

MULTI-LENGTH CRANK ADAPTER


A multi-length crank adapter is used for even greater adjustability of the range of motion.

1. Bolt adapter (B) to crank arm. Ensure locking support (C) on adapter is facing crank arm to secure adapter in place.
2. Attach pedal in appropriate adapter hole for desired range of motion.

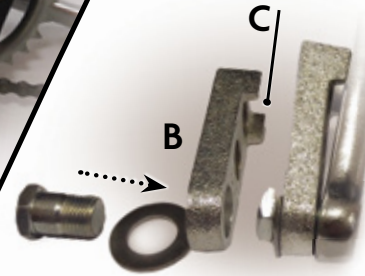

BOTTOM BRACKET AND CRANK ADJUSTMENT

1. Loosen 4 bracket bolts (D).
2. Remove roll pin (E).
3. Relocate bottom bracket (F) to appropriate desired position.
4. Lengthen or shorten chain with a chain break tool to correct length (sold separately).
5. Tension chain by moving bottom bracket (F) forward until there is no slack in the chain.
6. Tighten bolts (D), drill new hole in frame for roll pin using bracket hole for a guide and replace with new 1/8" roll pin (E).

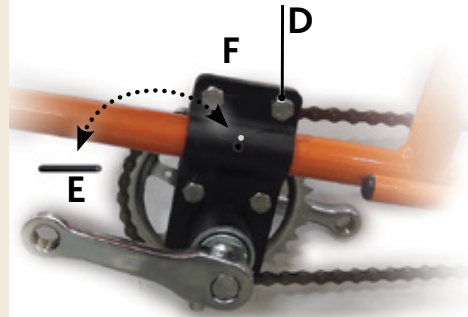

PEDAL SPACERS

1. Stack spacers (G) between foot plate (H) and foot pedal base (I).
2. Secure from the top with supplied screws (J) once desired height is determined through foot plate, spacers and into foot pedal base.



GEAR SHIFTING


MULTI-SPEED SHIFTER ADJUSTMENT

1. Turn top travel-limit screw marked (I), clockwise to reduce and counterclockwise to increase travel of chain beyond 7th gear.
2. Turn bottom travel limit screw, marked (J), clockwise to increase travel of chain beyond 1st gear.



Braking (Adjustments and Operations)

All Freedom Concepts' adaptive bicycles are adjustable to individual needs. Provided are some quick reference graphics to help you with your braking requirements.

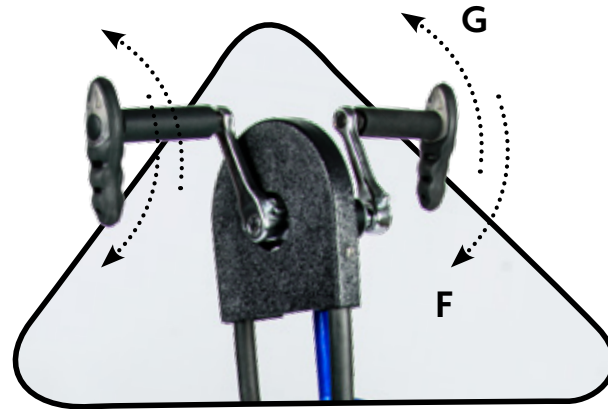


HAND COASTER BRAKE

Just like the conventional foot coaster brake design, stopping is created by changing your direction backwards (F) from which you are peddling (G).


Learn More With Our How-To Videos.


Scan the QR Code with your phone and you will be linked to our Ready to Ride Video Series.

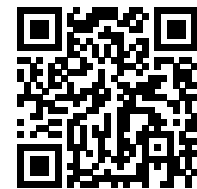

DISC BRAKE TENSION ADJUSTMENTS

1. Loosen the locknut (A) on the hand brake by turning counterclockwise.
2. Turn the barrel adjuster (C) counterclockwise to tighten tension or clockwise to loosen to desired tension. Then tighten (A).
3. Adjust disc brake at wheel by turning locknut (B) counterclockwise to loosen.
4. Alter barrel adjuster (D) to desired setting by turning counterclockwise to increase brake sensitivity or clockwise to decrease.
5. Tighten locknut (B).

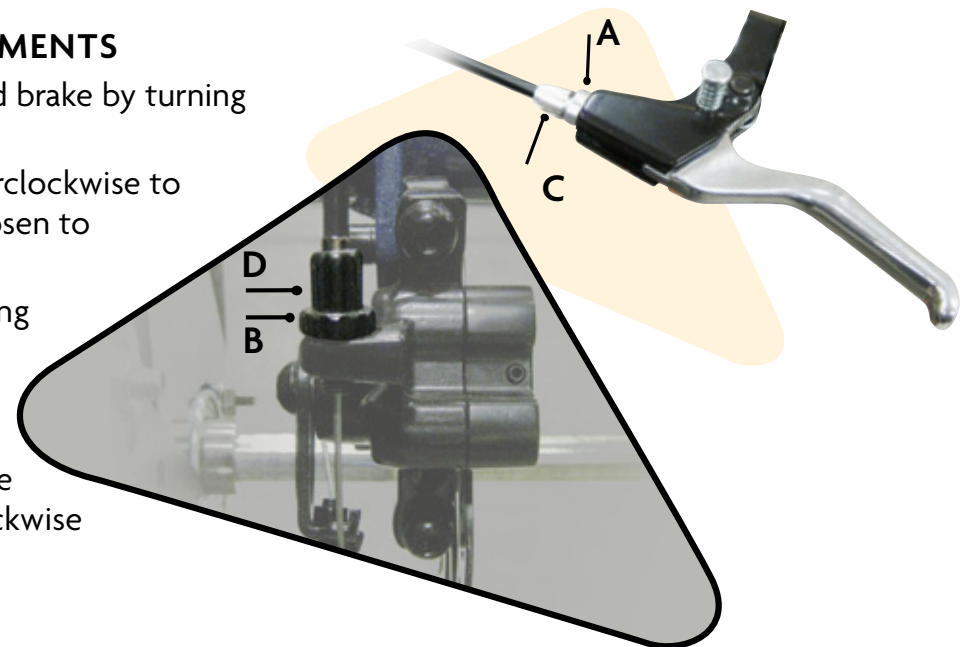

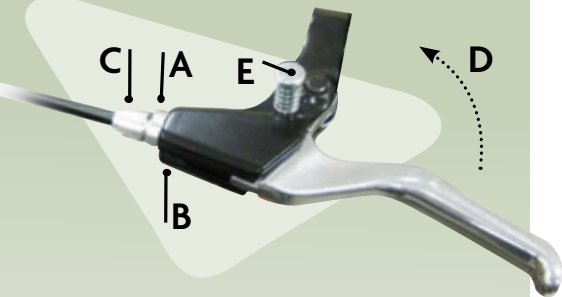

When transporting someone on or off the bike, always squeeze brake handle (D) and push in pin (E) to lock brake. This helps to stabilize the bike. To release brake lock, squeeze handle (D).



PARKING BACK BRAKE

(Not available for DCP Mini's)

Parking brakes are available for one or both rear wheels on most bike models. Parking brakes are ideal for transporting riders on or off the bike, by stabilizing the bike movement. To use the parking brake, simply push the brake lever down to lock and lift up to release.




AA: HAND BRAKE TENSION ADJUSTMENTS

1. Loosen the locknut (A) by turning away from housing (B).
2. Turn the barrel adjuster (C) towards the housing to loosen and away to tighten brake sensitivity.


Please note: when squeezing to apply the brakes, the brake lever should not touch the handlebar.

3. Turn locknut (A) clockwise until tight.

Caution: Over-tightening cable will cause brake pads to constantly rub on wheels.

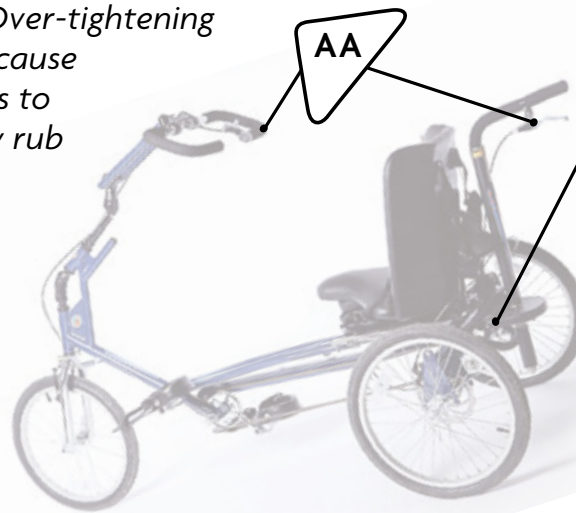

For instructions on cable adjustments to the calipers, follow caliper brake tension adjustments on page 17.

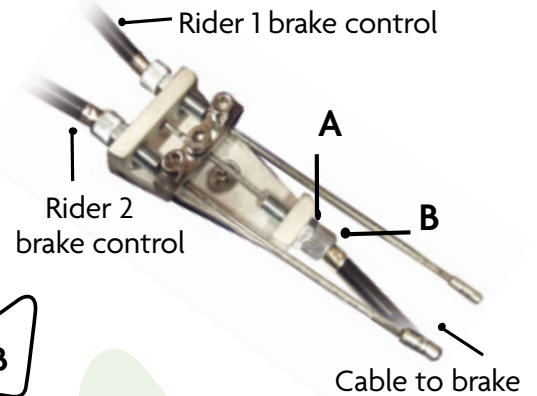

BB: DUAL BRAKE CABLE JUNCTION BLOCK ADJUSTMENTS

1. Loosen the locknut (A) by turning counterclockwise.
2. Turn the barrel adjuster (B) clockwise to tighten and counterclockwise to loosen brake sensitivity.

Please note: When squeezing the brake lever to apply the brakes, the lever should not touch the handlebar when closed.



3. Turn locknut (A) clockwise until tight.

Caution: Over-tightening cable will cause brake pads to constantly rub on wheels.

Always ensure brake pads only touch the rim of the wheel. Pads touching the tire can cause flats.


Learn More With Our How-To Videos.

Scan the QR Code with your phone and you will be linked.

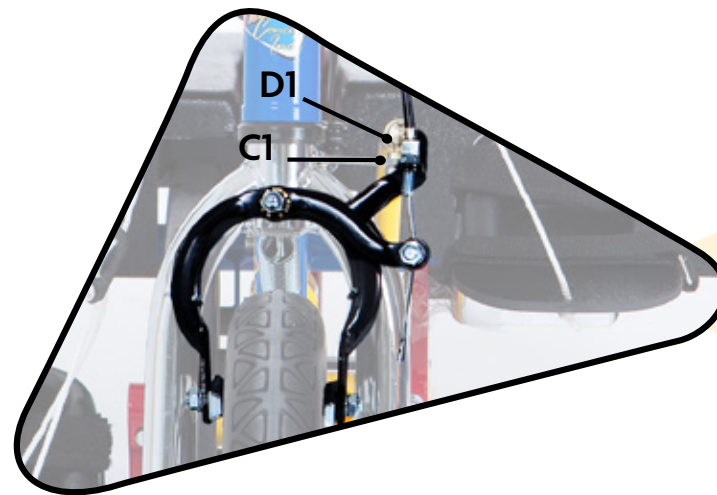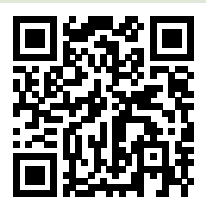

CALIPER BRAKE TENSION ADJUSTMENTS

1. Loosen the locknut (A1) on the hand brake by turning away from housing to allow for fine tune hand brake adjustments.
2. Turn the barrel adjuster (B1) counterclockwise to ensure brake pads stop tire from spinning when hand brake is squeezed and do not rub when released.
3. Loosen locknut (C1) on caliper brake.
4. Turn the barrel adjuster (D1) away from the housing to bring pads closer to rim to ensure brake pads stop tire from spinning when hand brake is squeezed and do not rub against the rim when released.
5. Tighten (A1 and C1).

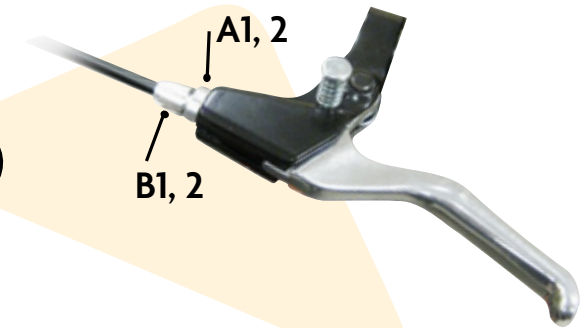

V- BRAKE TENSION ADJUSTMENTS

1. Loosen the locknut (A2) on the hand brake by turning counterclockwise.
2. Turn the barrel adjuster (C2) counterclockwise to ensure brake pads stop tire from spinning when hand brake is squeezed and do not rub when released.



*Please note:
Any further cable repairs should be handled by a qualified bike repair shop.*

Removable Rear Wheel*


*Available only in 20" models.

TO RELEASE THE WHEEL:

Open the quick release handle and rotate counterclockwise until it is completely off the drive shaft, and pull wheel out.

TO REPLACE THE WHEEL:

Insert and align wheel hub on axle dowel pin and insert quick release bolt, tighten and close handle to lock.

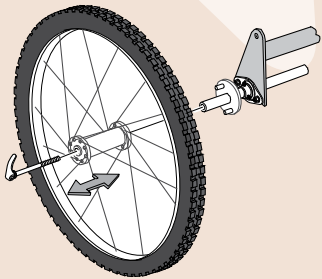

Wheels And Tire Systems

Depending on the model of bike you own, there are a variety of different wheels and tires available to you.

TIRE OPTIONS

For the best tire traction, we offer a variety of available options to fit your individual needs. Our smooth tires are ideal for the city streets and indoor riding. Our knobby tires work best for the rugged riders or off-road enthusiasts. Always keep tire pressure inflated to manufacturer's recommended pressure as shown on the side of tire. For maintenance-free riding, we offer the option of solid tires with the custom Freedom Concept gripper tread. The solid tires are ideal for multi-user environments, as they never go flat.

Please note: Not all tires are available in all sizes. Check online at www.freedomconcepts.com for availability on each model.



KNOBBY TIRES



SOLID TIRES



SMOOTH TIRES



WHEEL OPTIONS

We offer a variety of different wheel options. Aluminum, steel and ABS plastic.

Ask us about our different spoke covers to protect little fingers.



Racing Handcycles

Let nothing stop the ride! Individually tailored bikes focused on arm strength for propulsion. Ideal for paraplegics, quadriplegics or amputees. Call us today to learn more.

Learn More With Our How-To Videos.

Scan the QR Code with your phone and you will be linked to our Ready to Ride Video Series.



Customized Options

To help get the most out of your cycling experience, we offer additional items and accessories for your customization needs. These items can be ordered online or by calling us at 1-800-661-9915.

Don't see something you need or want?

Give us a call! We pride ourselves on making your experience as enjoyable as possible, and are constantly developing new products and items based on our clients' requests. The item you are looking for might be in development right now!




NEOPRENE CHEST HARNESS

To accommodate your child's growth, we offer chest harnesses in a variety of sizes (medium, large and extra large). Check online for pricing and stock availability. *Also ask about our overnight delivery to ensure they never have to stop enjoying outdoor adventures and fresh air.*

STATIONARY TRAINER

Don't let the weather slow down your exercise program. Take advantage of our Indoor Stationary Trainer and you can ride each and every day, no matter what the weather looks like outside. Ask us about shipping options to determine the best means for delivery.

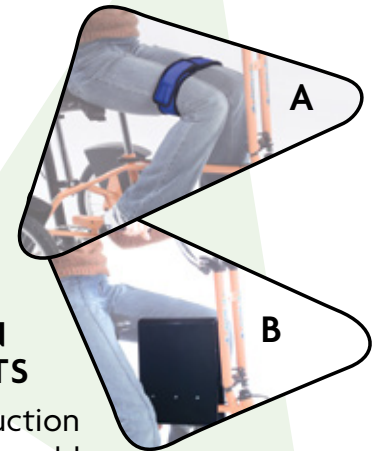

ABDUCTOR /ADDUCTION LEG SUPPORTS

Padded leg adduction straps (A) or removable abductor leg dividers (B) help individual needs to maximize each bike ride. Call us today or order online.

COMMUNICATION TRAY

Now non-verbal riders can carry their assistive technology devices for communicating with those around them. Our handy communication tray can mount firmly to the bike when used with the handlebar system in the locked position.



Accessories

Looking for something special? Listed below are some of the many items we feature on our website. Visit us at www.FreedomConcepts.com to learn more.



MultiTool



Rear Platforms



- Safety Flag
- Wheel Reflectors
- Squeaky Animal Horn or Bells
- Velcro Connector Gloves
- Bike Cover
- Tool Pouch
- Water Bottle & Holder
- Tool Kit in Water Bottle Shaped Case
- MultiTool (A bike's best friend)
- Cane/Crutch Holder
- Oxygen Tank Holder
- Sun Shade Canopy
- Communication Tray
- Locking Hand Brake
- Pedal Blocks
- Rear Platforms and more




Bike Cover



Cane/Crutch Holder



Sun Shade Canopy

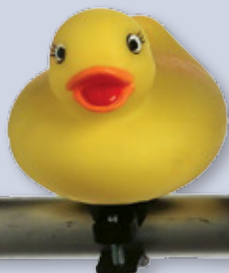

Squeaky Animal Horns



This warranty is made expressly in place of all other guarantees or warranties, expressed or implied, statutory or otherwise, including without limitation those with respect to quality, merchantability, or fitness for a particular purpose.

In no event shall Freedom Concepts be responsible for any losses or damages or any other type of liability including without limitation economic or consequential losses or damages, indirect, incidental or punitive damages, personal injury, or damage.

© 2013
FREEDOM CONCEPTS INC.
00004 (30/05/13)

Freedom Concepts Warranty



Freedom Concepts Inc. (FCI) is pleased to offer quality mobility devices we custom manufacture. All component parts of our devices are inspected at various stages of production, and each product is subjected to a rigorous final inspection before being shipped. Freedom Concepts provides a Standard Lifetime Warranty on all product frames. A one-year warranty is offered on parts and accessories, one-year on Essential foam seating, and three-years on TEC foam seating. The warranty commences on the date the product is shipped to the customer. This warranty does not include repair or replacement required as a result of misuse, abuse, use for any purpose other than intended, damage resulting from commercial or institutional use, or from normal wear and tear. The defective part(s) must be returned through an authorized dealer. If the warranty claim is approved by Freedom Concepts, the defective parts will be replaced, however, shipping costs will incur. Repair by any unauthorized service representative relieves Freedom Concepts of further liability, or obligation under this warranty. The sole obligation of Freedom Concepts with respect to its products and parts and this warranty shall be, at the sole option of Freedom Concepts, to repair or replace the part or product if the warranty claim is approved by Freedom Concepts, at its sole discretion. The foregoing is the customer's sole and exclusive remedy with respect to this warranty.

All Freedom Concepts mobility devices have a 30-day Satisfaction Guarantee not including the return shipping costs, which are the responsibility of the customer.



Register online at: www.freedomconcepts.com/warranty

All products and services are subject to change at anytime without notice, at the sole discretion of us. We reserve the right to discontinue any product at any time.



Freedom Concepts was born from a special request to build a bike for a special needs child. This one bike was such a huge success that it opened the door to many new requests and the birth of a company. Today, bikes from Freedom Concepts can be found throughout the world. We continue to innovate and develop not only bikes, but other unique items like Chill-Out Chairs, climbers and strollers*. Visit us online to see our latest creations or give us a call.

"Give us a need, we will find a concept."

Ken Vanstraelen
PRESIDENT

Help Us Help Others And Be Rewarded!

We truly appreciate all the joy and happiness we can bring to others. Nothing is more gratifying then when we receive referrals from those who recommend our products. To thank all who refer new customers to us, we offer a reward program. If you recommend one of our Adaptive Bikes or a Chill-Out Chair to a customer and it results in a sale, you will receive *a check for \$250.00!*

Simply fill out the following form and either fax, email, or go online with us. Upon confirmation, a check will be in the mail. Once again, thank you for your support in helping us make life enjoyable for all!

Original Purchaser's Name:
(YOUR NAME)

Your Bike Serial #

Your Email:

Referral Name:

Their Address:

Their City:

Their Prov/State:

Their Postal Code/Zip Code:

Their Phone Number(s):

\$250.00



Phone: 1 (800) 661.9915 Fax: 1 (204) 654.1149 www.freedomconcepts.com

CHECK
IT OUT
TODAY!

CHILL-OUT

Rock'er Roll'er

ALTERNATIVE SEATING™

Stimulating touch, sight, and smell, Chill-Out Chairs are a suitable alternative seating option for a wide range of disabilities. Built using a combination of high-grade foams, both The *Rock'er* and The *Roll'er* provide targeted pressure on the body and comfortably hug individuals in place, without the need for restraints. Each chair comes with a multi-purpose ottoman that can offer leg support, be used as additional seating, or work as a side table. Chill-Out Chairs are available in a variety of fabrics, options, and sizes. Visit our website to learn more or call us today to place an order.



1.800.661.9915
www.freedomconcepts.com

Great for at Home, Schools, Therapy Centers and Hospitals.