GUARDIAN

GUARDIAN® SAFE-T-POLE USER INSTRUCTIONS

Model: 98120

Please follow these instructions carefully before using your new Safe-T-Pole in order to familiarize yourself with the safe and correct usage.

Adjustment range: 7' 9" – 8' 5.5"

⚠ WARNING

Please read carefully before installation and use.

Failure to comply may result in serious injury.

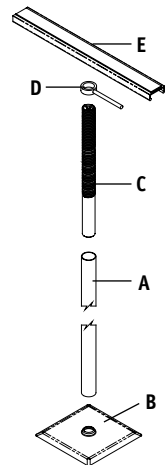
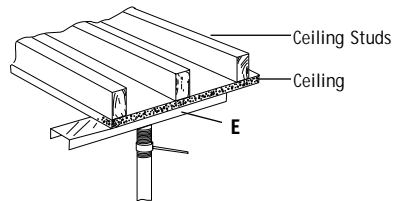
- 1. Adjusting collar must be tightened with sufficient force to prevent any movement during use, especially when being installed on carpeted floors.**
- 2. Ceiling and floor contact surfaces must be structurally sound, dry and free of dirt and debris. Make sure acoustic or textured ceilings are smooth at area where ceiling bracket will rest.**
- 3. Mount ceiling bracket (E) at right angles to ceiling studs to avoid ceiling and floor damage from excessive tightening.**
- 4. Check adjustment periodically to assure safe use of your Safe-T-Pole.**
- 5. Not to be used with suspended ceilings.**

INSTALLATION

Tool Required: Phillips Screwdriver

Parts List:

PolePart A
Floor Mounting Plate.....Part B
Threaded Adjusting PipePart C
Adjusting CollarPart D
Ceiling BracketPart E

- A. Screw the collar (D) onto the pipe (C). Tighten the screw in the bracket (E) into the nut inside of the pipe (C).**
- B. Place floor mounting plate (B) in the approximate floor position desired. Insert the pole (A) onto the receiver lip of the floor mounting plate (B).**
- C. Determine approximate distance from top of pole to ceiling and adjust the adjusting collar (D) on threaded pipe, slightly less than pole-to-ceiling distance.**
- D. Hold the ceiling bracket (E) against the ceiling. Turn the adjusting collar (D), adjusting the pipe until the Safe-T-Pole is sufficiently rigid.**

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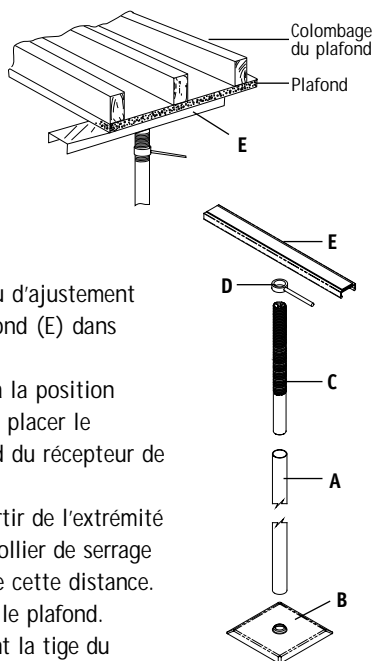
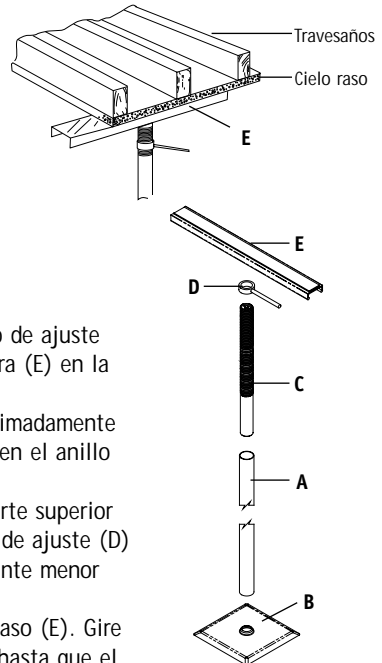
LIFETIME LIMITED WARRANTY

Your Guardian® product is warranted to be free of defects in materials and workmanship. This warranty is limited to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

In the event of a defect covered by this warranty, we will at our option repair or replace the product. For warranty service please contact the retailer where the product was purchased. If you have a question about your Guardian® brand product or this warranty, please call us toll-free at 800.333.4000.

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©2002, Sunrise Medical 5.02