# POWERING UP

---

# CONNECTING WRAPS

---


**IMPORTANT:** Read complete indications, contraindications, cautions, and warnings in User Manual before using this product

# MED4 ELITE™


## QUICK REFERENCE

This Quick Reference is for use after the Med4 Elite has been properly set-up in the clinical environment and is completely filled with distilled water.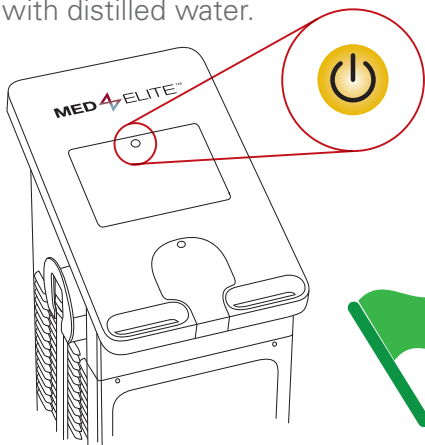

### ON/OFF


After you switch the power on, press the amber On/Off button. The Med4 Elite operating system takes up to 60 seconds to load.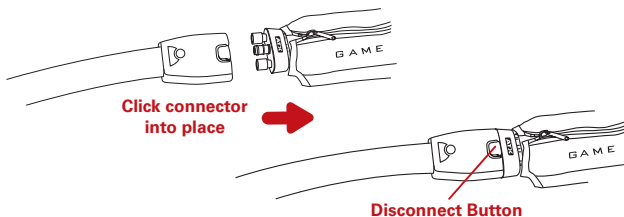

## CONNECT WRAP

Connect the desired Wrap to the connector Hose. The Hoses are marked with Patient Icons.

NOTE: Always return the Hose to the correct Hose holster and wrap the Hose around the Hose hanger when not in use.

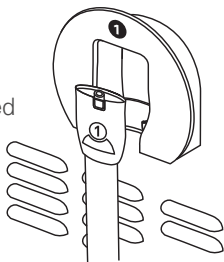# SELECTING

---

# A THERAPY

FOUR THERAPY OPTIONS

COLD  
HEAT

COMPRESSION-ONLY

RAPID CONTRAST


**IMPORTANT:** Read complete indications, contraindications, cautions, and warnings in User Manual before using this product

# MED4 ELITE™

## QUICK REFERENCE


1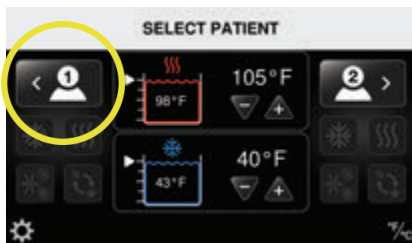

## SELECT PATIENT

After properly connecting the Wrap, select the Patient number that corresponds with the number on the Connector Hose.

2

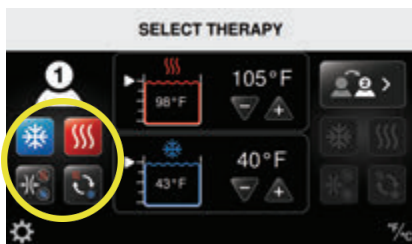## SELECT THERAPY

The Select Therapy screen appears with four active therapy icons. Choose the desired therapy type to begin setting it up.

## FOUR THERAPY OPTIONS

Cold  
Therapy

Heat  
Therapy

Compression-  
Only  
Therapy

Rapid  
Contrast  
Therapy

# STARTING A THERAPY

**IMPORTANT:** Read complete indications, contraindications, cautions, and warnings in User Manual before using this product

# MED4 ELITE™

## QUICK REFERENCE

### 3

### THERAPY SETTINGS

After selecting a therapy type, choose therapy settings:

TIME  
SETTINGS

COMPRESSION  
LEVELS

HEAT RESERVOIR  
TEMPERATURE  
SETTINGS

SWITCH  
PATIENT

SNOOZE SETTINGS  
(COLD THERAPY ONLY)

The system's master Snooze feature is disabled by default. Enable it in System Settings.

COLD RESERVOIR  
TEMPERATURE  
SETTINGS

°F/°C

TEMPERATURE  
SCALE

Use the decrease and increase buttons to adjust the various settings including temperature, time, and compression levels.

### 4

### START THERAPY

Press the green Start Button when you're ready to begin the therapy session.

Once a therapy has started, you may Pause and/or Stop at any time.

# PAUSING CHANGING, OR STOPPING THERAPY

**IMPORTANT:** Read complete indications, contraindications, cautions, and warnings in User Manual before using this product

# MED4 ELITE™

## QUICK REFERENCE

### PAUSING THERAPY SETTINGS


You may pause a therapy at any time. To resume the paused therapy without any adjustments, press the Start button.

1. Press the Pause button

2. Press the Start button to resume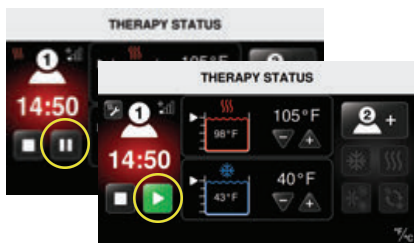

### CHANGING THERAPY SETTINGS

You may change therapy session settings only while treatment is paused.


1. Press the Pause button

2. Press the Adjust Therapy Settings icon and adjust settings as desired

3. Press the Start button to resume with new settings

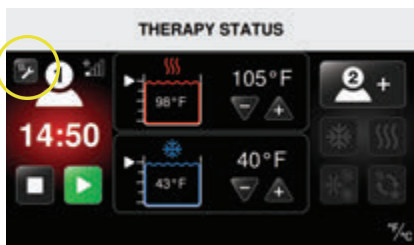### STOPPING A THERAPY

You may stop a therapy at any time. To stop, press the Stop button. The therapy session stops and the Select Therapy screen appears. The therapy settings cannot be adjusted. A new therapy session can then be started with new settings.

# TWO PATIENT THERAPY

**IMPORTANT:** Read complete indications, contraindications, cautions, and warnings in User Manual before using this product

# MED4 ELITE™

## QUICK REFERENCE

### TWO PATIENTS / SAME THERAPY TYPE

For two patient therapy sessions, the heat and cold reservoirs are shared between patients. Therefore, the temperatures will be the same for both patients.

When setting up a therapy for the second patient that uses the same reservoir as the first patient, the therapy temperature selection

(+ or -) buttons will be inactive. The reservoir temperature will remain at the temperature set for the first patient. If a different temperature is desired, all therapies must first be paused in order to adjust the reservoir temperature.

### TO ADJUST THE RESERVOIR TEMPERATURE:

1. Pause treatment on the first patient, and adjust the temperature (**This new temperature will be the same for both patients.**)
2. Resume therapy for the first patient by pressing the Start button
3. Now set up the second patient and Press the Start button to begin therapy

### TWO PATIENTS / DIFFERENT THERAPY TYPE

1. Connect the unused Hose to the second patient
2. Select Patient 1 or Patient 2 on the Select Patient screen to match the Hose selected
3. Set up the second patient therapy and Press the Start button to begin therapy

# TROUBLE-

---

# SHOOTING

**IMPORTANT:** Read complete indications, contraindications, cautions, and warnings in User Manual before using this product

# MED4 ELITE™

QUICK REFERENCE

## ALARMS

NOTE: Always use distilled water when filling the reservoirs.

## CONSULT USER MANUAL

Consult User Manual for troubleshooting alarms

## ERROR CODES

If the Med4 Elite displays an error message during a therapy session, the Control Unit will turn off the power supply. Make note of the error code, and in the U.S. contact Game Ready Technical Support at 1.888.GameReady (1.888.426.3732) or 1.510.868.2100. From outside the U.S., contact your local distributor.

## FOR REPAIR

In the event the Med4 Elite requires repair, work must be done by a Game Ready authorized agent. Contact Game Ready & Technical Support at:

 **1.888.GameReady**

(1.888.426.3732) or 1.510.868.2100.

From outside the U.S., contact your local distributor.

EC REP  
EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

COOLSYSTEMS®, INC.  
DBA Game Ready®  
1800 Sutter Street, Suite 500  
Concord, CA 94520 USA

1.888.GAMEREADY +1.510.868.2100

[www.gameready.com](http://www.gameready.com)

[www.gameready.com/patents](http://www.gameready.com/patents)

EMERGO AUSTRALIA  
201 Sussex Street  
Darling Park, Tower II, Level 20  
Sydney NSW 2000 Australia

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