MED4 ELITE™

QUICK START

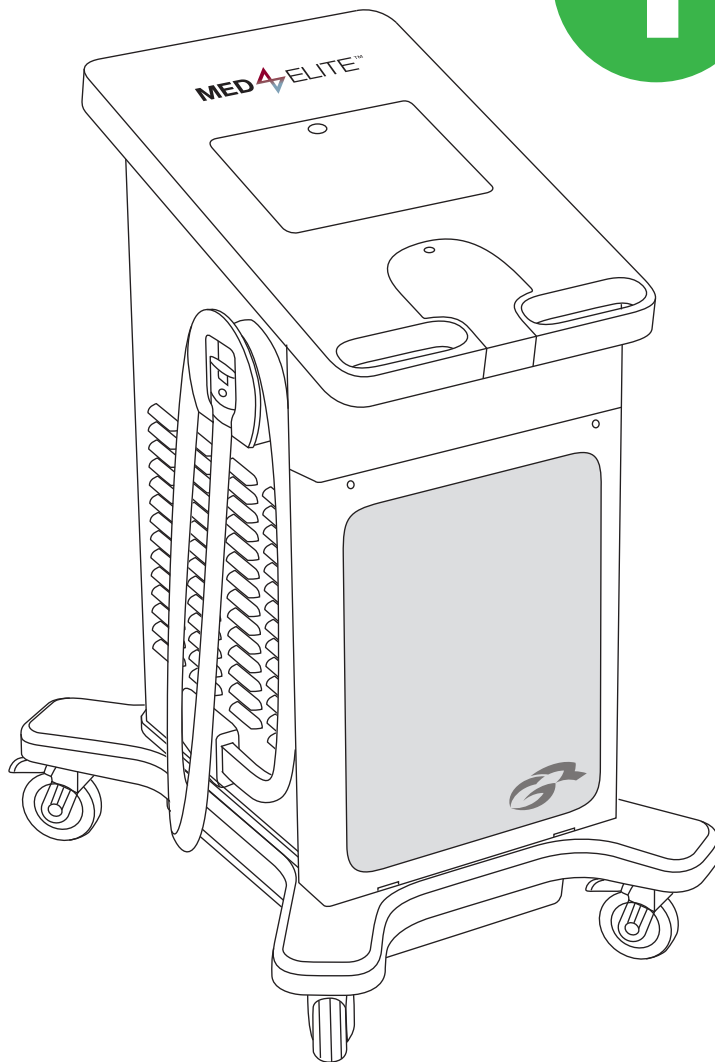
WELCOME TO MED4 ELITE

The Med4 Elite is a multi-modality recovery device for professional users. The system delivers iceless cold therapy, heat therapy, rapid contrast (alternating heat and cold) therapy, and intermittent pneumatic compression therapy providing healthcare practitioners with flexible treatment options for different patients, procedures, injuries and stages of rehabilitation.

START HERE

1

WARNING: It is mandatory to fully read and understand the User Manual before using the device. Failure to follow operating instructions could result in serious injury

IMPORTANT: Read complete indications, contraindications, cautions, and warnings before using this product

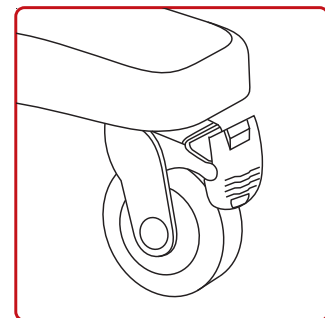
Use distilled water only. DO NOT use deionized or tap water. Failure to use distilled water may void the warranty of the device.

SETTING UP


Use the handles to move the Control Unit into place. The Control Unit should be set up:

- within six feet of a power outlet with a dedicated electrical circuit or enough power to supply 1200 watts
- with adequate access to the On/Off Switch on the back of the unit and with the power cord unobstructed
- on level ground
- indoors only
- in a dry location
- so that vents on all sides are unobstructed
- free of trip hazards

! Once the device is located in a suitable location, lock the casters in place by pressing down on all four caster locks until they click into the down and locked position.

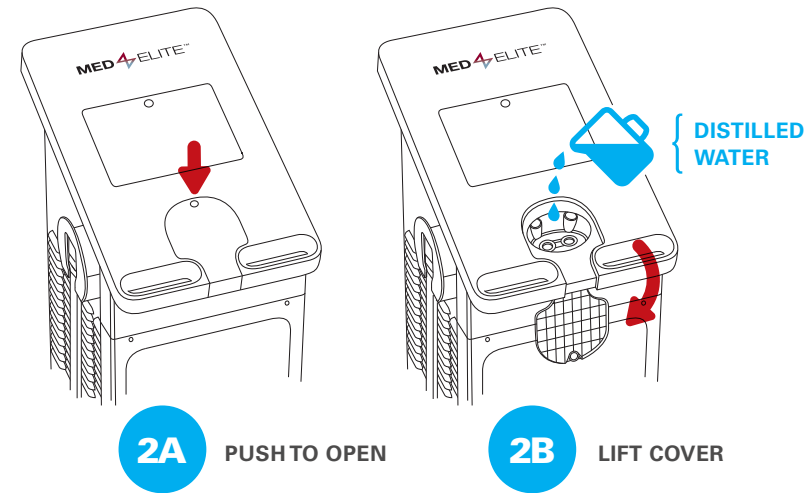
Game Ready Technical Support: 1.888.426.3732


2A PUSH TO OPEN **2B** LIFT COVER

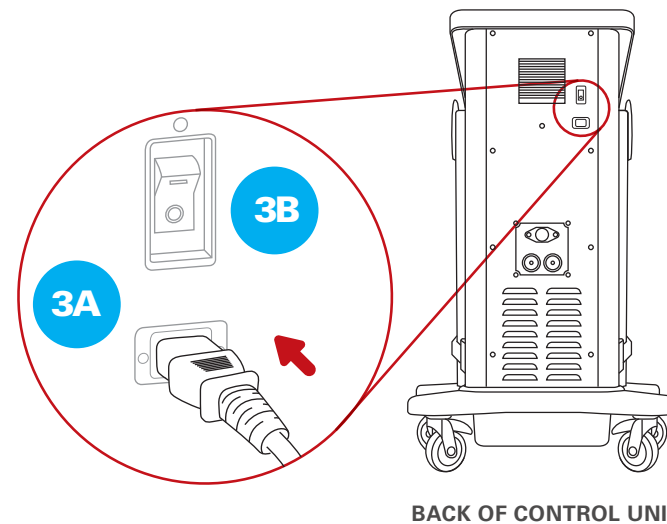! To prevent damage to your Control Unit, you must fill the reservoirs with **distilled** water BEFORE turning it on. If the Control Unit is turned on without water, or with an insufficient amount of water, you will be prompted to add water before continuing.

BACK OF CONTROL UNIT

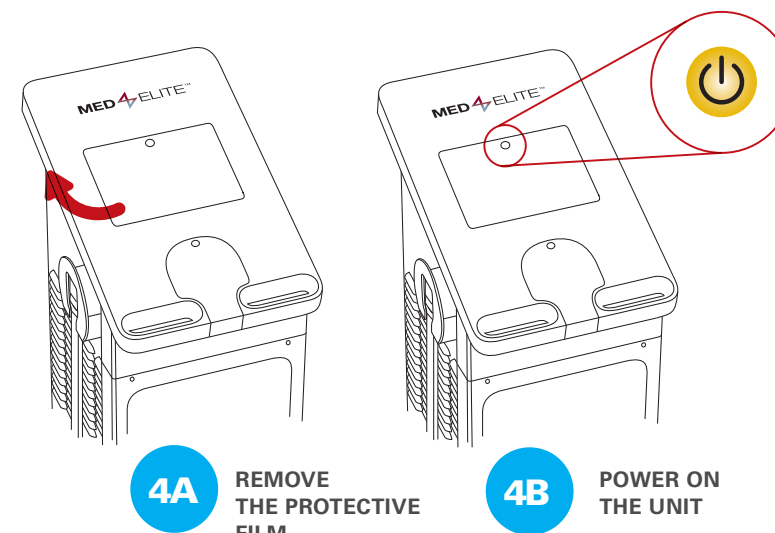4A REMOVE THE PROTECTIVE FILM **4B** POWER ON THE UNIT

2

FILL RESERVOIRS

The Med4 Elite ships with the reservoirs empty. You will need to fill the reservoirs prior to use. Two gallons (eight liters) of distilled water are needed to completely fill the reservoirs.

When finished filling the reservoirs, close the cover. Press firmly to engage the latch.

3

POWERING UP

After you have successfully filled the reservoirs, attach the power cord to the back of the Control Unit and plug the power cord into the appropriate power outlet. **(3A)** Turn on the power located on the back of the Control Unit. **(3B)**

4

ON/OFF

After you switch the power on, remove the protective film and discard. **(4A)** Press the amber On/Off button. The Med4 Elite operating system takes up to 60 seconds to load. **(4B)**

5

SELECT LANGUAGE

On the initial boot, the Select Language screen appears. Press the up or down arrows to scroll through the available languages. Follow the on screen prompts to confirm.

PRIOR TO OPERATING THE MED4 ELITE

REMOVE AND READ THESE INSTRUCTIONS FOR INITIAL SETUP

Contact Game Ready Technical Support at:

(1.888.426.3732) or 1.510.868.2100.

From outside the U.S., contact your local distributor.

 COOLSYSTEMS®, INC.
DBA Game Ready®
1800 Sutter Street, Suite 500
Concord, CA 94520 USA
1.888.GAMEREADY +1.510.868.2100

www.gameready.com
www.gameready.com/patents

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

EMERGO AUSTRALIA
201 Sussex Street
Darling Park, Tower II, Level 20
Sydney NSW 2000
Australia

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© 2018 CoolSystems, Inc. جميع الحقوق محفوظة.