



Grid Iron XR Walker

Single Patient Use Only

Rx Only



	Standard Walker	Low Top Walker
X-Small	15570004	15560004
Small	15570005	15560005
Medium	15570006	15560006
Large	15570007	15560007
X-Large	15570008	N/A

- Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or prolonged pain while using this product.
- Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

IMPORTANT INFORMATION

Please read all instructions before use. Correct application is essential to proper functioning of product.

INDICATIONS

Grade 2 and 3 ankle sprains; stable fractures of tibia, fibula, talus and tarsus; post operative immobilization (soft tissue reconstructions and open reduction internal fixation); metatarsal fractures

PRECAUTIONS

- This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which they are responsible. The physician or practitioner is responsible for providing use instructions and precautions to other healthcare practitioners, care providers involved in the patients' care and the patient.

INSTRUCTIONS FOR USE

1. Remove inner liner/wrap from brace (1a). Position foot and lower leg into the liner (1b).




2. Position tongue pad over top of foot before securing wrap closed.




3. Secure wrap snugly around limb. Fasten with "hook" fasteners to the outside of softgood (3a). Insert foot and lower leg into brace (3b).



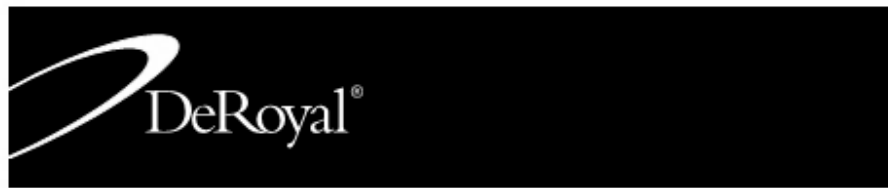

4. Carefully center the uprights over the ankle to ensure a comfortable fit and proper support (4a). Remove plastic covers from uprights and secure hook fastener to liner (4b).

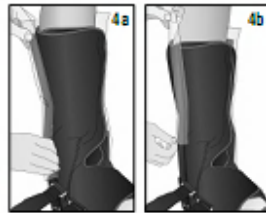

5. Fasten walker using hook and loop straps. Begin fastening the boot straps at the bottom (foot) and work your way up the leg. **NOTE:** Readjust as needed for a secure and comfortable fit.



Uprights can be contoured to accommodate tibial variations among different patients.

WASHING INSTRUCTIONS

Softgood should be hand washed using cold water and mild detergent. Air dry.