




WARNINGS

- While removing your Press Switch from your GlassOuse or other devices, pay attention not to damage the wire joining points on the 3.5mm jack connector and do not pull the cable harshly.

PRESS SWITCH 150GR

A durable switch option for users with less physical control. GS06 Press Switch provides 8.63cm activation surface, making the item comfortable enough for both user groups needing either a large or smaller target area to interact. Operated by 150g activation force, GS06 Press Switch has an auditory click and tactile feedback.

Press Switch comes with a 185cm long cable and a 3.5mm mono jack plug-in. Can be used with different mounting options or doublesided tapes.

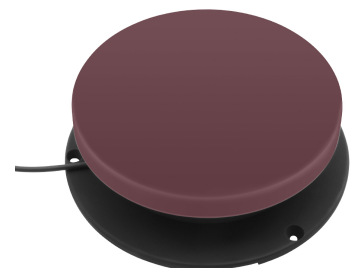

SWITCH CONNECTION

1. Plug GS06 Press Switch to GlassOuse V1.2, V2.0 or any 3.5mm jack adapted device via mono 3.5mm jack connector.

2.1 Place GS06 Press Switch on a flat surface and adjust the position until you feel comfortable enough to do the pressing action before using double-sided tapes to secure the switch.

OR

2.2 Use a mounting option to secure the position of GS06 Press Switch.

GS06 - PRESS SWITCH 150GR

Dimensions: Diameter: 96.9mm x Thickness: 27.55mm

Weight: 68±2g

Cable Length: 185±5cm

Activation Surface: 86.30mm

Activation Force: 150g

DEVICE

