

INSTRUCTIONS:

1. Place user's neck on the orange neck rest at the edge of the basin, laying head on round orange support inside the basin.
2. Use the soft water plug to block the drain hole before filling the basin with water. Water temperature should not exceed 100 degrees.
3. After washing hair, attach the drain hose and remove the water plug to empty the basin.
4. Remove the basin from underneath the user's head.
5. Rinse basin and wipe out any excess water.



The DMI Hair Rinsing Basin is intended to be used with the aid of a caregiver while the user is laying flat in bed.