

SERENE®

Vibrating Alarm Clock

Model VA-3



Thank you for selecting the Serene VA-3, vibrating alarm clock. The VA-3's powerful sound and vibration is designed to wake-up even the heaviest sleeper.



Battery Safety and Disposal

Model VA-3 is supplied with three AA batteries. Please do not dispose of used batteries in the household trash.


Please take the batteries to a retail or community collection point for recycling. For customers living in the European Union please take batteries to a recycle center, or return to factory for proper disposal.

PACKAGE CONTENT CHECK


VA-3 Vibrating alarm clock

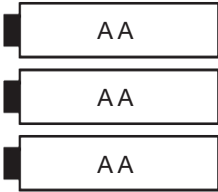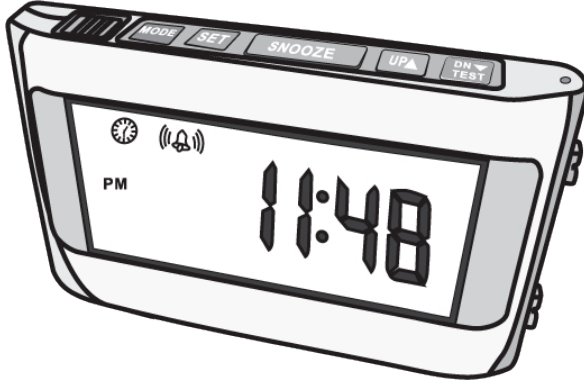

Three AA batteries

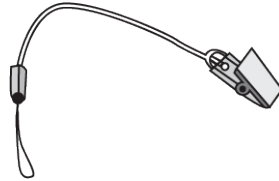


Pillow clip

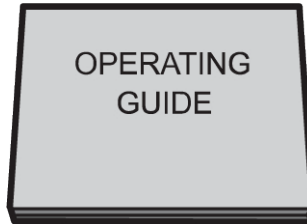

Operating Guide

FEATURE IDENTIFICATIONS

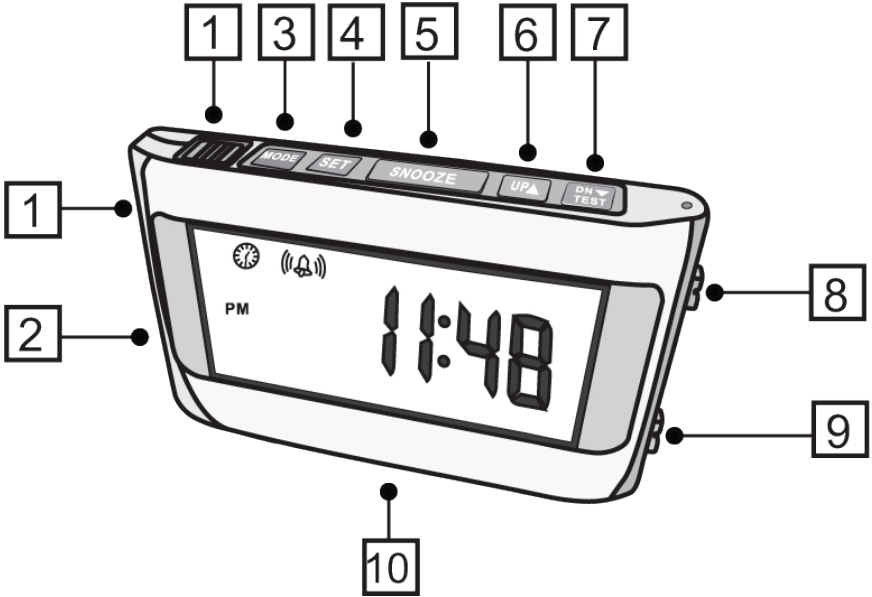

1. Built-in flashlight
2. Alarm mode switch (audio + shaker, shaker, audio, off)
3. MODE (alarm, timer, temperature)
4. SET to select setting choice
5. SNOOZE
6. UP
7. DN/TEST (down or test)
8. Audio alarm tone control (Hi-Med-Lo)
9. Audio alarm volume control (Hi-Med-Lo)
10. Battery Compartment (rear)

BATTERY INSTALLATION

Install three AA alkaline batteries (included).

Please observe battery polarity while installing.

A set of brand new alkaline batteries will last about six to eight months of normal usage.

OPERATION

1. To set time

1. Press and hold SET for 2 seconds
2. Press UP to display AM, PM, 12 or 24-hour clock; press SET to select
3. Press UP or DN to set the Hour, press SET to select. Repeat these steps to set Minute.

2. To set alarm time & snooze

1. Press MODE to go to Alarm mode, press and hold SET for 2 seconds to select
2. Press UP or DN to set the alarm hour, press SET to select. Repeat these steps to set Minute.
3. While SNOZ is flashing, press SET to turn on the snooze function or press UP to turn it off. It will automatically return to the normal clock mode after about 5 seconds.
4. If SNOOZE function is on---when the alarm sounds off, the alarm will continue for 40 seconds and then stop. If the SNOOZE button is pushed any time while the alarm is sounding off, the alarm will stop immediately but it will sound off again in eight minute. You can repeat this cycle for up to seven times. To stop the alarm completely any time, press any button other than the SNOOZE button on top of the clock.

5. If SNOOZE function is off---when the alarm sounds off, press the SNOOZE button will stop the alarm completely.

3. To set alarm on/off & alarm mode

1. Slide the switch to OFF to disable the alarm or to stop the on-going alarm action
2. Slide the switch to AUDIO, SHAKER or ALL (audio + shaker) to select your alarm mode and to set the Alarm On

4. To test your alarm setting

1. Press and hold TEST for 3 seconds, the alarm is triggered for a period of 10 seconds and then returns to the clock mode

5. To use clock as a Timer

1. Press MODE to go to the Timer mode, press and hold SET for 2 seconds to select
2. Press UP or DN to set the desired hour setting, press SET to select. Repeat these steps to set the desired minute setting
3. Press DN to start/stop the timer count-down, (if DN is not pressed within five seconds clock will return to regular time-keeping mode.)
4. When the set time expires, the alarm will sound-off (same alarm mode as the alarm clock setting) for 30 seconds (unless stopped by pressing any of the top keys)

6. To show the room temperature

1. Press MODE to go to the Thermometer mode, the room temperature appears
2. If needed, press UP to set reading in C or F
3. Press MODE to return to the clock mode

SPECIFICATIONS

Functions: clock, alarm clock, timer, thermometer and flash-light

Clock modes: 12 or 24 hours

Alarm modes: audio, vibration, audio & vibration

Thermometer modes: C or F

Size: 118 x 71 x 27 mm; 4.6 x 2.8 x 1.1 inches

Weight: (without batteries): 140 gm; 5 oz.

Power supply: 3 AA alkaline batteries (provided)

Battery life: 8-10 months of normal use

Vibration: 8200+-RPM at .21A

Flasher brightness: 2000 mcd

WARRANTY SERVICE INFORMATION

VA-3 comes with a one-year limited warranty. We warrant during this warranty period, from the date of purchase, to the original consumer, VA-3 to be free from defects in materials and workmanship under intended home use. In the event that VA-3 fails to function properly within one year of original purchase due to defects in materials or workmanship, return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations.

Serene Innovations will either repair or replace the unit (with a refurbished unit or unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase but within the warranty period, there will be a warranty handling charge of \$9.95 for each return. This warranty gives you specific legal rights, and you may also have other legal rights that vary from state to state.



VA-3 is a trade mark of Serene Innovations, Inc.

Serene Innovations, Inc.

14615 Carmenita Road, Norwalk, CA

Tel: 562-407-5400

www.sereneinnovations.com

Copyright 2010