



THE ROHO® HIGH PROFILE® AND LOW PROFILE® CUSHIONS

SKIN PROTECTION

For over 30 years, the ROHO HIGH PROFILE and LOW PROFILE Single Valve cushions have continued to be the market leaders in skin protection seat cushions - they are the benchmark by which all other skin protection seat cushions are compared. The HIGH PROFILE and LOW PROFILE cushions offer an optimal environment for clients with skin integrity issues. The

SHAPE FITTING

cellular design and the soft pliable material allow the client to be immersed into the cushion for maximum pressure redistribution, while decreasing friction and shear. The simple adjustment allows the cushions to fit the shape of the client, minimizing tissue deformation and maximizing blood flow.

PROVEN PERFORMANCE

No other air filled seat cushion can match the proven performance and quality standards of ROHO's DRY FLOATATION® Technology. Nothing feels like a ROHO!

LIGHTWEIGHT



HIGH PROFILE AND LOW PROFILE CUSHIONS

SKIN PROTECTION.

Thirty years of research, experience and refinement of ROHO DRY FLOATATION® technology provides the client with unparalleled skin protection.

SHAPE FITTING.

The unique cellular design of the HIGH PROFILE and LOW PROFILE allows the cushions to constantly conform and track movements of the body. The simple adjustment and conformity of the cells accommodates and meets the unique skin integrity needs of each client throughout the day.

PROVEN PERFORMANCE.

Laboratory tests prove ROHO will outlast other air filled cushions and maintain their effectiveness over time, even after exposure to contaminants and daily wear and tear.

LIGHTWEIGHT.

At only 4.4 (2.0 kg) and 3.2 lbs. (1.5 kg), the HIGH PROFILE and LOW PROFILE cushions are easy to transport and handle, minimizing fatigue of the client.



We highly recommend distribution through a certified rehab professional when ordering and fitting a ROHO Group product.

APPLICATIONS

THE HIGH PROFILE CUSHION IS RECOMMENDED FOR CLIENTS WHO:

- are at high risk for skin breakdown
- have no sensation
- have current impaired skin
- have history of skin breakdown
- need assistance with weight shifts and transfers

THE LOW PROFILE CUSHION IS RECOMMENDED FOR CLIENTS WHO:

- are at moderate risk to high risk for skin breakdown
- have no or partial sensation
- are able to do independent weight shifts
- propel their wheelchair with their feet

FEATURES & BENEFITS

ADJUSTABLE:

Shapes to the individual addressing their specific seating needs.

LIGHTWEIGHT:

Adds little weight to entire mobility system.

WASHABLE:

Cushion is easily cleaned with soap and water
Cover is machine washable.

REPAIRABLE:

Neoprene rubber cushion can be patched or repaired to extend use.

*Also available in Dual Compartment, for front to back or side to side positioning for clients with asymmetrical postures.

SPECIFICATIONS

INCLUDES:

HIGH PROFILE or LOW PROFILE Cushion, cover, hand inflation pump, repair kit and operation instructions

CONSTRUCTION:

Flame Resistant, Neoprene Rubber

SIZES:

Wide Range, to fit chair widths 12" to 28"/30cm to 71cm

WEIGHT:

Varies by size, approximately *4.4 lbs./2.0 kg. for HIGH PROFILE, *3.2 lbs./1.5 kg. for LOW PROFILE (based on 9x9 sizes)

*Varies by size

WARRANTY:

24-month Limited Warranty

WEIGHT LIMIT:

Unlimited¹

MEDICARE PART B CODE:

K0734/5, Adjustable skin protection Wheelchair Cushion

*DUAL COMPARTMENT MEDICARE PART B CODE:

K0736/7, Adjustable skin protection and positioning Wheelchair Cushion

¹ Cushion must be properly sized to client

MORE PRODUCTS FROM ROHO



CONTOUR SELECT® CUSHION



HIGH PROFILE® QUADRO SELECT®

FOR ADDITIONAL INFORMATION, PLEASE CONTACT:

