

Washing Instructions

immedia[®]
a part of Etac

Version 2 2017

The products should be checked regularly each time it is used, and especially after wash or cleaning. Never use a defective product.

3B Board, E-Board, Butterfly, Dolphin, 2Move, PediTurn Hard, NonSlip Square, CloverFoot



4WayGlide, TwinSheet4Glide, MultiGlide, TurningMattress, TransferMattress, GlideCushions, OneWayGlide, Swan, HandyTrans, Nylon Covers, SlingOn, WiseMover, MiniBoard, MultiGlide Glove, Rescue, BedString, OneManSling, Sling, HandyGrip



SupportBelt, SmartMove, Wedge



SatinSheet, In2Sheet



EasyTurn, AutoTurn, PediTurn Soft



EXPLANATION OF SYMBOLS FOR WASHING INSTRUCTIONS



Wash/temperature celsius



No dry cleaning



Do not bleach



Clean the product with a solvent-free cleaning agent with a pH level between 5-9, or a maximum 70% disinfectant solution.



Tumble drying low temperature



Wiping



Ironing, low temperature

PERIODIC INSPECTION

- Make sure that your Manual Transfer system always is in a perfect condition
- After wash - always check materials, stitching, handles, buckles
- If the product shows signs of wear and tear, it must be removed from service immediately

PLEASE NOTE

- Please check that there is no damage to seams or fabric
- Wash at max 80°C and if possible, wash at lower temperature
- Avoid drying for optimal life of the slide material
- Do not use fabric softener or bleach
- Recommended to wash before use