Paraffin Gauze Dressing

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DESCRIPTION

MedSupply Paraffin Gauze Dressing consists of a cotton tulle impregnated with white paraffin. This is a knitted open-weave tulle with a water resistant, non-adherent ointment base. The product is available in various sizes. MedSupply Paraffin Gauze Dressing allows the wound to remain undisturbed, facilitates the drainage of exudate and provides good ventilation to the wound. It prevents the dressing sticking to the wound and thus encourages the natural healing process. The neutral ointment base allows additional wound-specific medication.

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INDICATIONS

MedSupply Paraffin Gauze Dressing is intended to be used in the minor burns treatment and scalds, skin grafts, leg ulcers, lacerations and abrasions, minor burns and surgical wounds.

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CONTRA-INDICATIONS

MedSupply Paraffin Gauze Dressing should not be used in patients known to have sensitivity to any of the dressing components. Do not use in cavity wounds.

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METHOD OF USE

- 1) Before using the dressing, clean the wound area with a wound cleansing agent or normal saline and dry the surrounding skin.
- 2) Choose the appropriate size dressing to cover the wound area.
- 3) Cover and secure the MedSupply Paraffin Gauze Dressing with a secondary dressing.
- 4) The frequency of dressing changes will depend on the patient's condition and the level of exudation. Initially, the dressing may need to be changed every 24 hours. When the exudation decreases, so could the frequency.
- 5) Gently remove the secondary dressing. If the wound appears dry, soak the dressing in sterile saline before removing it.

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WARNINGS

- Before using this product, the types and causes of refractory wounds should be scientifically analyzed and corresponding treatment plans should be made.
- This product is a disposable aseptic product, re-use is strictly prohibited, inner packing damage is prohibited.
- If you have red and swollen skin, itching, rash or other discomfort during use, please stop using it immediately.
- Please use the validity period, read the instructions carefully before use.
- This product is for external use and is not edible. Keep out of reach of children.

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STERILIZATION

The product is sterilized by radiation.

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HOW SUPPLIED

Individually packaged for single use only. Do not use if package is damaged or open.

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STORAGE CONDITION

Store at room temperature. Avoid refrigeration and exposure to high humidity.

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SHELF LIFE

The product shall valid for 3 years, refer to the expiration date which is printed on each package.

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