

# Infant transport mattress warmer



Quick, safe warming for newborns with easy activation

Our infant transport mattress warmer with disc provides warmth during transport of an infant within the hospital or between hospitals. It's recommended for full term infants.

## Instruction for use

1. Locate disc and grasp with thumb and forefingers
2. Bend disc back and forth until crystals begin to form
3. Knead pack for 10–30 seconds to activate throughout and to ensure heat is evenly distributed
4. After pack is activated, indent center with palm to form a small nest
5. Lay infant in nest on the fabric side of the mattress
6. Periodically check the infant's skin and monitor core temperature

## Caution

If contents come in contact with eyes, flush them thoroughly with water and immediately notify the healthcare provider. Do not swallow. If contents leak and come in contact with the skin, wash immediately with water and mild soap. In addition, change affected clothing and bedding. Do not leave infant unattended. For external use only. Do not wrap device with any additional heat source, such as warm washcloth. Do not puncture or strike pack.



MDS1017

Contact your Medline Representative today for questions.

Medline Industries

© 2021 Medline Industries, Inc. All rights reserved. Medline is a registered trademark of Medline Industries, Inc. MKT19W2098008 / e21457 / 78