

# Deluxe Inflatable Bed Shampoo Kit



## Package Includes:

- 1 Inflatable Bed Shampoo Basin
- 1 Inflatable Pillow
- 1 Drain Tube
- 1 PVC Patch Repair Kit
- 1 Instruction Sheet



This inflatable shampoo basin is intended to comfortably support the head and neck of the user during the shampoo process. The DMI Deluxe Inflatable Bed Shampoo Kit is intended to be used with the aid of a caregiver while the user is laying flat in bed.

**NOTE:** Please inspect all parts carefully for cracks prior to use.

## INSTRUCTIONS:

1. Locate the inflation valves found on both the upper and lower portions of the basin. Inflate by mouth or with an air pump (air pump non included).  
Optional: For extra comfort, inflate pillow at this time as well.
2. With the user laying flat gently place the basin under the user's head with their neck resting gently in the outside opening.
3. Connect the larger end of the drain hose to the drain outlet on the underside of the basin. Be sure the drain hose is fully inserted until it cannot go in any further. Place the other end of the drain hose into a wastewater container. Be sure the wastewater container is positioned lower than the basin. Close the drain plug.
4. Wash the user's hair as usual
5. When finished shampooing, open the drain plug, and let all the wastewater completely drain from the basin.
6. Dry user's hair and remove the basin from underneath the user's head.
7. Rinse basin and wipe out any excess water.
8. Leave basin inflated for several hours until it is completely dry before deflating and folding for storage.