

RiJe® Petite Cup Instruction for Use
Model RJP

Indications

For people suffering with difficulty in swallowing (dysphagia), developmental disabilities, motor impairment, or requiring predetermined doses of liquid.


Precautions and Warnings

- Please seek the advice of a medical professional prior to using this cup.
- Instructions must be read prior to use.
- **CHOKING HAZARD. Small Parts. Adult Supervision Required.**
- Wash the cup prior to use.
- RiJe® Petite Cup is designed to be used with thin and slightly thick liquids (IDDSI Standards 0 and 1). Carbonated beverages will render RiJe® Petite Cup ineffective for delivering an accurate dosage.
- Do not use RiJe® Petite Cup for boiling liquids in a microwave.
- Hold RiJe® Petit Cup upside down when opening to prevent spillage.

Features

- Dispenses controlled boluses of thin or slightly thick liquids;
- Allows pre-setting the bolus volume from 4ml to 12ml. The volume range outlined in the table below;
- Holds up to 150 ml of liquid;
- Allows drinking the controlled liquid boluses from the cup's rim or through a regular straw.
- Good nose clearance;
- Ergonomic handles provide secure grip;
- Spill-control does not allow spilling more than one bolus of liquid if the cup is dropped;
- Stable base;
- Dishwasher safe;
- BPA Free.

PARTS

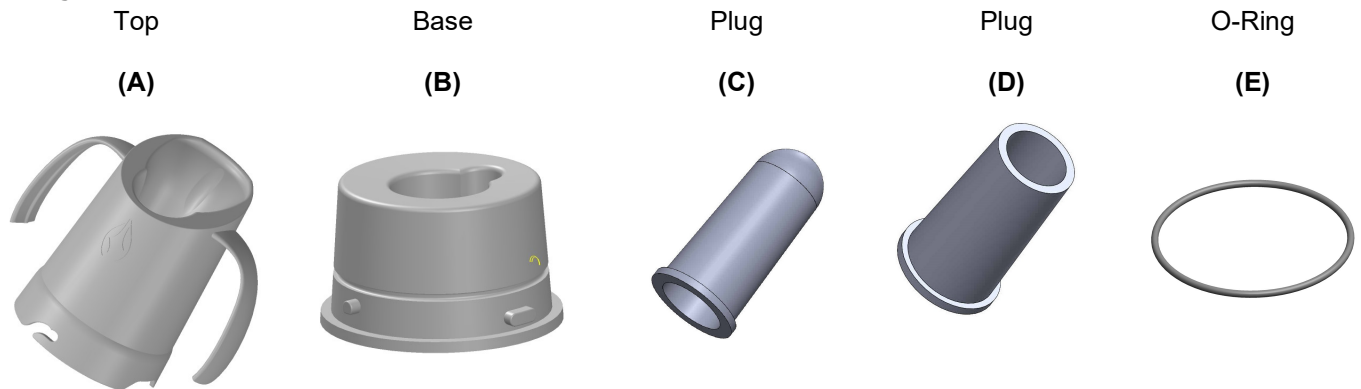

**Choosing correct Plug for Thin Liquids (IDDSI Standard 0) or Slightly Thick Liquids (IDDSI Standard 1).
Range of Dispensed Volumes.**

Note:


- The Plug should be chosen according to the viscosity (thickness) of a liquid being used at the moment.
- **The viscosity (thickness) of a slightly thick liquid is highly dependent on temperature of the liquid, the type of thickening product (if any) is being used, and the time it was allowed to thicken.**

Liquid's Viscosity	Dispensed Volumes		
	Using Plug C	Using Plug D	Using RiJe Petite Cup without Plug
Thin Liquids Cold or Hot. IDDSI Level 0 Thin Liquids	Dispenses 5 ml	Dispenses 8 ml	Dispenses 12 ml
Slightly Thick Liquids IDDSI Level 1 Slightly Thick	NOT TO USE	Dispenses 4-5 ml	Dispenses 8 ml

Assembly

Step 1:

Insert desirable Plug firmly by **PRESSING DOWN**. Use either **C** or **D** depending on the thickness of the liquid used at the moment (please see table above).

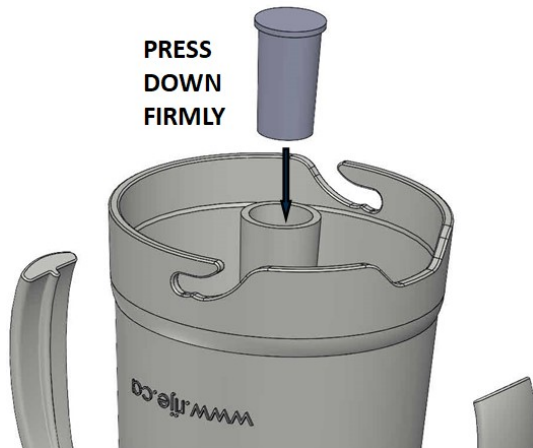


Step 2:

Fill liquid **ONLY** into Upper Cap A.




Step 3:

Put Base **B** over Upper Cap **A**. Make sure that the sealing O-ring sits in the groove of Base **B**.

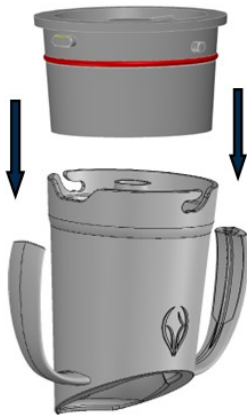

Step 4:

To lock the cup, turn Base **B** against Upper Cap **A**. Direct each knob to its corresponding slit as shown below. Turn tight until the knobs **reach to the end of their slits**.

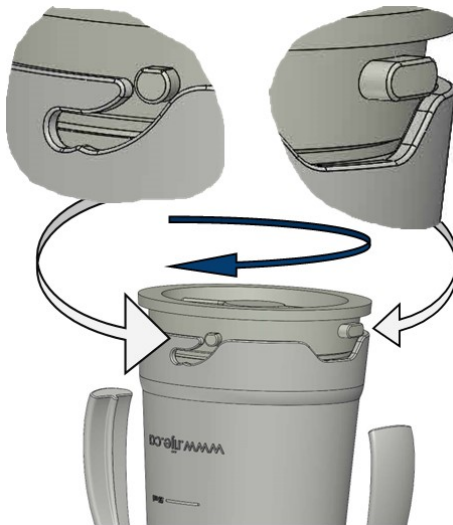

Step 5:

The cup delivers liquids in an upright position. **Put the cup upright after each sip** for 2-3 seconds to allow next portion to be delivered.



Correct drinking position.		Incorrect drinking position
Drinking controlled volumes from an open cup.	When drinking controlled volumes using a regular straw, always slightly incline the cup to ensure accurate dosage.	When using a straw slightly incline the cup to ensure accurate dosage.
