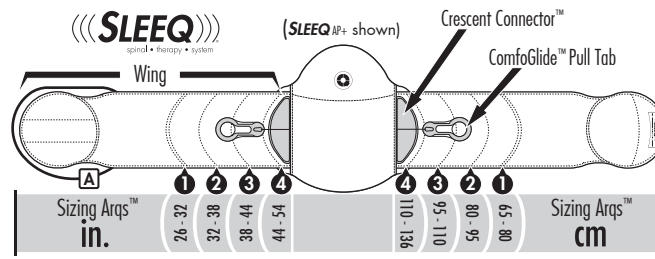
SLEEQ

spinal • therapy • system

INSTRUCTIONS FOR USE

SLEEQ AP SLEEQ AP+ SLEEQ APL

4615 Shepard Street, Bakersfield, CA. 93313, USA
800.432.3466 ThuasneUSA.com

1.0

Sizing

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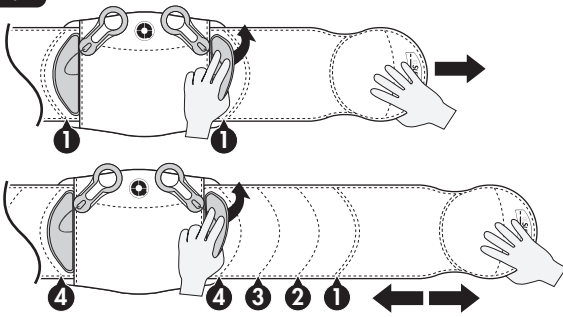
Größe

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1.1

Adjust
Move right wing in or out and align Crescent Connector to appropriate Sizing Arq. Repeat on left wing.

SLEEQ adjusts to fit a vast majority of patients. Measurement is closely related to waist circumference. Four Sizing Arqs (1, 2, 3, 4) provide landmarks for waist circumference and proper alignment. These only act as guides as there is infinite adjustability and customization options within Sizing Arq range.

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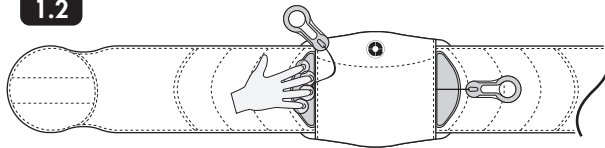
1.2

Secure
Firmly secure both Crescent Connectors to each wing. Return ComfoGlide Pull Tabs to ready position. Not quite right? No problem, re-adjust for best fit.

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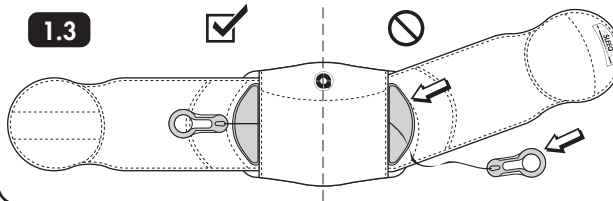
1.3

Check
Confirm Crescent Connectors are firmly secured, aligned to Sizing Arqs, and ComfoGlide Pull Tabs are in ready position.

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DO NOT IRON

Warnings

- Carefully read use/care instructions and warnings prior to use.
- If increased pain, swelling, sensation changes, or any adverse reactions are experienced while using this product, immediately consult your medical professional.
- For single patient use only.

Contraindications:

- SLEEQ braces are contraindicated for women who are pregnant.
- SLEEQ braces are contraindicated for persons with circulatory, pulmonary, cardiovascular or other conditions where increased abdominal pressure would be of concern.

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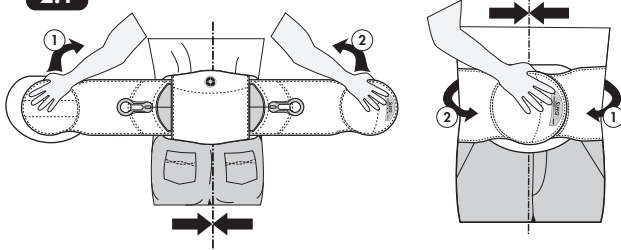
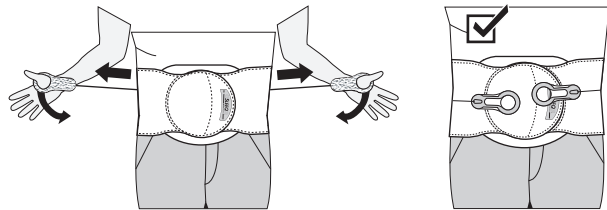
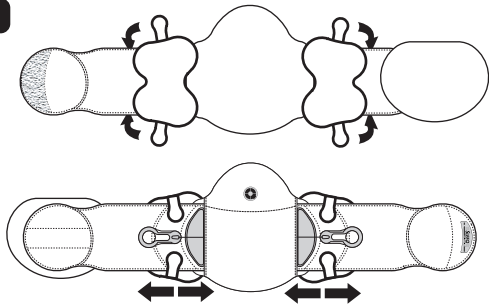
Serrare

2.3

SLEEQ APL Setup
Attach lateral panels.

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
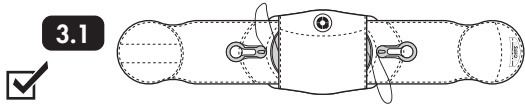
3.1

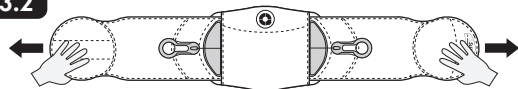3.2


When removing SLEEQ, always return ComfoGlide Pull Tabs to ready position and then pull each Wing outward to reset the TruGlide™ Tightening mechanism.

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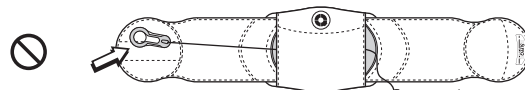(SLEEQ AP shown)

LIMITED WARRANTY

Townsend Design warrants to the original purchaser that this product is free from defects in materials and workmanship. Normal wear and tear during the use of a product is not considered a defect. This Limited Warranty is effective from the date of delivery to the original purchaser. This warranty does not apply if this product; (1) has been altered in any way; (2) was not purchased from Townsend Design or one of its authorized distributors; (3) is not used in accordance with Townsend Design's Instructions for Use (IFU). Under this Limited Warranty Townsend Design's sole obligation shall be to repair or replace the defective product or associated part(s) at no charge. This Limited Warranty does not cover damage as a result of mishandling, accidents,

improper use or alteration of the product. This warranty is in place of all other warranties, either express or implied, including but not limited to warranties of merchantability or fitness for a particular purpose. Townsend Design shall not be liable under this or any implied warranty for any personal injury, property damage, or special, incidental, or consequential damages arising out of, related, or incidental to the use of the product even if Townsend Design has been informed of the prospect of such possible loss or damage, unless state law otherwise prohibits this exclusion. User should seek advice from their healthcare provider immediately if user encounters any kind of adverse reaction related to the use of this product. User should consult with their healthcare provider with any questions or concerns relating to the proper use of this product for their specific medical condition. IMPORTANT-Read all Instructions for Use (IFU) and any other associated materials to ensure proper use.