

# Feeder Seat<sup>®</sup> Harness Replacement

WEB END [A1]

BUCKLE [A2]




STRAP A

CHEST STRAP [C]




1> Lay out the straps. You should find that you have 2 separate pieces: a long strap with a buckle and a chest strap.

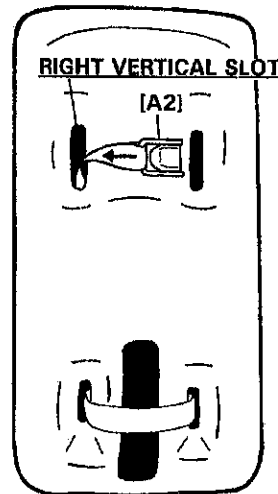

RIGHT VERTICAL SLOT

[A2]


2> To attach the long strap take web end [A1] and thread it from the back of the seat through the right vertical slot. Make sure buckle [A2] is facing outward.

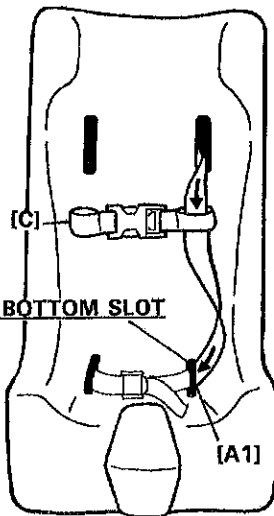

3> Then bring web end [A1] through the chest strap [C] and down through the right bottom slot.


RIGHT BOTTOM SLOT

[A1]

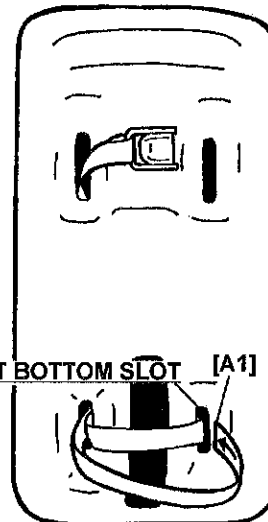

4> Bring the strap around the back of the seat and through the left bottom slot.

LEFT BOTTOM SLOT [A1]

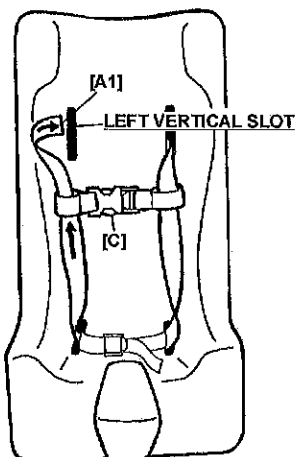


5> Then bring the web end [A1] through the other side of the chest strap [C] and through the left vertical slot.

[A1]

LEFT VERTICAL SLOT

[C]

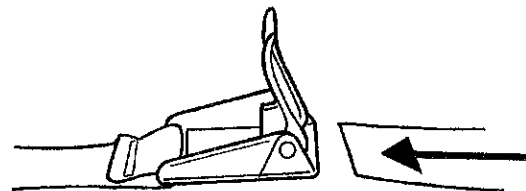

6> Fasten the buckle [A2] by lifting the tongue on the buckle and sliding the web end [A1] through the buckle then pushing down the tongue to secure.

LIT24

MADE IN THE USA

**PATTERSON**  
MEDICAL

Patterson Medical  
28100 Torch Parkway, Suite 700  
Warrenville, IL 60555-3938

Call toll free:  
**1-800-323-5547**  
In Canada call 1-800-665-9200

Fax toll free:  
**1-800-547-4333**  
In Canada fax 1-905-566-9207

**homecraft**  
AbilityOne

Authorized European Representative:  
Homecraft, Ltd.  
Shelley Close, Lowmoor Road  
Industrial Estate, Kirkby-in-Ashfield, Nottinghamshire, NG 17 7ET, UK  
Telephone: 44 1623 759146 Facsimile: 44 1623 726886

CE