



Transfer Bench


AVAILABLE COLORS:



B3300

USER MANUAL - TRANSFER BENCH

GENERAL EXPLANATION OF YOUR TRANSFER BENCH

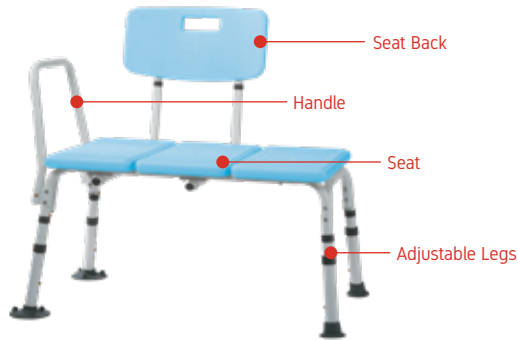

USER GUIDE

⚠ Prior to use please read all instructions.

We recommend you keep this User Guide in a safe place for future reference.

TABLE OF CONTENTS

SPECIFICATIONS	2	<ul style="list-style-type: none">• Easy snap together assembly, no tools needed!• Blow molded bench provides comfort and strength
SET UP	3	<ul style="list-style-type: none">• Lightweight aluminium frame
CLEANING	3	<ul style="list-style-type: none">• Angled legs and suction tips provide additional stability• Adjustable height
WARRANTY	3	



SPECIFICATIONS

NAME	Transfer Bench
WEIGHT	14 lbs
SEAT HEIGHT	18.5" TO 22.5"
SEAT DIMENSIONS	18" W X 16" D
UNIT DIMENSIONS	21" x 18"
WEIGHT CAPACITY	300 lbs


SET UP

Bench Assembly

- 1 Place the bench with the seat facing down on a level surface. Insert the 2 legs with LARGE suction tips into the end of the frame that has the seat handle slots. Depress the brass button at the top of the leg and push the leg into the frame. Ensure that the brass button "clicks" and locks into position. Following the same procedure, insert the 2 legs with small suction tips at the other end of the bench [A].
- 2 Insert the backrest extensions into the backrest tubes. Depress the brass button on the backrest tube and ensure that the brass button "clicks" and locks into position [B].
- 3 Turn the bench on its side and insert the backrest extensions into the 4 tubing slots under the seat. Depress the brass button on the backrest extensions and ensure that the brass buttons "click" and lock into position from the top tubing slots. NOTE: The backrest can slide on either side of the bench depending on the direction the you will be sitting [C].
- 4 Place the bench upright so that it is resting evenly on all four legs and insert the seat handle into the seat handle slots. Depress both brass buttons on each side of the handle and push down until the brass buttons "click" and lock into position [D].
- 5 To adjust the legs and seat height, depress the brass push button on the legs and slide each leg to a desired height. Be sure all 4 legs are level, stable, and all push buttons fully engaged before using. NOTE: The 2 legs with LARGE suction tips should be used inside of the shower.

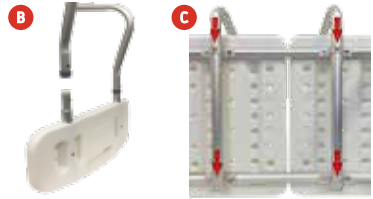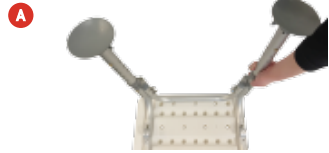

CLEANING

 Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.

- › Use a non-abrasive detergent or household cleaner with warm water.
- › Rinse with warm water and wipe dry.
- › DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

LIFETIME LIMITED WARRANTY

Your Lifestyle Mobility Aids product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

Should you have any questions regarding your Lifestyle Mobility Aids product or warranty please call your local dealer or contact us directly.



Some Additional Products:



Premium Shower
Chair with Back
and Padded Arms



Commando Memory
Forearm Crutch



Super Lite 19"
Aluminium
Companion Chair

Designed In The USA and Distributed By:
LIFESTYLE MOBILITY AIDS