

**LOCK PIN REPLACEMENT**  
**WALKER LOCK ARMS**

1. Remove the lock arm from the walker by removing the small screw from the underside of the elbow (on the horizontal) on the left front corner of the walker. Allowing you to pry back the elbow and slide the lock arm off. If this is too difficult then remove the screw on the vertical underside of the elbow allowing you to completely remove the elbow.
2. Remove old lock pin.
3. Place new lock pin with the metal tip end inside the slot.
4. Place the end of the lock arm against a hard edge with the "C" hanging over so that the lock pin is against the hard edge.
5. While holding in place carefully strike the opposite end with a rubber mallet there by driving the lock pin into place.
6. Place the lock arm back on the walker and secure the elbow with the small screw you removed in step one.