

# BLANKET LIFTER

LVA3013

**vive**  
vivehealth.com




To see all FAQs  
in one place,  
visit [vivehealth.com/h44](https://vivehealth.com/h44)

**OWNER'S MANUAL**

# BLANKET LIFTER

## WHAT'S INCLUDED

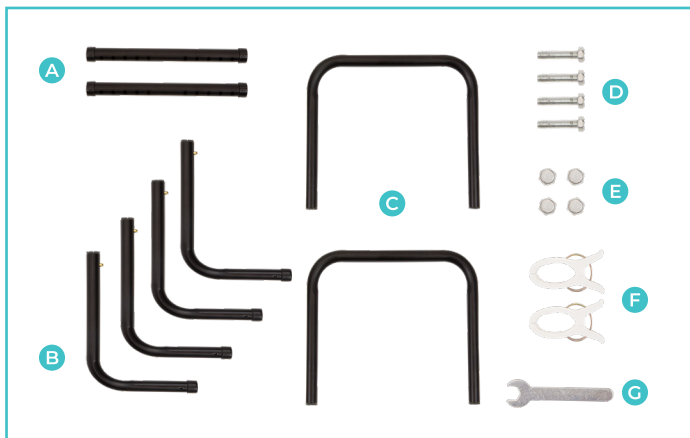

- 2x Base Bars
- 4x L Shaped Bars
- 2x U Shaped Bars
- 4x Hex Bolts
- 4x Nuts
- 2x Clips
- Wrench



## ASSEMBLY INSTRUCTIONS


1. Insert one of the (B) L Shaped Bars in one end of one of the (A) Base Bars. Make sure it clicks into the first hole. You can focus on adjusting the height of the device using these push pins after the set up is complete.



2. Repeat step one for all the remaining sides of the Base Bars. When complete you should have two long curved pieces like the picture below.




3. Insert one of the (C) U Shaped Bars into both ends of the long curved pieces you have created.
4. Repeat step 3 on the opposite end.

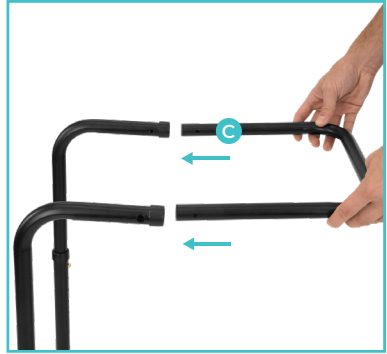

5. Insert (D) Hex Bolt into the four open Hexagon Shaped holes on the device so that the hex head fits snugly into the hex hole on the pipes.



6. Screw the (E) Nuts onto each of the four bolts so that the bolts are secured, again make sure the hexagon shaped heads of the bolts fit snugly into the hexagon shaped holes.

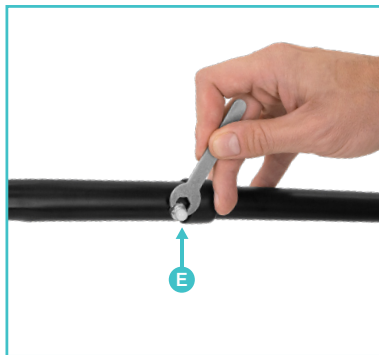

7. Decide whether the device will best fit either on the floor or under your boxspring and then adjust your height accordingly. Use your (F) Clips to hold your blanket neatly in place on your device.

## HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at  
[vhealth.link/h44](https://vivehealth.com/vhealth.link/h44) for helpful answers.



And if that doesn't answer your question, our  
customer service team would love to help!  
Feel free to connect with them by phone,  
e-mail, or chat on our website

[service@vivehealth.com](mailto:service@vivehealth.com)  
1-800-487-3808  
[vivehealth.com](https://vivehealth.com)

Distributed by

**vive**  
**health**

8955 Fontana Del Sol Way  
Naples, FL 34109  
1-800-487-3808  
[www.vivehealth.com](https://www.vivehealth.com)