GUARDIAN[®]

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Combination Padded Transfer Bench With Commode

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General Warnings

To ensure the safe assembly and operation of the product, these instructions must be followed. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT. If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- DO NOT exceed weight capacity of 400 lbs (181 kg).
- DO NOT make any adjustments while using the product.
- DO NOT attempt to use the arms to support full body weight.
- DO NOT stand on the frame of the product.

Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the commode. Prior to use, ensure the following:

1. All push buttons are fully protruded and are properly engaged.
2. There are no loose, broken or missing components and ensure all parts are secure.
3. All screws, nuts and/or bolts are tightened.
4. When using the backrest, ensure the product remains balanced. Excessive twisting or turning while seated on the bench may cause loss of balance and/or damage to the product.
5. All four legs touch the floor simultaneously at all times.
6. The tips on the leg extensions do not contain any rips, wear or are missing. Immediately replace missing or damaged tips.

Assembly

1. Remove contents from box.
2. Lay transfer bench upside down on a flat surface.
3. Insert one suction cup leg extension into the post of the bench seat by depressing the pushbutton and ensuring this snaps into place. Repeat for the second suction cup leg extension (Figure 1). Note: the suction cup leg extensions should be on the same side as the armrest and commode seat.
4. Insert one leg extension with tip into the bench seat by depressing the pushbutton and ensuring this snaps into place. Repeat for the second leg extension with tip (Figure 2).
5. Turn transfer bench right-side-up. Depress armrest pushbuttons and slide into the armrest receptacle tubes with armrest arching outwards (Figure 3).

Figure 1

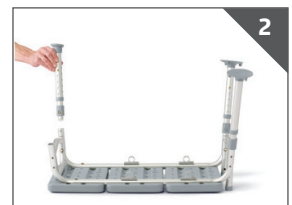
Figure 2

6. To assemble backrest, insert pole into backrest support by depressing pushbutton and sliding into receptacle tube until it locks into place. Repeat for second pole (Figure 4).
7. Insert backrest into base by depressing pushbuttons and sliding supports through both receptacle tubes. **CAUTION:** Backrest must be locked and secured in each receptacle tube (Figure 5).
8. Place legs with suction cups into bath tub.

Figure 3

Figure 4

Figure 5

Commode Feature

To use the commode feature:

1. Remove the lid from the bucket.
2. Position the bucket under the seat and push the bucket in from the front of the bench.
3. After use, pull bucket out from the front side. Empty the contents and clean immediately.

Leg Height Adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When your desired height is reached, lock legs into place by aligning buttons with holes until they snap into place.
2. Make sure legs are locked into place by pushing down or by pulling up on legs.

Cleaning

1. Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
2. DO NOT use abrasive powders, cleaners or scouring pads.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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