GUARDIAN[®]

User's Guide

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Aluminum Bathroom Corner Bench With Faux Wood Accents

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REF G2-501X1

GENERAL WARNINGS

Be sure to read all instructions and safety information before using this bench. If you have questions about using this product safely, please contact Medline at 1-800-MEDLINE | 1-800-268-2848. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.



- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- DO NOT use if components are missing or are damaged. |
- DO NOT exceed weight capacity of 200 lbs (90 kg).
- DO NOT make any adjustments while using the product.
- DO NOT stand on the bench.

CAUTIONS

Users with limited physical strength and/or mobility should be supervised and/or assisted while using this product. Prior to use, ensure the following:


1. All screws, nuts and/or bolts are tightened.
2. There are no loose, broken or missing components and ensure all parts are secure.
3. All four legs touch the floor simultaneously at all times.

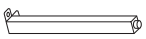PARTS LIST

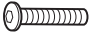
	Item		Qty
A	Seat		1
B	Connector Panel		1
C	Seat Leg C		1
D	Seat Leg D		2
E	Seat Leg E		1
F	Bolt (M6*20)		10
G	Allen Wrench		1

ASSEMBLY INSTRUCTIONS

1. Remove contents from box, making sure you have all parts listed.
2. Carefully stand up the Bench Legs (C,D&E) upside down on a flat surface. Attach the Connector Panel (B) to the Bench Legs(C,D&E) with Bolt (F).

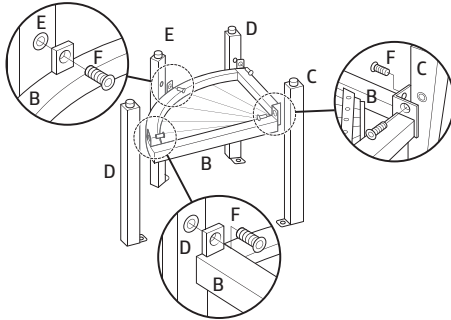3. Attach the assembled legs & connector panel (step 1) to the Seat (A) with Bolt (F).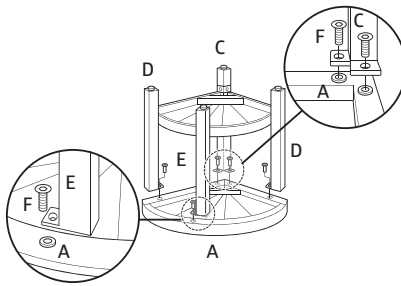

4. Tighten all the Bolts (F). Turn your bench right side up.

ADJUSTING LEG STABILIZERS

Before using the bench, make sure all legs are touching the floor simultaneously and are leveled.

1. To level legs, turn the stabilizer tip clockwise to lengthen, or counterclockwise to shorten, until all legs are simultaneously touching the floor.

MAINTENANCE

Periodic visual inspection of this corner bench is recommended to ensure that all parts and hardware are secure; that components are in good working order and not worn or loose; and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product.

CLEANING

To maintain appearance, dry after each use. DO NOT use abrasive powders, cleansers or scouring pads.

1. Clean with regular household cleaning products. Use a non-abrasive cleanser with warm water.
2. Wipe dry after cleaning.

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Limited Warranty

Medline guarantees for the original purchaser, that the frame of the chair will be free from defects in materials for the life of the chair.

Medline also guarantees the parts on the chair to be free from defect in materials and workmanship for a period of one (1) year from the date of purchase by the original purchaser.

This warranty does not cover device failure due to owner misuse, negligence or normal wear and tear. All warranty claims must include the product in questions unless otherwise authorized by Medline.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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