LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2010 Medline industries, Inc. Guardian is a registered trademark of Medline Industries, Inc.

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REF MDS86018CHRG **REF** MDS86024CHRG

REF MDS86032CHRG

DO NOT install this product without first reading and understanding this instruction manual. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

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WARNING

Failure to properly install grab bar may result in severe injury. Read these warnings carefully before using the product.

- Special fasteners may be required if wood stud is not in desired location. If you have any questions or concerns, we suggest installation by a qualified tradesman.
- Keep screws tight. All screws should be checked for tightness before use.
- Test unit before use. Before using the grab bar, test for stability by vigorously attempting to move the bar.

INSTALLATION INSTRUCTIONS*

Tools Required for Grab Bar Installation:

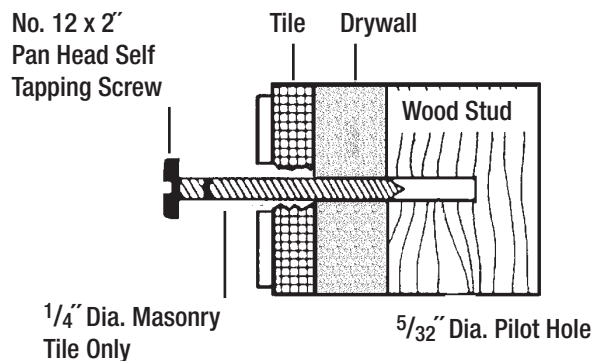
3/8" or 1/4" electric drill, extension cord, 5/32" drill bit for drilling pilot hole in wood stud, 1/4" drill bit for clearance hole with tile if appropriate, sharp center punch, level, soft grease pencil, hammer, screwdriver, stud finder, toggle bolts (if not able to install into wall stud).

Included: Included: 6- 2" screws and 2 rubber gaskets
(Gaskets Contain Dry Natural Rubber).

1. Position grab bar at angle which best accommodates user's needs and also allows at least one end to be mounted into the wall stud. Mark hole locations with soft grease pencil. When locating vertical or horizontal positions, use your level. If one end cannot be mounted into a wall stud, you must use a butterfly toggle fastener (not supplied).

2. MOUNTING DIRECTLY INTO WOOD STUDS:

- Drill 5/32" pilot hole. Place rubber gasket between bar and the wall. Insert the screws through hole in bar and hole in the rubber gasket.

MOUNTING THROUGH TILE: (Caution: Avoid drilling into water lines)

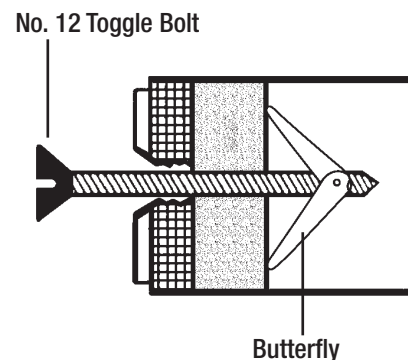
- At the mark for your hole, take your sharp center punch and gently chip the glaze away. The chipped area need be no larger than the hole you are going to drill.
- Once the glaze is chipped, take the 1/4" tile drill bit and drill into the wall. If you strike wood, stop drilling. Switch to the 5/32" drill bit and drill 1" into the wood. First insert the screw through the screw cover base. If you do not penetrate wood, see instructions for mounting with a toggle bolt.

MOUNTING USING A TOGGLE FASTENER (not supplied)

- Drill hole using drill bit large enough to receive the toggle fastener.
- Insert the toggle bolt through the hole for the screw cover base, spin butterfly tip on just enough to be secure, then feed bolt through large hole drilled.
- Once the butterfly is inserted it will fly open on the inside of the wall. Pull out on the grab bar to make sure the butterfly has opened properly (reference illustration). Drive screw down until you feel the bar snug to the wall using screwdriver

3. Test bar for tightness by applying full body force to the bar to make sure that it is solidly anchored.

*Manufacturer bears no responsibility for improper installation.

USER INSTRUCTIONS

Firmly grasp bar. Pull up firmly on the bar to move into a safe, comfortable position in the bathtub or toilet area. Afterward grasp the bar firmly to transfer back into your original position.

NOTE: THIS PRODUCT CONTAINS DRY NATURAL RUBBER.

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