GUARDIAN®

BARIATRIC BATH BENCH

WITH BACK

REF MDS89745AXW

WITHOUT BACK

REF MDS89740AXW

User Instructions and Warranty

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AVEC DOSSIER

REF MDS89745AXW

SANS DOSSIER

REF MDS89740AXW

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BANCO DE BAÑO BARIÁTRICO

CON RESPALDAR

REF MDS89745AXW

SIN RESPALDAR

REF MDS89740AXW

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General Warnings

PLEASE READ CAREFULLY BEFORE USING YOUR BATH BENCH. To ensure the safe installation, adjustment, and operation of Bath Bench, these instructions **MUST** be followed.

DO NOT install this product without first reading and understanding this instruction manual.

If you are unable to understand these warnings, cautions, and instructions, contact a healthcare professional or dealer before attempting to install this equipment.

- Prior to each use check that all joints and fittings on the bath bench are tight.
- **DO NOT** place bath bench on uneven surfaces.
- **DO NOT** use if wobbly, unstable or not level.
- **DO NOT** exceed user weight capacity of 550 lbs.
- **DO NOT** twist or turn while seated on bath bench, this may cause frame to bend.
- **DO NOT** stand on bath bench.
- **DO NOT** use bath bench for any purpose that it is not intended to be used for.
- Check rubber tips for rips, cracks or wear. If these conditions exist, **DO NOT** use the tips and replace **IMMEDIATELY. (Rubber tips can be ordered--item MDS87755T)**

Note: Tips On This Product Contain Dry Natural Rubber.

FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURIES.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2010 Medline Industries, Inc. Guardian is a registered trademark of Medline Industries, Inc.

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