



## Aluminum Folding Rollator

**MHRL8ARE: Red**

**MHRL8ABE: Blue**



Weight capacity 300 lbs


**DO NOT ATTEMPT TO ADJUST OR OPERATE THIS ROLLATOR WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

## **ASSEMBLY INSTRUCTIONS**

Remove all packaging including tape, cardboard and plastic before use. Lay out all the pieces and make sure to inspect them prior to assembly. If any parts are missing or damaged, DO NOT attempt to assemble the rollator. Contact your distributor immediately.

**NOTE: DO NOT use excessive force when installing any parts.**

### **INSTALL HANDLE BARS AND ADJUST HEIGHT**

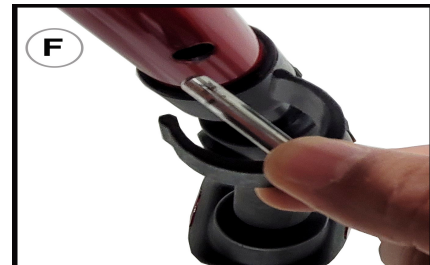

1. Open your rollator. Pull open the folding hinges on either side until straight to secure the unit in the proper position ((Fig. A).

2. Place a brake handle with the pre-assembled loop lock in the corresponding brake handle receptacle and adjust to the desired height (Fig. B).


3. Use the height adjustment knobs to secure the handle bars in this position being cautious to ensure that the bolts are well-tightened in the receptacle (Fig. C).

4. Test and adjust the brake mechanism tension to ensure proper and secure locking before use (Fig. D and E).

5. Insert backrest/push bar into frame receptacles until snap buttons engage and an audible click can be heard on both sides.

6. Insert the frame part with the front wheels into the rollator frame and adjust the height with the **C-Clip (Fig. F) buttons.**

7. Insert the two rear wheels into the openings of the rear frame part while pressing the silver push buttons. Adjust the desired height by inserting the silver buttons into the opening provided for this purpose. Make sure that the brake cables are not pinched. (Fig. G)

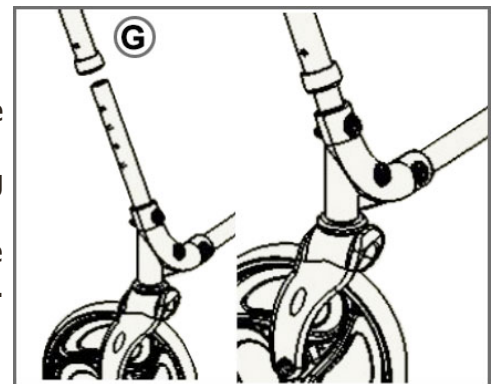

**NOTE:** Make sure knobs are fully tightened and are at equal heights. Always test for stability before applying weight to hand grips.

## **OPERATING INSTRUCTIONS**

### **FOLDING UP YOUR ROLLATOR**

1. The rollator is easily folded from the user's side by grasping the nylon cord under the seat and pulling toward the user. The rollator will begin folding as the user pulls on the cord. Make sure items in the pouch have been removed, if in use.

### **UNFOLDING YOUR ROLLATOR**

1. With all wheels touching the ground, gently pull the frame apart until the folding hinge is completely straightened on both sides {Fig. A).

2. Make sure the rollator is fully opened, the seat is down and the backrest/push bar is locked in before use.

## BREAK OPERATION

1. To lock the brakes, press down on the loop brake handle (Fig. E).
2. To release and/or engage brakes, squeeze the loop brake handle up towards the hand grip (Fig. D).

**NOTE:** Test the loop brakes before operation. When wheels are locked, the wheel lock should prevent the rear wheels from turning. If the wheels turn while locked, immediately discontinue use and return to your provider for adjustment. When the locks are released, wheels will turn freely.

## POUCH INSTALLATION



1. With rollator in unfolded position, lift seat up to rest against the backrest.
2. Unsnap all buttons on nylon pouch.
3. Hang the pouch between the front and back frame tubes, looping each of the snap tabs over the bars, then secure with snap buttons. Make sure ALL snap buttons are attached to the frame.

## WALKING WITH THE ROLLATOR

1. Standing behind the rollator, slowly roll forward so that the rear wheels are a few inches in front of your feet.
2. Step forward and place your foot in line with the rear wheels, but slightly inside of the wheels.
3. Maintaining a steady roll forward, place your other foot in line with the rear wheels.
4. Determine a safe gait to keep your feet in line with the wheels. Do not go on steep slopes, uneven surfaces or ramps that might cause you to lose control of the rollator.

## MAINTENANCE

- Wipe rollator frame with a clean soft cloth at least once a week. All metal parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately.
- Wipe the hand grips with cold water and a clean towel. **NEVER** use ammonia-based cleaning solutions or soap on the hand grips. This may cause them to loosen or slide off the frame.

- **NEVER** use organic solvents, abrasive cleaners or scouring pads on any part of the rollator.
- Check all the parts and movable connections before each use to ensure that the rollator is safe to use. If deemed unsafe, immediately discontinue use and replace any or all parts in question.
- Check the wheels/casters for rips or excessive wear. Immediately discontinue use and replace any or all parts if any of these imperfections exist.

## **WARNING**

### **DO NOT ATTEMPT TO ADJUST OR OPERATE THIS ROLLATOR WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

- If there are any problems with your rollator, **DO NOT** attempt to repair it yourself. Contact the provider or store that you purchased the rollator from for any needed parts or repair.
- Make sure your rollator is adjusted to your correct height.
- **DO NOT** stand on rollator.
- Inspect all parts for shipping damage before assembly. If there is shipping damage - **DO NOT USE**. Contact dealer/ carrier for further instructions.
- This product should not be used without instruction from a healthcare professional.
- Always lock loop brakes before sitting down or standing up from seat. Make sure the front casters are in the forward position before transferring into or out of the seat. **DO NOT** use seat when on an incline or uneven ground.
- **DO NOT** use your rollator on a rough or uneven surface. Doing so may cause it to tip over, resulting in injury. Never use on stairs or escalators while using the rollator. **DO NOT** use on inclines greater than 8% grade.
- **DO NOT** use this rollator in pools, showers or other water environments, as this could result in corrosion of parts that result in damage to the rollator and possible injury to the user.
- **DO NOT** exceed the maximum weight capacity intended for this product.
- Always avoid unsafe conditions, such as soft, slippery or moving surfaces, uneven paths and obstacles.
- If you are transferring to or from a wheelchair to use this product, be sure the wheelchair wheels are securely locked.
- This rollator is not intended for use on stairs.
- Check to make sure the height adjustment knobs are securely tightened on a daily basis.
- **DO NOT** push one side of the rollator ahead and then other side without lifting the rollator first.
- Check the wheels for rips, wear or tears regularly. Immediately replace any or all if any of these imperfections exist.

- **DO NOT** use the hand grips to fully support the user's body weight.
- **DO NOT** store in direct sunlight or outdoors. Exposure to extreme temperatures (heat, cold, moisture, etc.) may make surfaces hot to the touch, weak or brittle.
- This is not a toy. **DO NOT** let children play with or around the rollator.
- Users with limited physical capabilities should be supervised or assisted when using the rollator.
- Assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.

**FAILURE TO USE COMMON SENSE AND HEED THE ABOVE WARNINGS FURTHER INCREASES RISK OF SERIOUS INJURY. USE AT YOUR OWN RISK AND WITH APPROPRIATE AND SERIOUS ATTENTION TO SAFE OPERATION. USE CAUTION.**

# LIFETIME LIMITED WARRANTY

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.