

## Lifetime Limited Warranty

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.



## 24" Heavy Duty Wheelchair



**Weight Capacity: 500 lbs**



## Setup and Usage

**UNFOLDING** - Locate the two rails on the seat rest. This is where the seat fabric is attached to. Push down on the seat rails and the chair will widen.

**FOLD** - Depress locking tabs on the back of the backrest "canes" to allow backrest to fold. Turn the right and left foot plates upward to the vertical position. Pull up on the middle of section of the cushion to fold.

**WHEEL LOCKS** - Adjust the locks by loosening or tightening the nut on the carriage bolt. The lock shoe should be embedded at least 1/8" by sliding the clamp toward the rear wheel while the handle is in the locked position. Secure the nut and bolt tightly. Test the locking mechanism to ensure proper function before using the chair.

**WHEELS** - If a wheel wobbles or feels loose, release the lock nut and slightly tighten the axle bolt. To check for correct tightness, tilt the chair to one side and spin the adjusted wheel.

**TIRES** - Replace the tires if they show significant wear or visible cracks.

**FRONT CASTERS** - Regularly inspect the casters and apply lubrication as needed. To lubricate, remove the fork and grease the caster stem bearings. Additionally, lubricate the wheel bearings by detaching the wheel from the fork. When reassembling, tighten the nuts just enough so that there is no looseness, but the wheels still turn smoothly. If the nuts are too loose, the caster may wobble; if overtightened, steering the chair will become difficult.

## Safety First

- Parts should be inspected periodically.
- Ensure the armrest is securely locked BEFORE using the wheelchair.
- Do not stand on the frame of the wheelchair.
- Always use the hand rims for self propulsion.
- Do not grip the rear wheels themselves.
- Keep the footrests at least 2-1/2 inches above the floor; do not adjust them lower.
- The wheel locks are used while the occupant is sitting and stationary, or getting into or out of the wheelchair. The wheel locks are not meant to arrest the chair while moving. Doing so could cause injury and/or wheelchair damage.
- The wheelchair is suitable for indoors and out, but avoid prolonged bumpy terrain. Use proper caregiver techniques when negotiating obstacles. Consult with your MOBB Healthcare dealer for proper usage.

## Setup and Usage

**DETACHING THE SWING-AWAY FOOTRESTS/ LEG RESTS** - Unlock by pressing the front rigging lever. Swing the footrest/leg rest out to the side and lift it off to remove.

**FOOTREST/LEG REST LENGTH ADJUSTMENT** - While seated in the wheelchair with feet resting on the footplates, adjust the length of the footrests or leg rests by loosening the clamp bolt on the support assembly. Set the footplate to a height that provides the best support for your thighs. Make sure the front rigging does not press uncomfortably against your thighs and that the footrest is no lower than 2-1/2" from the ground to ensure sufficient clearance.

**LEG REST ADJUSTMENT/ELEVATION** - Move the footplate on the leg rest assembly to the chosen position. The leg rest will lock into place automatically once you release it. To lower the leg rest, push the lever outward and adjust the footplate down to the desired level.

**REMOVABLE ARMS** - To detach the arms, press the spring button and pull the arms out of their front and rear sockets.