

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/y4n](https://vivehealth.link/y4n) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com



X002NGZAS7

Carbon Fiber Standing Cane
MOB1038BLK



(01)00810041983398(241)MOB1038

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China



[vivehealthusa](https://vivehealthusa.com)

vive®

CARBON FIBER STANDING CANE

Owner's Manual
MOB1038BLK

vivehealth.com



To see all of the FAQs in one place
visit vhealth.link/y4n

FIND YOUR CORRECT CANE HEIGHT


1. Let your arm hang naturally by your waist. Adjust the cane height so the handle touches your wrist.



CANE HEIGHT ADJUSTMENT

1. Loosen the locking latch.

2. Slide extension tube up or down to desired height. Button snap should secure cane into place.
4. Tighten the locking latch.

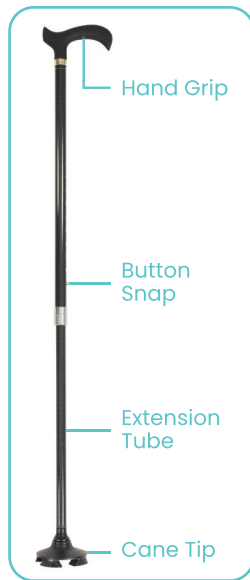

WARNINGS

- Make sure the brass pin is locked into place before use.
- Weight Capacity: 250 LBS