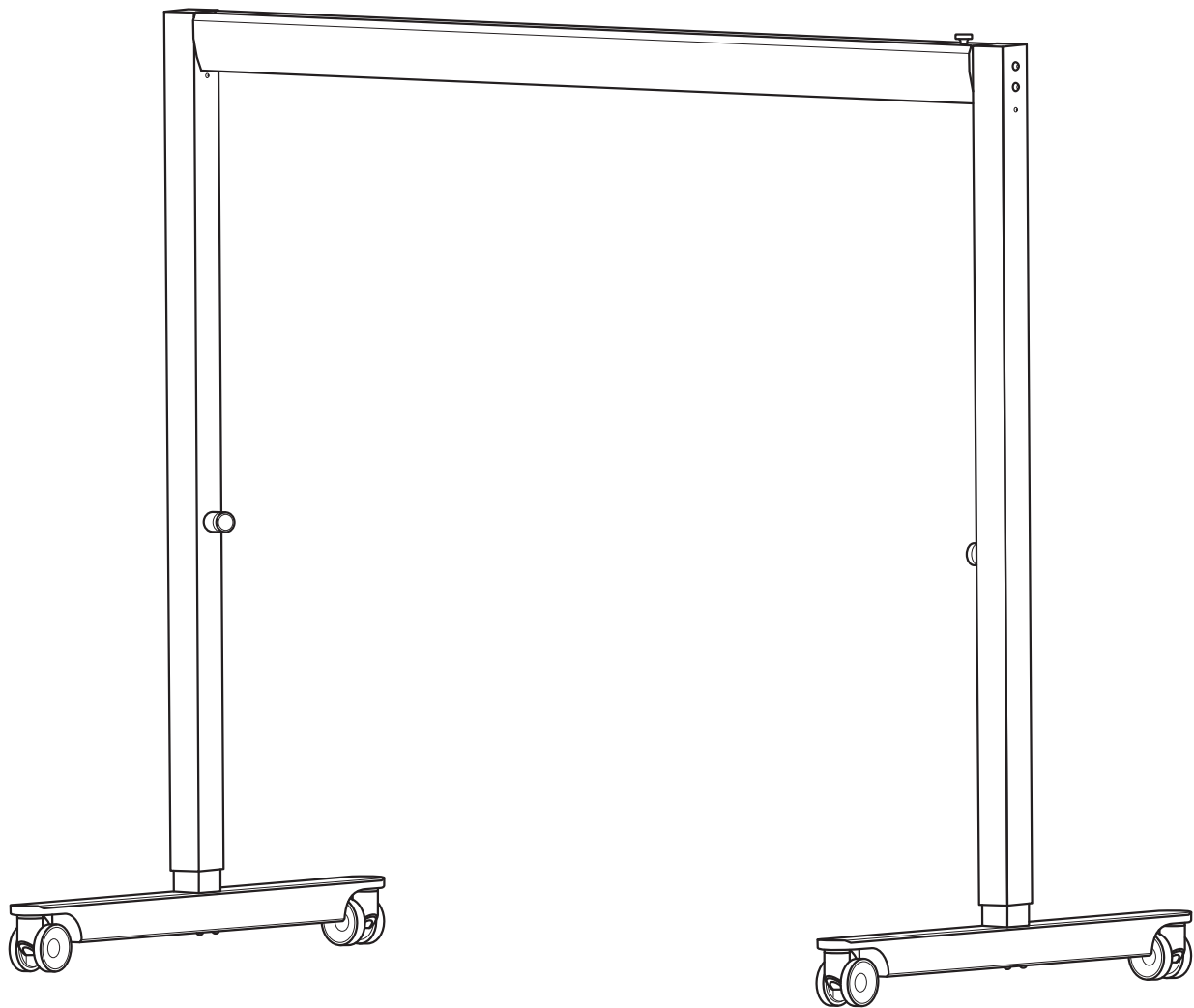
MRS Duo Flex

molift[®]
by Etac

EN - User manual
DE - Gebrauchsanweisung
NO - Brukermanual
SV - Bruksanvisning
FI - Käyttöohje

IT - Manual utente
FR - Manual d'utilisation
ES - Manual de usuario
NL - Handleiding
DK - Brugsvejledning

BM21099 Rev E 2021-06-16

MRS Duo Flex

- 1 EN - English Manual
- 11 DE - Deutsche Bedienungsanleitung
- 21 N - Norsk manual
- 31 SV - Svensk manual
- 41 FI - Suomenkielinen käyttöohje
- 51 IT - Manuale in italiano
- 61 FR - Manuel français
- 71 ES - Manual en español
- 81 NL - Nederlandse handleiding
- 91 DA - Dansk vejledning

English Manual

Content

MRS Duo Flex2
 MRS components2
 About MRS Duo Flex2
 General.....3
 Declaration of conformity.....3
 Conditions for Use3
 Warranty3
 Product identification.....3
 Technical Data MRS Duo Flex 3000.....4
 Technical Data MRS Duo Flex 4000.....5
 Assembly6
 Supplied parts6
 Mounting telescopic boom6
 Mounting base6
 Mounting charger to column6
 How to use Molift MRS Duo Flex7
 Adjusting telescope boom.....7
 Adjusting heights of post7
 Castors.....7
 Lifting user7
 Transport and storage7
 Accessories8
 Battery charger8
 Molift AIR.....8
 Molift Nomad.....8
 Telescope Boom8
 Maintenance9
 Cleaning and disinfection9
 Reconditioning.....9
 Recycling9
 Spare Parts9
 Periodic Inspection9
 Periodic Inspection Scope.....9
 Troubleshooting 10

Important

This User Manual contains important safety instructions and information regarding the use of MRS Duo Flex and accessories.

In this manual the user is the person being lifted. The assistant is the person operating the lifter.

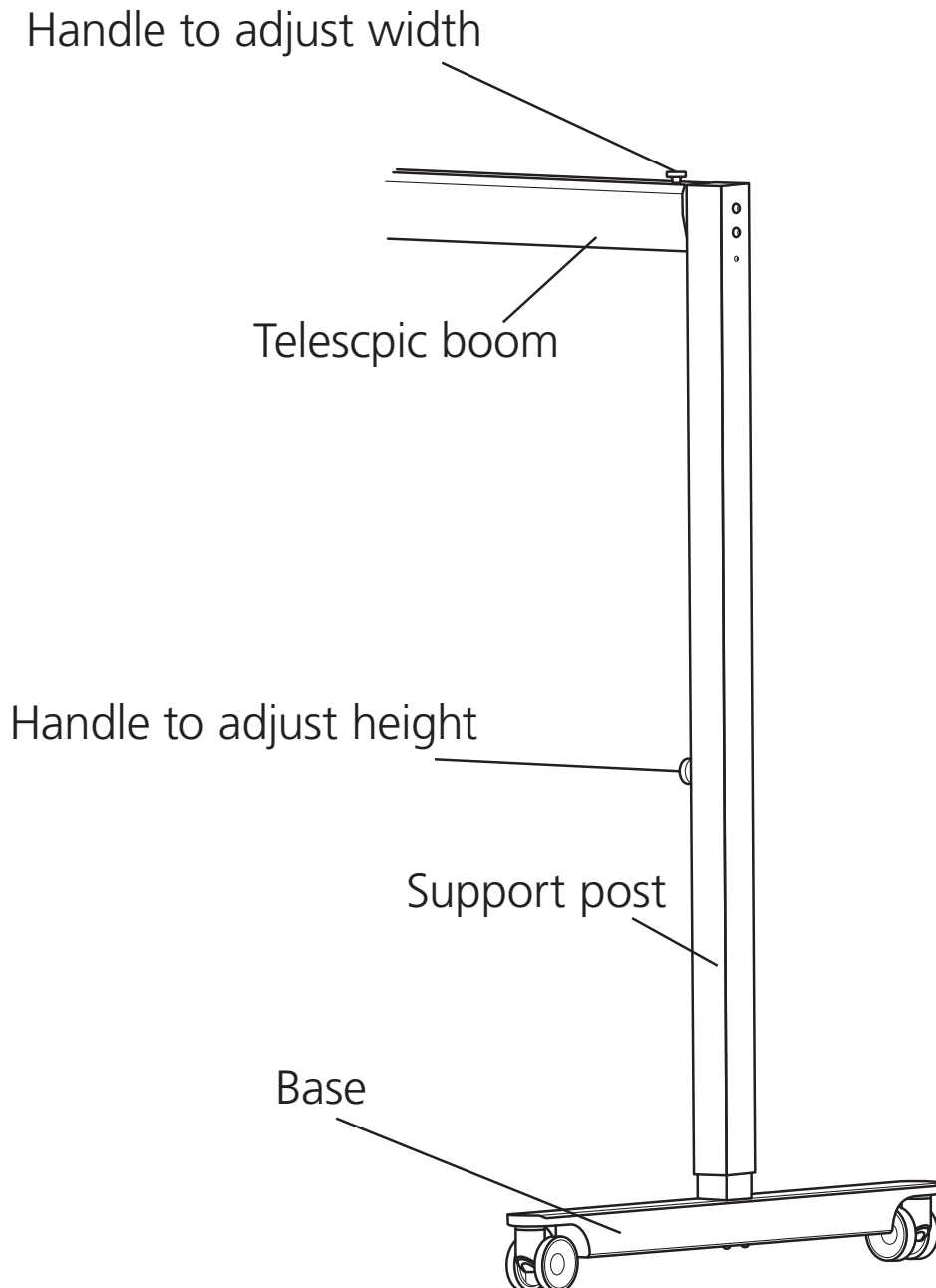
Warning!
 This symbol indicates important information related to safety. Follow these instructions carefully.

Read User Manual before use!
 It is important to fully understand the content of the user manual before attempting to use the equipment.

MRS Duo Flex

MRS Duo Flex components:

About MRS Duo Flex

MRS Duo Flex is a mobile 2 post gantry intended for Molift Air and Nomad lifters. The purpose is for lifting and transferring a person from/to bed, floor, chair, wheelchair or toilet with a sling. MRS Duo Flex can be disassembled and transferred to alternative locations, and has adjustable posts with min/max heights from 1,89 to 2,54 meters.

Available telescope boom lengths are 2 - 3m and 3 - 4m. The load capacity is 275 kg (606 lbs). It is ideal for hospitals and institutional care settings, but can also be used in private's homes. The gantry should only be used indoor on flat surfaces. Such as Bedroom, bathroom living room etc.

General

CE declaration

The product, and its related accessories, described in this instruction for use (IFU), is in compliance with the regulation (EU) 2017/745 of 5. April 2017 – as a medical device, risk class I.
Any serious incident that occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Conditions for Use

Lift and transfer of a person will always pose a certain risk and only trained personnel are allowed to use the equipment and accessories covered by this user manual.

Modifications and use of components made by other manufacturers.

We recommend only using Molift components and spare parts. Declaration of conformity is not valid and Etac is not responsible for warranty if any modifications are made to the product. Etac shall not be liable for faults or accidents that can occur when using components made by other manufacturers.

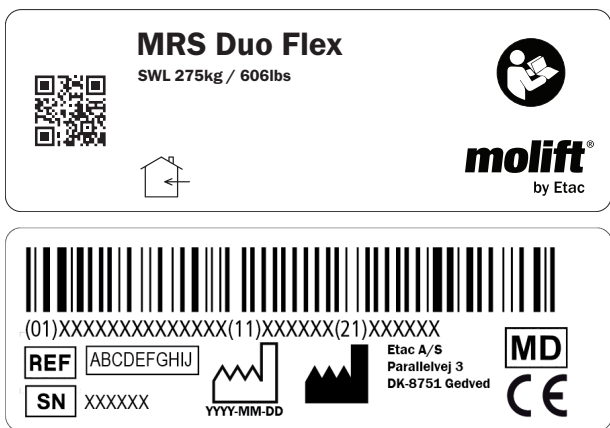
Only certified personnel are allowed to perform repairs.

Warranty notice
Two-year warranty against defects in workmanship and materials of our products.

Product identification

Product label (example)

The Product label contains article number, max width, mounted trolley and safe working load.

Warning labels and Symbols

Symbols used on the product, explained in more detail:

- 
Refer to user manual
- 
Indoor Use only
- 
Manufacturer
- 
Date of manufacture
- 
Catalogue number
- 
Serial number
- 
Medical Device

Technical Data MRS Duo Flex 3000

Safe Working Load (SWL)

275 kg (606 lbs)

Weight of unit

Support Post: 18 kg/39,7 lbs

Telescopic boom: 28 kg/61,7 lbs

Base: 12,5 kg/26,5 lbs

Total weight (2x post, 2x base, x boom):
89 kg/196,2 lbs

Telescope boom length:

2000 - 3000mm

Dimensions:

Height (a): Min 1773,5 mm

Width (b): 3160 mm

Width (c): 2160 mm

Base height (d): 120 mm

Base depth (e): 1150 mm

Height (f): 1890mm

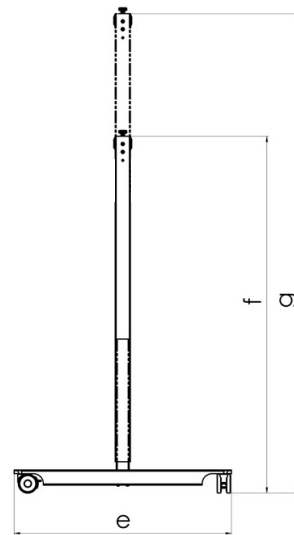
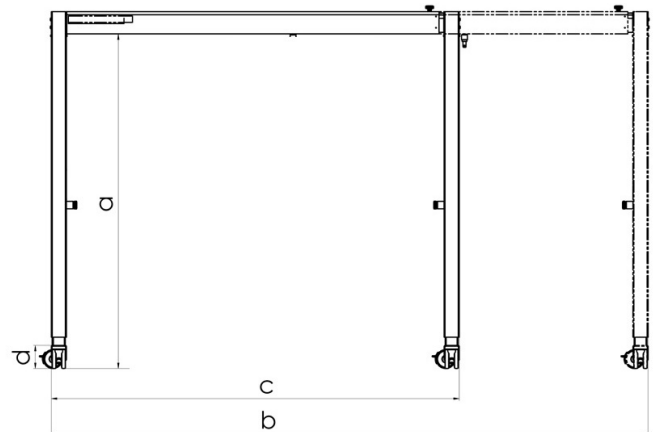
Height (g): 2540 mm

Material:

Painted steel and aluminum, Plastic

Expected Lifetime:

MRS Duo Flex has an expected lifetime of
10 years


Technical Data MRS Duo Flex 4000

Safe Working Load (SWL)
275 kg (606 lbs)

Weight of unit

- Support Post: 18 kg/39,7 lbs
- Telescopic boom: 38 kg/83,8 lbs
- Base: 12,5 kg/26,5 lbs
- Total weight (2x post, 2x base, 2x boom):
99 kg/218,2 lbs

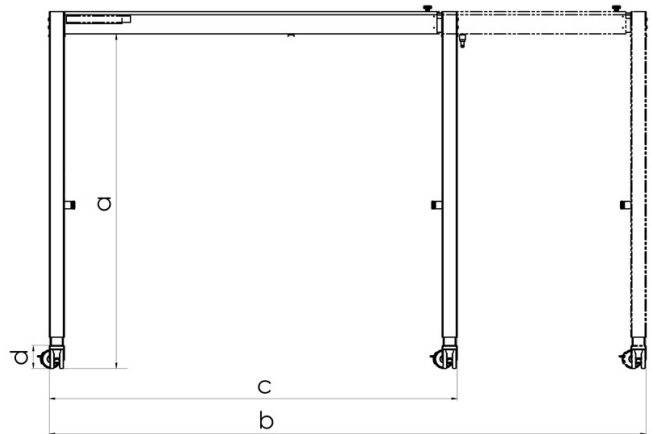Telescoop boom length:
3000 - 4000mm

Dimensions:

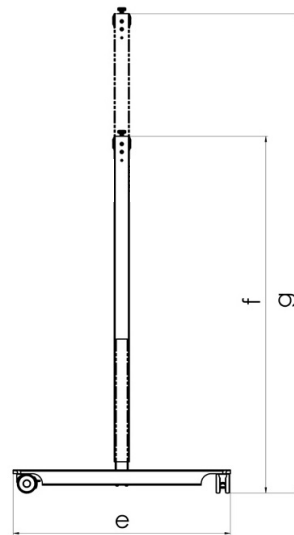
- Height (a): Min 1773,5 mm
- Width (b): 4160 mm
- Width (c): 3160 mm
- Base height (d): 120 mm
- Base depth (e): 1150 mm
- Height (f): 1890mm
- Height (g): 2540 mm

Material:

Painted steel, Aluminum, Plastic


Expected Lifetime:

The gantry has an expected lifetime of 10 years

Assembly

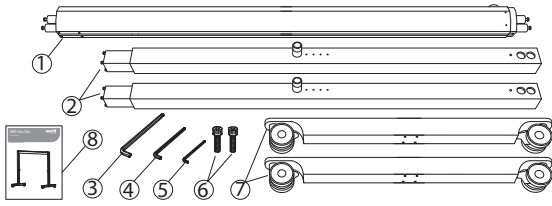Supplied parts


1	Telescopic boom	1x	5	Allen key 4mm	1x
2	Post	2x	6	DIN 912 M5x12	2x
3	Allen key 6mm	1x	7	Base	2x
4	Allen key 5mm	1x	8	User manual	1x

Make sure the following parts are included in the package.

Beware of sharp edges at the ends of the boom.

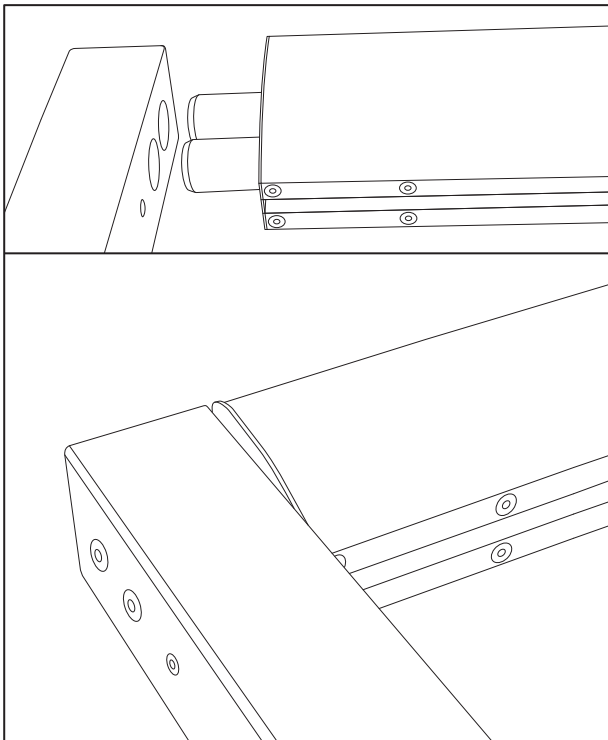Mounting telescopic boom


1. Place the column on the floor with the 2 holes facing sideward.
2. Place the boom between the columns.
3. Connect the columns with the boom and tighten the 2 screws (M10).
4. Repeat the procedure at the opposite side.

Ensure that the joint is stable, otherwise the 2 post system will be unstable and not safe to use

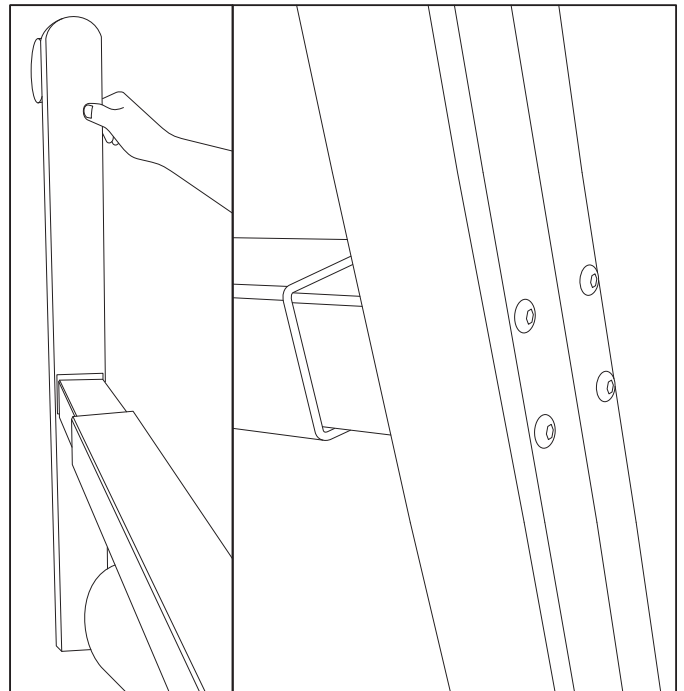Mounting base


1. Lift the column slightly and connect the base.
2. Tighten the 4 screws (M8) under the base.
3. Repeat the procedure at the opposite side.
4. Raise the MRS Duo Flex to an upright position. It can be an advantage to lock the castors.

Ensure that the joint is stable, otherwise the 2 post system will be unstable and not safe to use

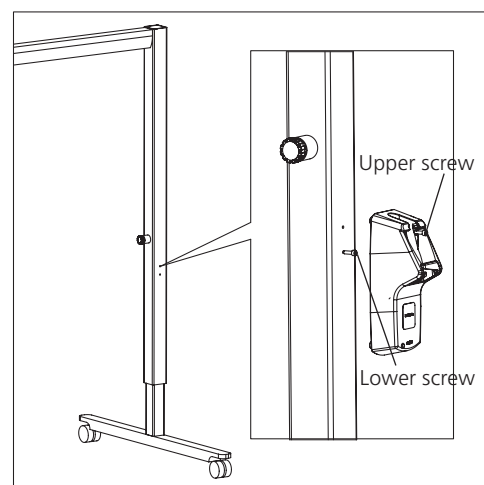
Mounting charger to column

1. Fasten lower screw first (M5).
2. Place charger on screw and insert upper screw (M5).
3. Use allen key and tighten bolt.

How to use MRS Duo Flex

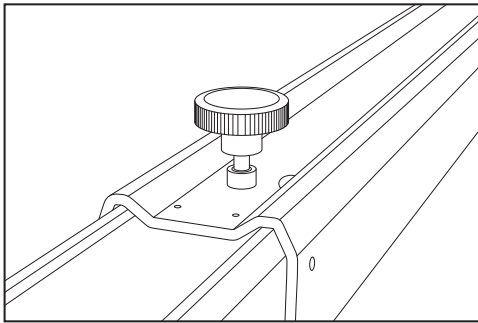Adjusting telescope boom

Easy to adjust width without tools. Loosen the locking handle, slide boom to wanted position, then tighten handle.

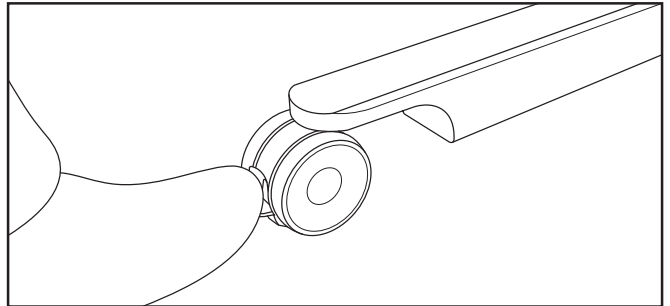
Remember to tighten the locking handle when the boom is adjusted


Don't stand under boom when adjusting post height

Castors

Two sets of castors with brakes. Simply step on the pedal to activate the brake.

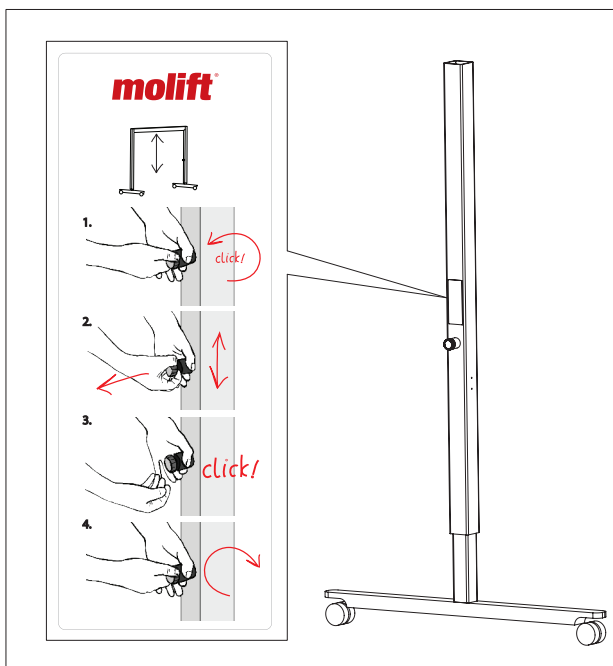
Adjusting heights of post

Instant height adjustment (gas spring mechanism). A simple user guide is placed above the handle.

1. To adjust the height, loosen the height adjustment handle. Turn the handle counterclockwise until a faint click is heard.
2. Pull out the handle and adjust the height to desired position.
3. When the handle is released it is important that there will be a clear click to ensure that the height is adjusted and thus locked.
4. Tighten the adjusting handle by turning it clockwise.

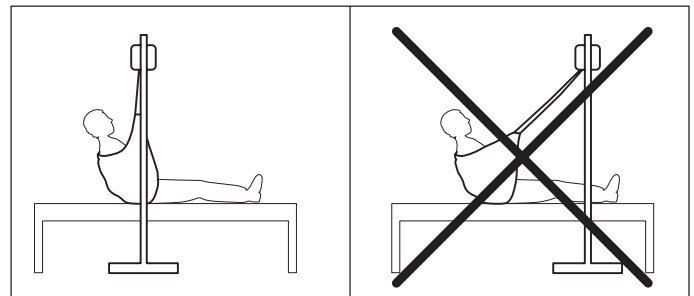Lifting user

1. Make sure the floor is level.
2. Place the gantry straight above the user.
3. Make sure that all four brakes are activated.
4. Read the user manual for Molift Air or Nomad before lifting user.

Never leave a user unattended in a lifting situation.

Never lift the user higher than necessary to carry out a lift.

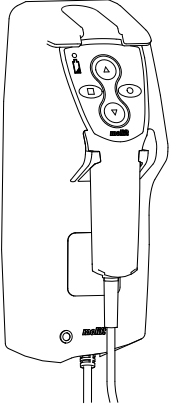
Transport and storage

The lifter can be stored and transported under temperatures between -25 - 70 °C and Relative Humidity between 15 - 93 %.

Accessories

Battery charger


Molift Air or Molift Nomad ceiling hoist is charged through the hand control. The hand control charger can be mounted on the MRS Duo Flex.

Charger for handcontrol Molift Nomad/AIR.
Art. no.: 1240100


Molift Air

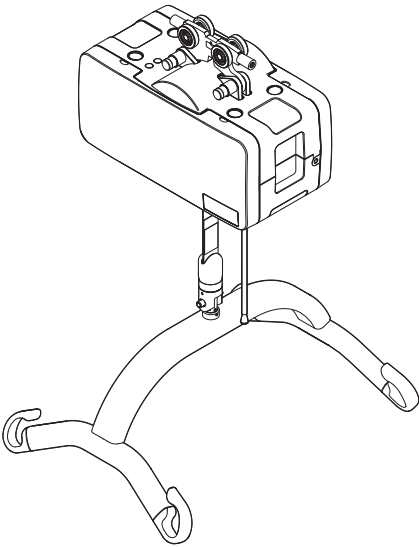Molift Air 205 is suitable for use with Molift MRS Duo Flex.

Molift Nomad

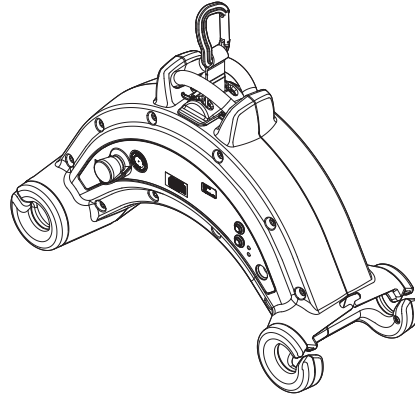Molift Nomad portable hoist is suitable for use with MRS Duo Flex.

Molift Nomad is available with 160, 205, 230 or 255 kg SWL.

- Molift Nomad 160 kg SWL
- Molift Nomad 205 kg SWL
- Molift Nomad 230 kg SWL
- Molift Nomad 255 kg SWL

Telescope boom

MRS Duo Flex can be delivered with 2 different telescope boom lengths from 2-3 and 3-4 meters.

Maintenance

Cleaning and disinfection

Clean on a regular basis. Clean surfaces with a damp cloth using an appropriate pH-neutral detergent. Do not use solvents or strong liquids, this may damage surfaces on the lifter. For disinfection when needed; use isopropyl alcohol. Avoid abrasive cleaning products. Remove hair and pile from the castors and verify that the castors rotates freely. Clean contact and hand control with isopropyl alcohol to remove grease and dirt.

Make sure not to damage or remove labels when cleaning.

Reconditioning

Follow cleaning and assembly procedure, complete periodic inspection and use checklist after assembly to recondition MRS Duo Flex.

Recycling

Molift MRS Duo Flex can be disposed of as metal waste except the battery charger. Battery charger must be removed and disposed of as Electrical equipment.

Spare Parts

A list of spare parts is available on demand

Periodic Inspection

Periodic Inspection Scope

Periodic inspection is a visual examination (particularly MRS Duo Flex structure, bolts and lifters with attachments, safety devices and person-support devices) according to Periodic Inspection Report for MRS Duo Flex.

In the event of danger to safety, the gantry shall immediately be taken out of service and marked clearly with "out of order" and shall not be used until MRs Duo Flex is repaired

Periodic Inspection shall be performed at least once a year or more frequently if required by local requirements. The inspection should be performed by service personnel authorized by Etac. Contact Etac for training and authorization or recommendation of an approved service partner.

When performing a periodic inspection, the inspector shall fill out the inspection report for MRS Duo Flex. The reports should be retained by the person(s) responsible for servicing the lifter.

When periodic inspection is completed the inspector shall mark the gantry with a sticker on the control label showing the date when periodic inspection is performed and a label (with 6-digit ID from Etac or distributor) showing who has performed the periodic control. The control label can be found on the post, and this will then indicate when next service should be performed.

CONTROL

12 Month

Mark label with month and year of inspection

Troubleshooting

Symptom	Possible Cause/Action
Wobbly between base and support column.	Check bolts in base. Make sure bolts is properly fastened.
Wobbly between boom and post.	Check bolts between bracket and boom. Make sure bolts are properly fastened.
The height adjustment does not work	The gas spring is defective. Replace with a new gas spring.
Molift Air or Molift Nomad hits post during use.	Adjust end stopper to correct position.

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Etac A/S