



INFECTION CONTROL DESIGN

Mor-Medical International
People Matter™

Tel: 956-627-4406 Fax: 956-928-9574 • 2500 N McColl Rd, STE 21 • McAllen, Texas 78501
www.mormedicalinternational.com • info@mormedicalintl.com

Mor-Medical International wants to thank you for buying the New Era PVC Shower Commode Chair.

Seamless Infection Control Patent Pending Design-No Fittings Exclusively by Mor-Medical designed for efficient infection control. Structural frame held in place with high tensile strength stainless steel bolts.

Patent Pending US62/782,082



DNE – 118-3TWL 310 LBS. CAP.



DNE – 122-3TWL 385 LBS. CAP



DNE – 126-4TWL 435 LBS. CAP

Please remove all the components from the package / box and lay them on the floor / table etc. The list below will detail all components / parts.

1. 2 each side braces, Left and Right (Left is tag PINK), (Right is tag BLUE) when facing you.
2. 3 each 20.5" Cross Braces, 2 will have pilot holes drilled to insert the wire hangers for commode pail (They are the top cross braces facing each other when assembled and CABLE TIED in the package, the YELLOW tag one is front brace, the RED tag one is back brace), the third goes in the front LOWER SECTION and will have a GREEN tag. NOTE: All cross braces have a retainer screw underneath, the retainer screw should face down.
3. 2 each wire hangers, must be inserted into the cross braces with pilot holes. Insert wire hangers to the front side cross brace first then proceed to insert wire hanger to the back cross brace. The front cross brace must be secured to the side panels.
4. 7 quart Commode Pail with Lid and handle
5. Snap On Soft Seat
6. PVC back with Vinyl straps (remove temporary cross brace after installation of top retaining screws. Proceed to install bottom retaining screws after temporary cross brace is removed)
7. 4 each 3" Twin Locking Casters

All Components / Parts



Assembly Steps

Left side brace (Pink tag) with bottom cross brace (Green tag) secured (Step 1)



Both left and right side braces (Blue tag) secured with bottom cross brace (Step 1)



Both Left and Right side braces secured with two cross braces (Go ahead and insert the wire hanger to the front top brace Yellow tag) (Step 2)



Both Left and Right side braces secured with all 3 cross braces (Prior to completely securing the back top brace (Red tag) inset the backside of the wire hanger) Then proceed to secure the cross brace and side braces (Step 3)




Shown with the base frame complete and with 7 quart commode pail (Step 4)



Shown with the base frame complete, 7 quart commode pail, and clip on soft seat (Step 5)



Shown with base frame complete, 7-quart commode pail, clip on soft seat and PVC back support (Step 6)

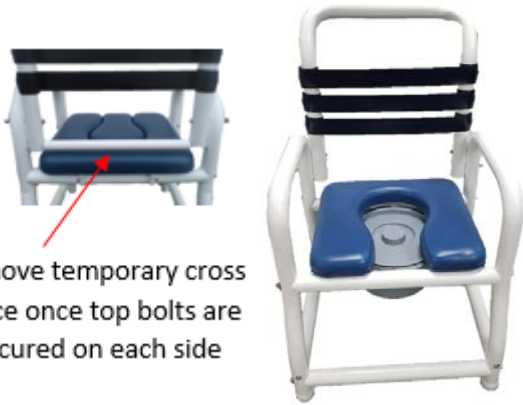

Remove temporary cross brace once top bolts are secured on each side

Shown with base frame complete, 7-quart commode pail, clip on seat, PVC back support and casters. Congratulations, the New Era Chair is complete and ready to use. (Step 7)



Tools needed to fully assemble the New Era PVC Shower Commode Chair.



Channel Lock Pliers



Crescent Adjustable Wrench



Phillips Screwdriver

Assembly Components



(6 each) 3/8 x 2-1/2 Lag Screw Stainless

For Side Panels – will secure the width left to right with the cross braces 3 per side



(6 each) 3/8 Flat Washer 18-8 Stainless

For Side Panels – will secure the width left to right with the cross braces 3 per side




(4 each) 1/4-20 x 4 Hex C/S Stainless

For PVC Back Rest – 2 per side



(4 each) 1/4-20 Acorn Cap Nut Stainless

PVC Back Rest – 2 per side

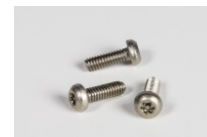

(4 each) 8 x 1/2 Phil Pan T/S SS 18-8

To secure caster wheel and PVC Cap to PVC Frame (front and back)

Component Placement

(4 each) 1/4-20 Acorn Cap Nut Stainless
PVC Back Rest – 2 per side

(4 each) 1/4-20 x 4 Hex C/S Stainless
For PVC Back Rest – 2 per side

(6 each) 3/8 x 2-1/2 Lag Screw Stainless
For Side Panels – will secure the width left to right with the cross braces 3 per side

(6 each) 3/8 Flat Washer 18-8 Stainless
For Side Panels – will secure the width left to right with the cross braces 3 per side

(4 each) 8 x 1/2 Phil Pan T/S SS 18-8
To secure caster wheel and PVC Cap to PVC Frame (front and back)