

# Omni-Jugs®

for collection & aspiration of  
large volumes of medical / surgical fluids

## How to Assemble

### Step 1:

Align lid with handles and place on the jug.

### Step 2:

Press firmly to snap all eight lock-tabs around entire perimeter of lid.

### Step 3:

Connect vacuum port on the jug to suction source on wall or ceiling.

### Step 4:

Connect suction tubing from field to appropriate ports on the jug's port cap.

### Step 5:

Place jug in standard ring stand or floor dolly.

### Step 6:

Seal unused ports.

