

OPERATION INSTRUCTIONS

SHOWER CHAIR COMMODE MODELS:

900# CAPACITY - BSC880 P

700# CAPACITY - BSC660 P, BCC661 P (no wheels), VO OF20 P 700FR

550# CAPACITY – SC550 P

500 # CAPACITY – VL OF20 P 500

450# CAPACITY - SCC250 OS FS, SCC9250 OS, KD SCC9250 OS, SCC750 OS N

400 # CAPACITY – VL OF9250 OS

375# CAPACITY - SCC9250 MS, SCC750 MS N

350# CAPACITY – VL OF9250 MS

325# CAPACITY – ELT817 P/820 P Series, PLT817 P/820 P Series

300# CAPACITY - SCC767 G/N/L, SCC777 G/N/L, SCC787 G/N/L, SCC700 G/N/L,

SCC701 G/N/L, SCC9257 G/N/L, SCC9150, SCC9250, SC9111 P, SC9211 P

ESC17 P, ESC20 P, ESC37 P, ESC40 P, SCC255 BP G/N/L, CC201, CC201 WL,

VL SC16 P, VLSC17 P, VL SC19 P, VL SC20 P, VL SC16 P FR, VL SC16 P FRSB,

VL SC16 P FRLB, VL SC16 P FRLDA, VL SC17 P FR, VL SC17 P FRSB,

VL SC17 P FRLB, VL SC17 P FRLDA, VL OF17 P, VL OF20 P, VL OF17 P FR,

VL OF20 P FR, VL OF17 P FRSB, VL OF20 P FRSB, VL OF17 P FRLB,

VL OF20 P FRLB, VL OF17 P FRLSA, VL OF20 P FRLSA,

ESC OF817 P/OF 820 P Series

PARTS LIST – ON REVERSE SIDE

ASSEMBLY INSTRUCTIONS – Attached when assembly is required

WARRANTY – ATTACHED

OPTIONS – ATTACHED

PURPOSE – To provide a comfortable and mobile product for showering and/or toileting.

USAGE RECOMMENDATION – Recommended for showering people who are unable to stand but able to sit and for toileting needs.

OPERATING INFORMATION:

SHOWERING - All chairs are equipped with at least 2 locking casters. PUSH THE CHAIR BACKWARD SO THE WHEELS ARE FACING OUT AND LOCK ALL LOCKING CASTERS BEFORE ANY TRANSFER. Transfer user into the chair. Move user into shower area and lock casters. When shower is finished, dry user and chair as completely as possible. Transport user to desired area. USE CAUTION: CHAIR MAY BE SLIPPERY. See option page for special needs users.

TOILETING – Lock all locking casters. Transfer user onto chair. Refer to user's program as to timeframe for chair usage. When toileting program is finished, transfer user off chair. Remove pail out the side, empty and clean. Note: chair without pail may be used over toilet.

PRECAUTIONS:

Never leave an at-risk user unattended in the chair.

Exaggerated user movement in any direction or sitting on the edge of the seat may cause the chair to tip.

If a seat is removed for any reason, make certain it is snapped back into place.

SAFETY/MAINTENANCE INFORMATION:

Make certain chair is assembled according to enclosed instructions.

Check pipe and fittings for hairline fractures monthly.

Check all junctures monthly to make certain the pipe and fittings do not pull apart.

Clean and lubricate casters monthly to avoid rust and wheel lock-up.

Do not subject product to temperatures below 40 or above 140.

CLEANING INSTRUCTIONS:

Spray and wipe seat, fabric, pail, pipe and fittings with in-house disinfectant after each use. Wipe down casters after each use. Avoid using solvent based or abrasive cleaners.