

THE TOILET SEAT RISER



Maintaining Your Riser:

**Check the mounts regularly to make sure they are secure
Use mild detergent and warm water to clean Test cleaning agents on small section on inside before use**

Assembly and Installation Instructions



Warranty

Warranty: Alex DME products offer a Limited Lifetime Warranty. This warranty does not extend to non-durable components; i.e. rubber accessories and grips, which are subject to normal wear and tear. In the event of a defect covered by this warranty, we will repair or replace the device or part, at our option. Refer to your product's warranty statement on our website for specific terms and limitations.



Included Items and Hardware



2

Position the Toilet Seat Riser on the rim of the of the toilet bowl and center the screw holes.



3

Place the original toilet lid on top of the Toilet seat riser while aligning all the holes.

1



Remove the existing toilet seat and lid from the toilet.

Note: Most toilets will need a screwdriver. Reach underneath the toilet and hold the nut while using a screwdriver to loosen the toilet lid.

4+

Use the mounting hardware provided to attach the original lid, along with the new Toilet Seat Riser and secure with the long screws, washers, and nuts. Secure everything hand tight.

Check the fittings before using and make sure it is securely mounted and does not move.

Toilet Seat Riser Mounting Instructions: