

# THE TOILET SEAT RISER WITH ARMS



## Maintaining Your Riser:

**Check the mounts regularly to make sure they are secure  
Use mild detergent and warm water to clean Test cleaning agents on small section on inside before use**

## Warranty

Warranty: Alex DME products offer a Limited Lifetime Warranty. This warranty does not extend to non-durable components; i.e. rubber accessories and grips, which are subject to normal wear and tear. In the event of a defect covered by this warranty, we will repair or replace the device or part, at our option. Refer to your product's warranty statement on our website for specific terms and limitations.

## Assembly and Installation Instructions



1

### Assembly:

Remove product from the box and read all the instructions carefully before assembling.



2

### Assembling Handles:

**NOTE: Handles are marked Right and Left.**

- Place one handle at a time into the tubes on the side of the raised seat. Each side has a left and right mark to the corresponding handle.
- Depress the brass button on each handle and slide into position until it snaps into position and locks. (You should hear a "click" when the button pops through the locking hole.
- Repeat this process for the other side.



### To Remove Handles:

- Remove the handles, depress the brass buttons while pulling out the on the handles. Repeat for the other side.



4

Position the Toilet Seat Riser on the rim of the toilet bowl and center the screw holes.



5

Place the original toilet lid on top of the Toilet seat riser while aligning all the holes.

6

Use the mounting hardware provided to attach the original lid, along with the new Toilet Seat Riser and secure with the long screws, washers, and nuts. Secure everything hand tight.

Check the fittings before using and make sure it is securely mounted and does not move.



Included Items and Hardware

### Toilet Seat Riser Mounting Instructions:

3

Remove the existing toilet seat and lid from the toilet.

Note: Most toilets will need a screwdriver. Reach underneath the toilet and hold the nut while using a screwdriver to loosen the toilet lid.

