


Pressalit Care LS-FO Giraffe Changing Table


(Models R8671/R8672/R8673)


Installation Drawing

Call us at 800 577-1555 for installation support or technical assistance.

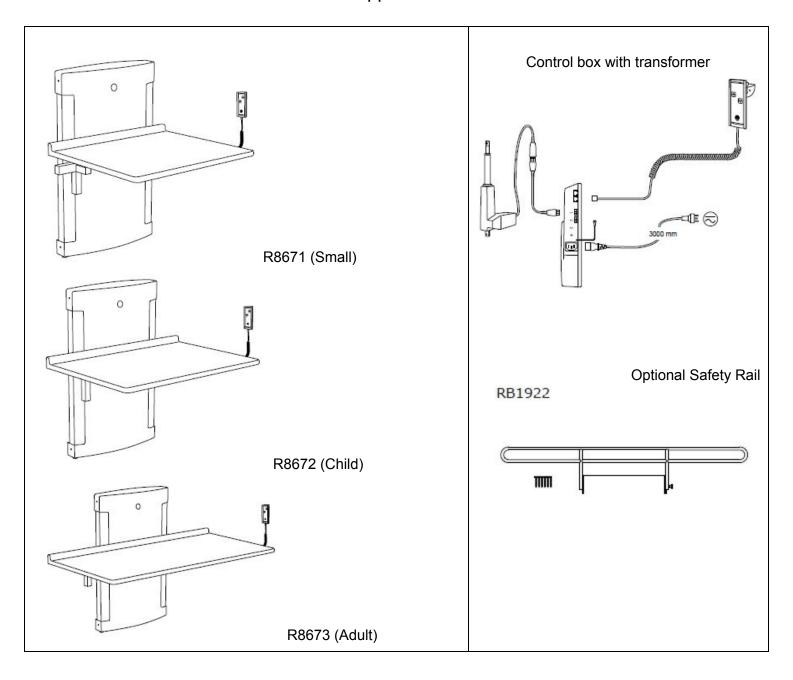




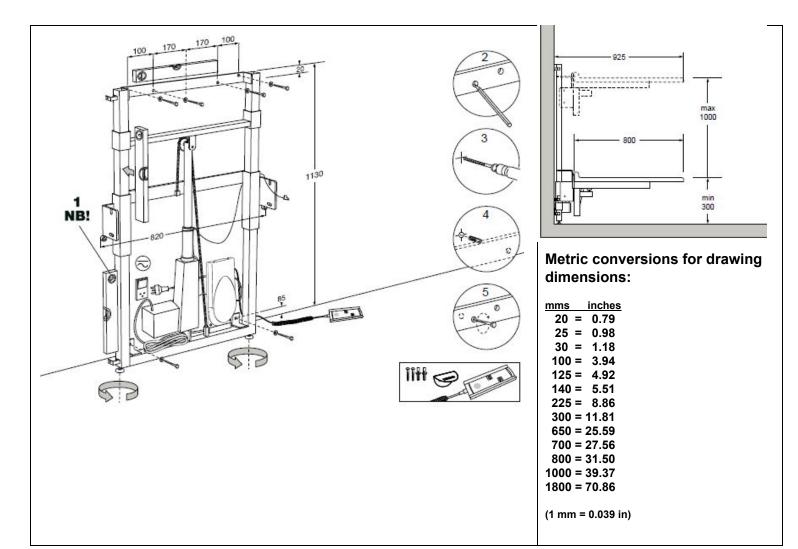


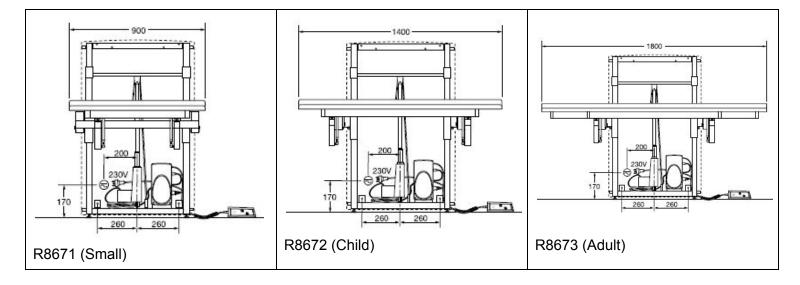



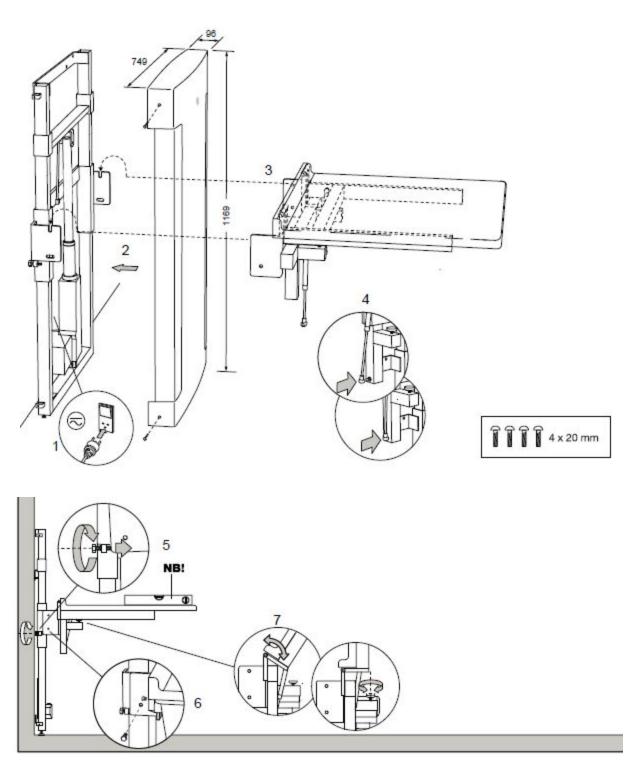



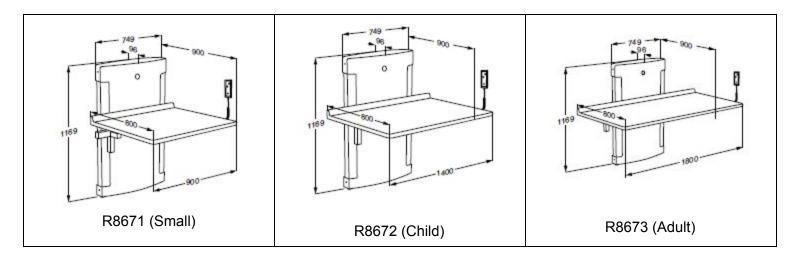

WARNING: Follow all instructions or serious injury or product damage may result. If any parts are damaged, contact MAX-Ability. Do not attempt to remove, modify, or repair damaged parts unless so instructed by MAX-Ability. Product should be installed by a qualified professional, e.g., a licensed contractor. MAX-Ability is not liable for injury or damage caused by improper mounting.

- Use indicated dimensions to mount changing table.
- Flexible drain hose may be excluded from installation if table will not be used for showering/bathing.
- Please protect wall panel components from dust during installation.

Installation location This product is designed for indoor use in a wet environment, such as a bathroom or washroom, therefore, insure that the power supply is adequately shielded. Additionally, choose an installation location with sufficient space for access to and function of the product.

Electrical Connections Insure that all power components are connected securely. Connect control box either by means of an approved moisture-resistant wall box (Fig 2.) or by leading an approved cable through the wall for connection outside the room (Fig 3.) Use a standard single gang flush mount outlet box. See the drawing for proper location of the outlet box. The wall outlet power supply is 120v 15 amp. Use GFI as required. The GFI control can be located at the wall panel or outlet. The product includes a built-in 24v/1amp transformer, 8' power cord and 3-prong wall plug. Alternatively, product can be hard-wired directly into the electrical power supply by a qualified, licensed electrician. Secure all cords/cables to protect users from tripping.

Troubleshooting Should you experience any difficulty, verify the correct current, and check the power and hand control plugs at the control box, making sure they are firmly connected. Check the operation manual for further troubleshooting tips. Please contact MAX-Ability for assistance.



