DCS™ - Plantar Fasciitis Sleeve

The DARCO® DCS™ Plantar Fasciitis Sleeve is woven to be anatomically correct. Wave compression technology on the plantar surface of the foot provides a sustained stretch on the plantar fascia. The DCS™ gives relief from the discomfort associated with plantar fasciitis by promoting circulation, reducing edema and foot

- › Wave compression technology provides a sustained stretch to the plantar fascia.
- › Comfort Zone - Special knitting pattern at the Anterior Ankle Fold to help prevent pinching and binding.
- › 24 hour support - wear while sleeping or under socks when active.
- › Constructed of a medical grade fabric that is latex free.
- › Anatomically designed for better comfort.

DCS™ Sizing (Limited U.S.A. availability)

| Size: | Arch Circ. | Women: | Men: | Product Codes: | | |
|-------|------------|----------|---------|----------------|-----------|-----------|
| | | | | Blue: | Black: | Tan: |
| S | 5 - 8 in | 4 - 6.5 | 3 - 5.5 | DCS-PF1 | DCS-PF1-B | DCS-PF1-T |
| M | 7 - 10 in | 7 - 10.5 | 6 - 9.5 | DCS-PF2 | DCS-PF2-B | DCS-PF2-T |
| L | 9 - 13 in | 11+ | 10 - 13 | DCS-PF3 | DCS-PF3-B | DCS-PF3-T |
| XL | 11 - 15 in | - | 13.5+ | DCS-PF4 | DCS-PF4-B | DCS-PF4-T |

Available in 3 colors:

Instructions for use located on inside.

Manufactured for:
DARCO® International

DARCO® (Europe) GmbH

DCS-PKG-21_Rev0321

DARCO®

Plantar Fasciitis Sleeve

DCS™

Indications

> Plantar fasciitis, heel spurs, arch pain, arch weakness, edema

Contraindications

> Poor circulation, diabetic foot/neuropathy

Cleaning Instructions/Maintenance

> When the DCS™ is slightly dirty, clean it with a damp cloth or a soft brush.
> Allow the DCS™ to air dry only.

Instructions for use

> The top opening is indicated with the DARCO® logo.
> Pull the sleeve over the foot until the heel is properly positioned in the heel pocket.

Caution

Please always follow these instructions for use. In case of pain or swelling, discontinue use and consult your medical provider immediately. This device is designed to assist recovery and should be used exclusively under the instruction of and in consultation with a healthcare professional. DARCO® International Inc. does not assume any responsibility for misuse of the product. Dressings or socks can be worn with the DCS™. Walk more slowly than normal. This device is for single-patient use only.

To the patient/user: This device has been designed and manufactured by DARCO® International, Inc. to be state of the art for treatment of a specific medical condition. Should any serious incident occur in relation to this device it should be reported to the manufacturer at the contact information listed on this document and the proper authority of the country or state where you are located. When treatment is complete, dispose of this device according to local laws and ordinances.

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