Rolyan® Thumb Hole Wrist Cock-Up Splint:

A159-1P, A159-3A1, A159-220; A160-1P, A160-3A1, A160-320; A925-900; A926-000; 5514-27; 5504-51, 5504-52; 5532-98, 5532-9801, 5532-9901; 9344-01, 9344-02; 5560-45, 5560-45-01; 5664-42 through 5664-46

A

B

C

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to all of the splints listed.

INDICATIONS

- To immobilize the wrist
- May be used for arthritis, carpal tunnel syndrome, sprains, and fractures

CONTRAINDICATIONS

- Not for the acute phase of most wrist fractures
- Not in the presence of active MCP synovitis
- Not in the presence of severe edema

PRECAUTIONS

- **This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

Splints made from Rolyan AirThru® material

- **When opening bag after heating material, avoid contact with hot vapors from bag.**

CARE AND CLEANING

- The splint will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.
- The splint can be cleaned with soap and lukewarm water and allowed to dry thoroughly before reapplication.

PROCEDURE

Splints made from AirThru material: These splint blanks fit the right or left hand. **Other splints:** The splint blank can be applied to either hand—simply invert the splint blank.

The splint blank can also be formed into a radial gutter splint—see instructions 9–14.

1. Heat splint blank as follows:

Splint blanks made from AirThru material— Soften these splints, which are packaged in a heat-sealed plastic bag, in heat pan of water at a temperature of 160° to 170°F (70°–75°C) for approximately 2 to 3 minutes. Cut open bag between heat seal and pressure-sensitive strip. If reheating is necessary, return splint blank to bag, close pressure-sensitive seal completely, and heat.

All other splints— Soften in water heated to approximately 160°F (70°C) until pliable. (Refer to the instructions for Rolyan splinting materials for information about working with

specific splinting materials.) Remove from heat pan and place on a towel on a flat surface. Blot excess water.

2. Roll back the edge of the thumb hole until it accommodates the patient's thumb.
3. If possible, place patient's hand in supination to achieve a gravity-assisted position.
4. Standing in front of the patient, insert the thumb through the thumb opening and roll the edge of the thumb hole back to clear the thenar eminence (A).
5. Mold the splint blank on the forearm as shown in A.
6. Fold back the distal edge to clear the distal palmar crease.
7. The splint can be trimmed and the edges flared if necessary.
8. Apply appropriate straps. A completed splint is shown in B.

Fabricating a radial gutter splint

9. Follow steps 1 through 3.
10. Mold the splint around the radial border of the hand and forearm (C).
11. Roll the edge of the thumb hole back to clear the thenar eminence.
12. Fold back the distal edge to clear the distal palmar crease on the volar side and the MCP joints on the dorsum.
13. The splint can be trimmed and the edges flared if necessary.
14. Apply appropriate straps.

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Patterson Medical Ltd.