

INSTRUCTION MANUAL OF PORTABLE HIGH-FREQUENCY INSTRUMENT (P-02)

The principle of High-frequency

Portable High-frequency instrument stimulate the skin through high pressure currents to speed epidermal cells fall off and it has sterilization, promote blood circulation effect and so on. Portable High-frequency instrument with very low current flow and safe for using.

- a: To stimulate the blood circulation
- b: It helps to promote glandular activity
- c: To promote the metabolism
- d: Kill the bacteria
- e: To produce heat under skin
- f : It helps the nutrition penetrate to deep skin.

Way of usage

1. Make sure the switch is turned off.
2. Connect the power supply.
3. Select the appropriate kind of electrode tube to fix at the front of the instrument.
4. Get the glass electrode pipe touched treatment body part, then slowly adjust the output energy to appropriate current output.
5. It can be treated directly in case of when using cream and mask on customer face.
6. Move slowly on the treatment parts as per your own skills
 - A: Mushroom-shaped electrode pipe –for the cheek , forehead and neck and other parts of a large area.
 - B: Spoon-shaped electrode pipe – for the jaw and other middle area of the site.
 - C: Point-shaped electrode pipe -, used to nest site area, such as the nose.
 - D: Comb-type electrode pipe - for the head or hair of customers.

The way of direct operation

- A: Beautician hand the handle of high-frequency ,do not touch the electrode pipe or contact metal objects .And then turn the knob to the minimum energy status.
- B: Turn the switch of high-frequency on. Adjust high-frequency output energy to the smallest status, and then put the glass electrode pipe on customers treatment skin area, adjust energy to higher. Sliding the glass electrode pipe from customers neck to the face, cheek, nose and forehead after customer face cared with cream or mask. Energy should be slowly adjust to desired strength in this process.
- C: Because high-frequency current generated heat in the facial skin, Use high- frequency directly can comfort and calm down the skin. At mean time to kill bacteria, it effects to oily and acne skin.
- D: When the operation is completed, slowly minimize high-frequency energy first, and turn off power switch of high-frequency. Then remove the the electrode pipe from customer face.

The way of indirect usage

Let customers holding electrode pipe, then open the power switch, and gradually adjust high-frequency energy to right intensity, then beautician give massage the customer. Beautician do not need to touch electrodes at all in the process, before take out the electrode from customers hands, it must turn off the power of high-frequency instrument first. The indirect method can strengthen and comfort skin.

Cautions

- A: If there is sparks when hand-holding electrodes pipe is less than an inch away from skin contact, energy need to be reduced. If customers feels sparks still exist, a small piece of dry cotton towel is needed to put on customers face (not use the nylon towel or other synthetic fiber towel).
- B: To prevent electric shocked, we must turn on the switch after electrode pipe touched customers face. then slowly increasing energy of high-frequency electrotherapy.
- C: Before move electrode pipe away from the skin, pls minimize the energy and turn off power of the instrument.
- D: It is necessary that lowest output status and turn off power switch before install and uninstall the electrode pipe.
- E: Every glass electrode pipe need to maintain a clean, dry status, must be sterilized before and after use.
- F: "If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard."