



# Quick Start Guide



## SITTER ELITE

1. Connect AC power cord or insert batteries\*
2. Connect sensor
3. Turn alarm on

## ADJUST PATIENT WHILE ON SENSOR

**Hold:** Quick tap of button = 30 second delay

**Suspend:** Longer tap of button (3 sec) = 5 minute delay

## PROGRAMMING

Press indicated button to advance through options. The last option or audible sample selected will be the one programmed for use.

1. **Select MODE:** 4 options (voice & tone, tone, voice, mute)
2. **Select TONE:** 5 options
3. **Select VOLUME:** 3 options (high, medium, low)
4. **Record a Message:** Press and hold RECORD; wait for beep; record message; release button

**GREEN** flash **GO** / **RED** flash **NO**

*\*If the alarm low battery alert is chirping, or the alarm does not power up, the batteries are depleted and must be removed.*





# Sitter Elite Alarm Product Details



**WARNING:** Before use, make sure to read all alarm and sensor instructions. Test alarm and sensor to verify that each function works properly before leaving a patient unattended.

A copy of the alarm manual can be downloaded at [www.posey.com](http://www.posey.com).

