**EN - CONTOURED / NARROW CONTOURED CHEST HARNESS & STANDARD BACKPACK STYLE HARNESS**

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- P-CH340XS • P-CH360XS • P-CHN360M • P-CH720XS
- P-CH340S • P-CH360S • P-CHN360L • P-CH740XS
- P-CH340M • P-CH360M • P-CH740S
- P-CH340L • P-CH360L • P-CH740M
- P-CH340XL • P-CH360XL • P-CH740L
- P-CH740XL

Thank you for choosing this **POZIFORM® POZIFLEX CHEST STRAP**

Please note that the Poziform should be used with the hardware supplied and **must not** be used as a transportation safety belt in a moving vehicle.

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**CONTENTS / CONTENIDO / CONTENU / INHALT**

EN

- |  |                    |
|--|--------------------|
| 1. Contoured or Rear Pull Backpack Style Chest Harness x 1 | 3. Upper Strap x 2 |
| 2. Lower Strap x 2   | 4. Cam Buckle x 4  |

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## EN - ASSEMBLY INSTRUCTIONS

**BEFORE** fitting the chest belt to any wheelchair/seating system, please consult the wheelchair/seating system manufacturer's manual (or contact them) to find out if it is acceptable to do so.

### Solid Backrest Installation with Cam Buckles (Direct Mounting)

The following fitting instructions are only guidelines as seating systems vary according to manufacturer.

1. Identify mounting location for all four cam buckles on the backrest. Drill holes or locate T-nut positions and fasten the cam buckles with appropriate screws or bolts (not included). See Fig 1 & 2.

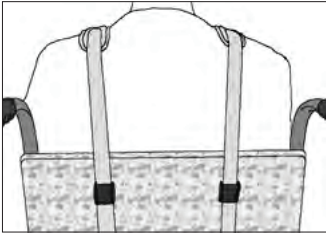
Figure 1

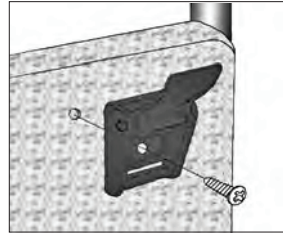
Figure 2

2. Thread upper straps through upper cam buckles and lower straps through lower cam buckles (Fig. 3).

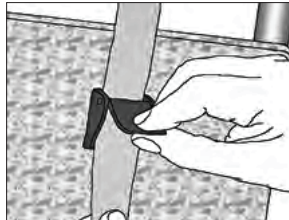
Figure 3

3. Place the harness over the user's head and shoulders. With the cam buckle open, pull the strap for a secure fit to the occupant's and close the cam buckle to lock in position. Repeat for all four straps.

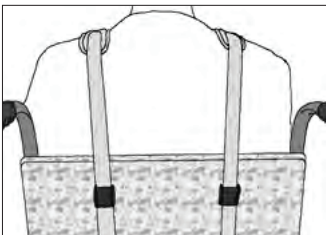
Figure 4

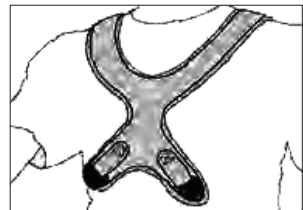
Figure 5

## EN - ASSEMBLY INSTRUCTIONS

### Cleaning Instructions

- Clean with a damp cloth and a mild detergent.
- When wet, allow to air dry.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in a washing machine.
- **DO NOT** iron.

### Maintenance Instructions

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** and make sure that all hardware is secured and not damaged, especially after making any adjustments.
- **PERIODIC** cleaning will help prevent damage to the buckles.

Every Inspired product comes with our one year manufacturer's warranty. This warranty ensures that when you receive our products, they will arrive free of defects in material and workmanship. We will repair or replace any part or entire product at our discretion for a period of up to one year from the date of delivery to the original purchaser. This warranty does not include cost of inconvenience, shipping, property damage, misuse, abuse, accident, or similar incidents. The warranty will not apply if the product has been modified, tampered with, repaired by unauthorized individuals or assembled incorrectly. Custom orders are non-refundable and non-returnable.

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© 2016 Inspired by Drive™ - A Drive DeVilbiss Healthcare Affiliate