



PREMIUM KNIT SUPPORT, ELBOW INSTRUCTIONS FOR USE

Single Patient Use Only

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE:

- Joint swelling/arthritis
- Posttraumatic irritation
- Medial and lateral epicondylitis

WARNINGS & PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

- Never wear product over an open wound.
- Do not sleep in this product.
- Do not use cream or lotions under this product.

CLEANING INSTRUCTIONS:

1. Clean material in warm water using mild soap.
 - a. Do NOT use soap with lotion.
 - b. Rinse thoroughly.
 - c. If not rinsed thoroughly, residual soap may cause irritation and affect the life of the product.
2. Lay flat and air dry ONLY.


WARRANTY:

Manufacturer guarantees all products to be free from defects in materials and workmanship for a period of 6 months from date of purchase.


PART NUMBER: 900250-XX.

| -XX | Size | Circumference 4.5" below elbow |
|-----|------|--------------------------------|
| -01 | XS | 7.50" - 8.25" |
| -03 | SM | 8.25" - 9.00" |
| -05 | MD | 9.00" - 9.75" |
| -07 | LG | 9.75" - 10.50" |
| -08 | XL | 10.50" - 11.50" |
| -09 | 2XL | 11.50" - 12.25" |

APPLICATION INSTRUCTIONS:

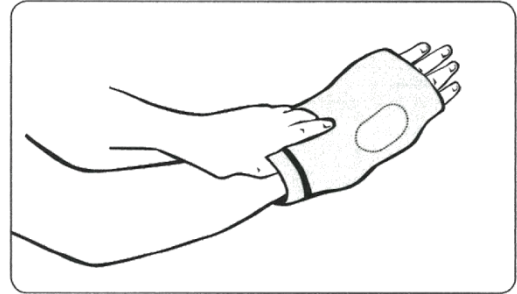

1. Pull support over hand and up arm. Pull support over elbow.

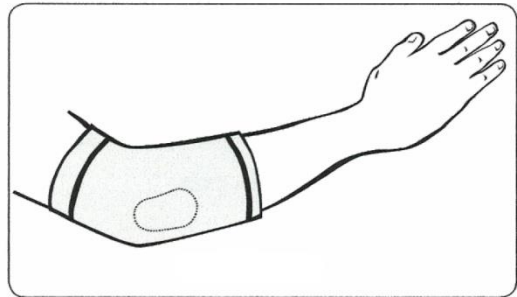

2. Place silicone pads at or just below the medial and lateral epicondyles.

To remove support:

- Pull down from lower end of support and over hand.