

THANK YOU FOR USING THE VERTALOC® PRO PLUS BACK BRACE

(MODEL# VL231LSOPP)

Your Lumbar-Sacral Orthosis (LSO) has been designed to maximize support of your lower back while providing you with the comfort you require.

YOU HAVE RECEIVED:

- ONE (1), ONE-PIECE LUMBAR-SACRAL ORTHOSIS BRACE WITH ATTACHED CLOSURE SYSTEM
- TWO(2) ABDOMINAL SUPPORT PANELS (REMOVABLE)
- CUSHIONED BACK INSERT (REMOVABLE)
- TWO(2) SIDE PANELS (NON-REMOVABLE)

FIT YOUR BRACE IN 3 EASY STEPS



1. OPEN ALL STRAPS. CENTER THE POSTERIOR PANEL OF THE BRACE IN THE MIDDLE OF LOWER BACK.



2. FASTEN THE RIGHT END OF THE ABDOMINAL SUPPORT PANEL ON THE TOP OF THE LEFT PANEL. TIGHTEN TO ALLOW OPTIMAL SUPPORT.



3. PULL THE COMPRESSION STRAPS TO ACHIEVE OPTIMAL SUPPORT. FASTEN THEM TO THE ABDOMINAL PANEL SURFACE.

CARING FOR YOUR PRO PLUS LSO: *Hand wash in warm water using mild soap. Rinse thoroughly. Lay flat to dry. Do not machine wash or dry. Do not expose to direct heat sources such as heaters, radiators, direct sunlight, etc.*

VERTALOC® PRO PLUS WARRANTY:

- ✓ YOUR VERTALOC® PRO PLUS IS WARRANTED TO BE FREE OF MATERIAL OR WORKMANSHIP DEFECTS.
- ✓ ANY DEFECTIVE COMPONENTS OF YOUR VERTALOC® PRO PLUS MAY BE REPLACED OR CREDITED AT THE DISCRETION OF VERTALOC, INC.
- ✓ THIS PRODUCT IS WARRANTED FOR ONE YEAR FROM THE DATE OF PURCHASE.

