

## Lifetime Limited Warranty

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.



## COMFORT HAIR WASH BASIN




## CONTAINS

- Inflatable Hair Wash Basin x1
- Water Bag x1
- Sprinkler x1
- Tube x1
- Pump x1

## USING STEPS

1

Use the air pump to inflate the hair wash basin properly, avoid inflating too full or insufficient. Next close the drain outlet under the hair wash basin. And then inject the warm water(under 50<sup>0</sup> C)

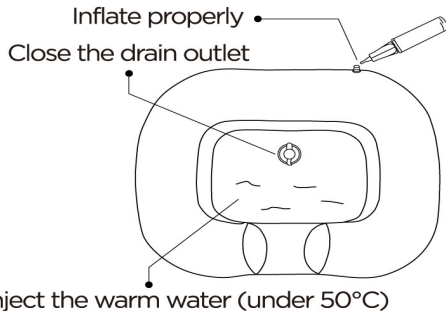


2

Connect the water bag with tube and sprinkler.


3

Inject the warm water(under 50<sup>0</sup> C) into the water bag and hang it up on infusion stand, hanger, wall or other places

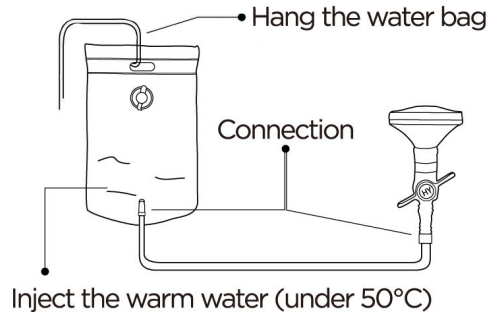

4


Prepare a bucket and then put the drain inside. Please insure the drain is without any crease.

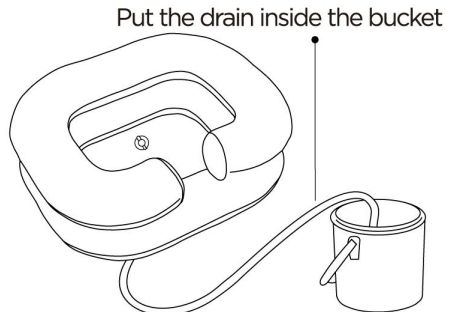

5

Adjust the water.


Open the sprinkler to adjust the water

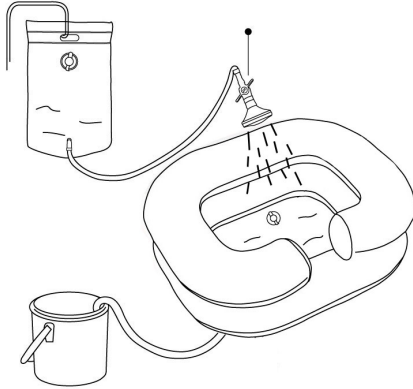

6

Drain the water.

Open the drain outlet

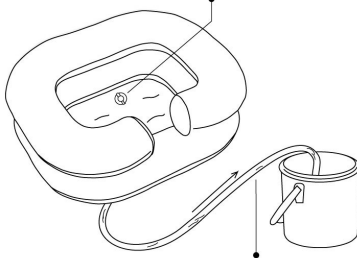

Drain away the water into the bucket