

## Lifetime Limited Warranty

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.



## Locking Raised Toilet Seat with Removable Arms 300 lbs weight capacity

### USER MANUAL



## assembly instructions

1. Ensure you inspect the product for any visual damage before use.
2. Carefully remove the existing toilet seat and lid from the toilet.
3. To install arms, you must depress the push pins on the arms and insert them into the mounting tubes. Ensure they are locked in place. To remove the arms, reverse this step.
4. Carefully place the seat onto the toilet bowl. Position the raised toilet seat correctly on the rim of the toilet ensuring the front lip is outside the bowl. (see fig. 1).

fig. 1



5. Ensure the front adjustment bracket clears the rim so that the toilet seat fits securely.

6. To tighten, turn the hand wheel clockwise (fig. 2).

fig. 2



7. Check the stability before use.
8. To remove, simply unscrew the hand wheel and remove the raised toilet seat by lifting up.