

Lifetime Limited Warranty

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.



2" Raised Toilet Seat with Arms

Weight Capacity: 300lbs



INSTRUCTIONS INSIDE

MOBB Healthcare Ltd.

116A Viceroy Rd. Unit 5-6
Concord, Ontario, Canada
L4K 2M1

CARE AND CLEANING

- REGULAR CLEANING IS RECOMMENDED
- USE A NON-ABRASIVE CLEANER OR DETERGENT. RINSE WITH WARM WATER AND DRY COMPLETELY.
- ABRASIVE CLOTHS MAY DAMAGE THE SURFACE
- PLEASE REGULARLY INSPECT YOUR RAISED TOILET SEAT WITH ARMS AND IT'S PARTS FOR ANY CORROSION, BREAKAGE, COMPRESSION OR WEAR.

ASSEMBLY and INSTALLATION

1. Remove all parts from the box:
 - One Seat Riser, two padded arms, two mounting screws, two metal washers and two wing nuts.
2. To attach the arms to the seat riser, align each arm with the seat riser tubing. The longer side of the arm should be facing the front of the seat riser. Press the lock buttons on the arm tubes and insert the arm tubing into the seat riser tubing. You should hear a click when the lock buttons are through the first tubing holes (the second tubing hole on the seat riser is for assembly purposes only).
3. Remove the existing toiletseat and lid from the toilet frame by unscrewing the seat screws located at the rear of the toilet seat.
4. Place the seat riser on the toilet frame. The mounting holes on the rear of the seat riser should line up with the mounting holes on the rear of the toilet frame.
5. Place the toilet seat on the seat riser and align mounting holes with toilet frame. Place the metal washers onto the mounting screws, and place into mounting holes of toilet seat, the seat


ASSEMBLY and INSTALLATION

riser and toilet frame. Then place the plastic washers and the wing nuts onto the mounting screws. Do NOT tighten the wing nuts completely.

6. Be sure the toilet seat is straight and sits flush on the seat riser, then tighten the wing nuts.



SEAT RISER

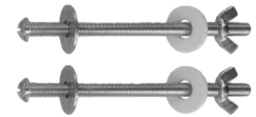

MOUNTING SCREWS, METAL WASHERS & WING NUTS



LEFT AND RIGHT PADDED ARMS